

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
HLA-B allele ^{1,2,3}																				
*49:01:01-49:01:07, 49:25, 49:28-49:30, 49:32-49:35	1				5	6														
*49:02		2	3		5	6														
*49:03				4	5	6										16				
*49:04:01	1						7												19	
*49:04:02	1						7	8											19	
*49:05	1						7	8												
*49:06	1	2			5	6														
*49:07, 49:21 ⁴	1				5	6			9											
*49:08, 49:16 ⁴	1				5	6				10										
*49:09	1			4	5	6														
*49:10, 49:24, 49:31	1				5	6						12								
*49:11	1				5	6							13							
*49:12	1				5	6								14						
*49:13-49:14 ⁴	1				5	6									15					
*49:17	1				5	6								14	15					
*49:18	1															16				
*49:19N	1				5	6		8						14						
*49:20	1				5	6	7													
*49:22	1		3		5	6														
*49:23	1				5	6												17		
*49:26		2			5	6														
*49:27	1				5	6													18	
*07:78, 13:31, 13:41, 15:303, 54:16, 54:20, 54:33, 55:09, 55:21, 55:46, 55:52, 55:56, 56:01:05, 56:13, 56:43					5			8												
*07:84, 13:35, 40:44, 41:21, 51:90, 52:12								8											19	
*08:03, 08:52, 08:78, 37:34, 38:30, 44:06, 53:22, 57:09, 57:24											11									
*08:09, 08:84, 13:59, 13:62, 15:83, 27:14, 27:81, 27:130, 40:06:01:01-40:06:11, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:93, 40:95, 40:103, 40:127, 40:131, 40:148, 40:161-40:162, 40:165, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268-40:270, 40:275, 40:294-40:295, 40:298, 41:01:01-41:01:03, 41:05-41:07, 41:09, 41:12, 41:14, 41:16-41:17, 41:20, 41:22, 41:25-41:26, 41:28-41:29, 41:32, 41:35, 42:04, 50:20, 51:01:01:01-51:01:18, 51:01:20-51:03, 51:05, 51:07:02, 51:09:01-51:12, 51:14, 51:16, 51:18-51:19, 51:21-51:22, 51:24:01-51:24:05, 51:26-51:35, 51:38-51:41N, 51:43, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:80, 51:82-51:89, 51:91, 51:93, 51:95-51:96, 51:98N-51:105, 51:107, 51:109-51:111, 51:113-51:125, 51:127-51:129, 51:131-51:134, 51:136-51:138, 51:140, 51:142-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01, 51:159-51:161, 51:163-51:171, 51:173Q-51:176, 51:178N-51:185, 52:01:01:01-52:01:01:03, 52:01:03, 52:01:05-52:01:08, 52:01:10-52:01:21, 52:02:02, 52:04-52:08, 52:10:01-52:10:02, 52:11, 52:13, 52:16-52:18, 52:20, 52:22-52:24, 52:26-52:31:01, 52:32, 52:34-52:38, 52:40, 52:42-52:44, 54:04, 54:11, 55:17, 55:20, 55:28, 56:05:01-56:06, 56:15, 56:21, 58:08:01-58:08:02, 59:02, 78:01:01-78:08								8												
*13:01:01-13:01:09, 13:06-13:07N, 13:11-13:12:02, 13:17, 13:22:01-13:22:02, 13:25, 13:28-13:29, 13:36, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N-13:64, 13:76N-13:78, 13:80, 13:83, 15:86, 15:224, 40:48, 40:71, 56:02, 56:04, 56:10						6							13							
*13:02:01-13:02:05, 13:02:07-13:02:14, 13:02:16-13:03, 13:08-13:09, 13:14, 13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44-13:45, 13:49N, 13:53-13:54, 13:56:01N-13:56:02N, 13:58, 13:65-13:70, 13:75, 13:81-13:82, 35:60, 46:11, 46:18, 51:15, 51:157, 54:01:01-54:03, 54:05N, 54:07-54:08N, 54:12-54:13, 54:17-54:19, 54:21-54:30, 54:32, 54:34, 55:02:01:01-55:02:08, 55:07, 55:10, 55:12, 55:16, 55:18-55:19, 55:22, 55:26, 55:30, 55:34-55:35, 55:41, 55:43, 55:47-55:48, 55:50, 55:62-55:63, 55:65, 55:67, 55:69-55:70, 56:01:01:01-56:01:04, 56:01:06-56:01:08, 56:07-56:08, 56:14, 56:16-56:17, 56:19N-56:20:02, 56:23-56:24, 56:26-56:29, 56:33, 56:35-56:36, 56:38N-56:42, 56:44-56:45, 59:01:01:01-59:01:01:02, 59:04-59:08					5	6		8					13							
*13:02:06, 13:74					5	6							13							
*13:02:15, 13:47, 54:31, 55:39, 55:42, 55:61					6			8					13							
*13:04, 15:04:01-15:04:02, 15:16:01-15:16:02, 15:67, 15:95, 15:155, 15:222, 15:254, 15:293, 15:310, 35:265, 40:159, 46:32, 51:37, 54:14, 55:23, 55:68							7	8											19	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
*13:15, 56:34				4	5	6		8					13							
*13:16, 13:48, 15:42, 55:01:01-55:01:03, 55:01:05-55:01:06, 55:01:08-55:01:14, 55:03, 55:05, 55:11, 55:15, 55:24-55:25, 55:29, 55:31, 55:33, 55:36, 55:38, 55:40, 55:45, 55:54-55:55N, 55:57, 55:59-55:60, 55:64, 55:66, 56:25, 56:30, 56:37					5	6		8												
*13:18, 54:10, 55:37					5			8					w							
*13:20, 39:17, 50:12, 50:16-50:17, 50:41-50:42, 55:01:07, C*15:91						6														
*13:23		2				6							13							

Negative Control

[illegible]