

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
HLA-B allele ^{1,2,3}																				
*49:01:01-49:01:08, 49:25, 49:28-49:30, 49:32-49:36, 49:38-49:39	1				5	6														
*49:02		2	3		5	6														
*49:03				4	5	6										16				
*49:04:01	1						7												19	
*49:04:02	1						7	8											19	
*49:05	1						7	8												
*49:06	1	2			5	6														
*49:07, 49:21 ⁴	1				5	6			9											
*49:08, 49:16 ⁴	1				5	6				10										
*49:09	1			4	5	6														
*49:10, 49:24, 49:31	1				5	6						12								
*49:11	1				5	6							13							
*49:12	1				5	6								14						
*49:13-49:14 ⁴	1				5	6									15					
*49:17	1				5	6								14	15					
*49:18	1															16				
*49:19N	1				5	6		8						14						
*49:20	1				5	6	7													
*49:22	1		3		5	6														
*49:23	1				5	6												17		
*49:26		2			5	6														
*49:27	1				5	6													18	
*49:37	1				5															
*07:78, 13:31, 13:41, 15:303, 54:16, 54:20, 54:33, 55:09, 55:21, 55:46, 55:52, 55:56, 56:01:05, 56:13, 56:43					5			8												
*07:84, 13:35, 40:44, 41:21, 51:90, 52:12								8												19
*08:03, 08:52, 08:78, 37:34, 38:30, 44:06, 53:22, 57:09, 57:24											11									
*08:09, 08:84, 13:59, 13:62, 15:83, 27:14, 27:81, 27:130, 40:06:01:01-40:06:12, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:93, 40:95, 40:103, 40:127, 40:131, 40:148, 40:161-40:162, 40:165, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268-40:270, 40:275, 40:294-40:295, 40:298, 40:306-40:307, 41:01:01-41:01:04, 41:05-41:07, 41:09, 41:12, 41:14, 41:16-41:17, 41:20, 41:22, 41:25-41:26, 41:28-41:29, 41:32, 41:35, 41:37, 42:04, 50:20, 51:01:01:01-51:01:18, 51:01:20-51:03, 51:05, 51:07:02, 51:09:01-51:12, 51:14, 51:16, 51:18-51:19, 51:21-51:22, 51:24:01-51:24:05, 51:26-51:35, 51:38-51:41N, 51:43, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:80, 51:82-51:89, 51:91, 51:93, 51:95-51:96, 51:98N-51:105, 51:107, 51:109-51:111, 51:113-51:125, 51:127-51:129, 51:131-51:134, 51:136-51:138, 51:140, 51:142-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01, 51:159-51:161, 51:163-51:171, 51:173Q-51:176, 51:178N-51:190, 52:01:01:01-52:01:01:03, 52:01:03, 52:01:05-52:01:08, 52:01:10-52:01:24, 52:01:27-52:01:28, 52:02:02, 52:04-52:08, 52:10:01-52:10:02, 52:11, 52:13, 52:16-52:18, 52:20, 52:22-52:24, 52:26-52:31:01, 52:32, 52:34-52:38, 52:40, 52:42-52:45, 52:47-52:50, 54:04, 54:11, 55:17, 55:20, 55:28, 56:05:01-56:06, 56:15, 56:21, 58:08:01-58:08:02, 59:02, 78:01:01-78:08								8												
*13:01:01-13:01:10, 13:06-13:07N, 13:11-13:12:02, 13:17, 13:22:01-13:22:02, 13:25, 13:28-13:29, 13:36, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N-13:64, 13:76N-13:78, 13:80, 13:83, 15:86, 15:224, 40:48, 40:71, 56:02, 56:04, 56:10						6							13							
*13:02:01-13:02:05, 13:02:07-13:02:14, 13:02:16-13:03, 13:08-13:09, 13:14, 13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44-13:45, 13:49N, 13:53-13:54, 13:56:01N-13:56:02N, 13:58, 13:65-13:70, 13:75, 13:81-13:82, 13:84, 35:60, 46:11, 46:18, 51:15, 51:157, 54:01:01-54:03, 54:05N, 54:07-54:08N, 54:12-54:13, 54:17-54:19, 54:21-54:30, 54:32, 54:34, 55:02:01:01-55:02:09, 55:07, 55:10, 55:12, 55:16, 55:18-55:19, 55:22, 55:26, 55:30, 55:34-55:35, 55:41, 55:43, 55:47-55:48, 55:50, 55:62-55:63, 55:65, 55:67, 55:69-55:71, 56:01:01:01-56:01:04, 56:01:06-56:01:08, 56:07-56:08, 56:14, 56:16-56:17, 56:19N-56:20:02, 56:23-56:24, 56:26-56:29, 56:33, 56:35-56:36, 56:38N-56:42, 56:44-56:49, 59:01:01:01-59:01:01:02, 59:04-59:09					5	6		8					13							
*13:02:06, 13:74					5	6							13							
*13:02:15, 13:47, 54:31, 55:39, 55:42, 55:61						6		8					13							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
*13:04, 15:04:01-15:04:02, 15:16:01-15:16:02, 15:67, 15:95, 15:155, 15:222, 15:254, 15:293, 15:310, 15:362, 35:265, 40:159, 46:32, 51:37, 54:14, 55:23, 55:68							7	8												19
*13:15, 56:34				4	5	6		8					13							
*13:16, 13:48, 15:42, 55:01:01-55:01:03, 55:01:05-55:01:06, 55:01:08-55:01:14, 55:03, 55:05, 55:11, 55:15, 55:24-55:25, 55:29, 55:31, 55:33, 55:36, 55:38, 55:40, 55:45, 55:54-55:55N, 55:57, 55:59-55:60, 55:64, 55:66, 55:73-55:75, 56:25, 56:30, 56:37					5	6		8												
*13:18, 54:10, 55:37					5			8					w							

Negative Control

*13:20, 39:17, 50:12, 50:16-50:17, 50:41-50:42, 55:01:07, C*15:91						6														
*13:23	2					6						13								
*13:46, 15:137, 35:276, 39:06:01-39:06:05, 39:33-39:34, 39:50, 39:57, 39:62, 39:64, 39:83, 39:90, 39:102, 40:86, 41:30, 51:13:01-51:13:02, 51:23, 51:63, 51:108, 51:172, 52:14-52:15, 52:39, 55:13, 55:32, 55:58, 56:22, 59:03, 73:01-73:02							7	8												
*13:55	2				5	6		8				13								
*13:72, 15:16:03, 35:37, 45:09							7											19		
*13:73												w								
*14:37				4	5	6														
*15:32:02, 18:04, 40:01:11				4													16			
*15:33, 15:248, 44:02:32, 44:03:16, 44:07, 44:49, 44:81, 44:111, 44:145, 44:194, 44:200, 57:54, C*03:87:01-03:87:02, C*05:27, C*05:39, C*08:115			3																	
*15:46, 15:53, 15:106, 18:107, 35:47, 35:154, 40:10:01:01-40:10:02, 40:40, 40:49, 40:129, 40:164, 40:210, 40:219, 40:229, 44:02:13, 44:02:19, 44:05:03, 44:09, 44:11, 44:46, 44:50:02, 44:75, 44:90, 44:129, 44:152, 44:170, 44:196, 44:201, 47:01:01:01-47:03, 47:05-47:10											12									
*18:09, 27:01, 27:142, 37:10, 39:31, 58:13	2																			
*35:01:10, 35:04:02, 40:28, 50:14, 51:56:01-51:56:03, 53:28, C*02:02:23, C*12:03:31																16				
*35:235							7											w		
*40:13, 40:19, 40:117, 40:292, 53:39, 57:51	1																			
*40:47, 40:157, 40:201, 44:55, 44:103, 44:188, 44:212		2	3																	
*40:58				4								12				16				
*40:96		2	3					8												
*40:109, 51:112	1							8												
*40:110			3					8			12									
*41:33								8			12									
*44:02:01:01-44:02:09, 44:02:11-44:02:12, 44:02:14-44:02:18, 44:02:20-44:02:31, 44:02:33-44:03:15, 44:03:17-44:05:02, 44:05:04, 44:12-44:14, 44:16-44:17, 44:19N, 44:21-44:24, 44:26-44:33, 44:34:02-44:39, 44:41:01-44:43:02, 44:45, 44:48, 44:51-44:54, 44:56N, 44:58N-44:59:02, 44:61N-44:74, 44:76-44:80, 44:82, 44:84:01-44:89, 44:91-44:94, 44:96-44:99, 44:101-44:102, 44:104-44:110, 44:112-44:128, 44:132-44:133, 44:135-44:144, 44:146-44:151, 44:153-44:155, 44:157-44:169, 44:171N-44:187, 44:189-44:193, 44:195N, 44:198N-44:199, 44:202-44:209, 44:211, 44:213-44:223, 47:04, C*16:85				3							12									
*44:02:10, 44:34:01			3	4							12					16				
*44:10			3			6					12	13								
*44:15		2	3		5		7													
*44:18	1				5		7				11									
*44:20, 44:47, 44:100, 44:197			3				7	8			12									
*44:25, 44:95	1										11	12								
*44:50:01, 53:30, 57:45, 57:69	1										12									
*45:01:01-45:01:02, 45:03, 45:05-45:07, 45:11-45:13, 45:15-45:16					5		7													
*45:02, 50:09, 50:33							7													
*45:04, 45:08, 50:39					5															
*45:10					5				9											
*45:14					5		7	8												
*50:01:01-50:01:07, 50:04-50:08, 50:10-50:11, 50:15, 50:18-50:19, 50:32, 50:34-50:38, 50:40					5	6														
*50:02					5	6	7													
*50:13, 50:43					5	6					12									
*50:31					5	6			9											
*51:07:01, 52:01:02, 52:01:04, 52:01:09, 52:02:01, 52:03, 52:09, 52:10:03, 52:21, 52:31:02				4				8									16			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
*51:08:01-51:08:02, 51:20, 51:36, 51:44N, 51:141, 51:153, 52:19								8			11									
*51:62				4		6														
*51:92:01-51:92:02							7	8											w	
*51:97							7	8			11									
*51:106				4		6		8												
*51:126			3					8												
*51:130								8						14						
*51:158:02								8								16				
*51:177								8						14	15					
*52:25				4	5	6		8												
*52:33								8		10										
*55:01:04					5	6		8								16				
*55:27																			19	
*55:44, 55:53						6		8												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20

Negative Control

Negative Control