

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
<b>HLA-B allele<sup>1,2,3</sup></b>																					
*49:01:01:01-49:01:10, 49:25, 49:28-49:30, 49:32-49:36, 49:38-49:40, 49:42-49:46, 49:48-49:49	1				5	6															
*49:02		2	3		5	6															
*49:03				4	5	6										16					
*49:04:01	1						7													19	
*49:04:02	1						7	8												19	
*49:05	1						7	8													
*49:06	1	2			5	6															
*49:07, 49:21 <sup>4</sup>	1				5	6			9												
*49:08, 49:16 <sup>4</sup>	1				5	6			10												
*49:09	1			4	5	6															
*49:10, 49:24, 49:31	1				5	6						12									
*49:11	1				5	6							13								
*49:12	1				5	6								14							
*49:13-49:14 <sup>4</sup>	1				5	6									15						
*49:17	1				5	6								14	15						
*49:18:01-49:18:02	1																				20
*49:19N	1				5	6		8						14							
*49:20	1				5	6	7														
*49:22	1		3		5	6															
*49:23	1				5	6														17	
*49:26		2			5	6															
*49:27	1				5	6														18	
*49:37, 49:41	1				5																
*49:47	1					6															
*49:50	1						7														
*07:78, 13:31, 13:41, 15:303, 54:16, 54:20, 54:33, 55:09, 55:21, 55:46, 55:52, 55:56, 56:01:05, 56:13, 56:43					5			8													
*07:84, 13:35, 40:44, 51:90, 52:12								8													19
*08:03, 08:52, 08:78, 37:34, 38:30, 44:06, 53:22, 57:09, 57:24										11											
*08:09, 08:84, 13:59, 13:62, 15:83, 27:14, 27:81, 27:130, 27:153, 40:06:01:01-40:06:16, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:93, 40:95, 40:103, 40:127, 40:131, 40:148, 40:161-40:162, 40:165, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268-40:270, 40:275, 40:294-40:295, 40:298:01-40:298:02, 40:306-40:307, 40:311, 40:313, 40:317-40:318, 40:341, 40:348-40:350, 40:354, 40:361N-40:364, 41:44, 42:04, 51:01:01:01-51:01:18, 51:01:20-51:03, 51:05, 51:07:02, 51:09:01-51:12, 51:14, 51:16, 51:18-51:19, 51:21-51:22, 51:24:01-51:24:05, 51:26-51:35, 51:38-51:41N, 51:43, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:80, 51:82-51:89, 51:91, 51:93, 51:95-51:96, 51:98N-51:105, 51:107, 51:109-51:111, 51:113-51:125, 51:127-51:129, 51:131-51:134, 51:136-51:138, 51:140, 51:142-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01-51:161, 51:163-51:171, 51:173Q-51:176, 51:178N-51:193, 51:195-51:209, 51:211-51:212, 51:215, 51:217-51:219, 51:221, 51:224, 51:226, 51:228-51:234, 52:01:01:01-52:01:01:05, 52:01:03, 52:01:05-52:01:08, 52:01:10-52:01:30, 52:02:02, 52:04-52:08, 52:09:02-52:10:02, 52:11, 52:13, 52:16-52:18, 52:20, 52:21:02-52:24, 52:26-52:31:01, 52:32, 52:34-52:38, 52:40, 52:42-52:74, 54:04, 54:11, 55:17, 55:20, 55:28, 55:81, 56:05:01-56:06, 56:15, 56:21, 58:08:01-58:08:02, 59:02, 78:01:01:01-78:09, C*14:92								8													
*13:01:01:01-13:01:13, 13:06-13:07N, 13:11-13:12:02, 13:17, 13:22:01-13:22:02, 13:25, 13:28-13:29, 13:36, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N-13:64, 13:76N-13:78, 13:80, 13:83, 13:87, 13:90, 13:92, 13:95, 13:106-13:107, 13:109, 15:86, 15:224, 40:48, 40:71, 54:38, 56:02:01-56:02:02, 56:04:01-56:04:02, 56:10, 58:73						6							13								
*13:02:01:01-13:02:05, 13:02:07-13:02:14, 13:02:16-13:03, 13:08-13:09, 13:14, 13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44-13:45, 13:49N, 13:53-13:54, 13:56:01N-13:56:02N, 13:58, 13:65-13:70, 13:75, 13:81-13:82, 13:84-13:85, 13:88, 13:91, 13:93-13:94, 13:96-13:105, 35:60, 46:11, 46:18, 51:15, 51:157, 54:01:01-54:03, 54:05N, 54:07-54:08N, 54:12-54:13, 54:17-54:19, 54:21-54:30, 54:32, 54:34-54:37, 55:02:01:01-55:02:10, 55:07, 55:10, 55:12, 55:16, 55:18-55:19, 55:22, 55:26, 55:30, 55:34-55:35, 55:41, 55:43, 55:47-55:48, 55:50, 55:62-55:63, 55:65, 55:67, 55:69-55:72, 55:77, 55:80, 55:82-55:83N, 56:01:01:01-56:01:04, 56:01:06-56:01:11, 56:07-56:08, 56:14, 56:16-56:17, 56:19N-56:20:02, 56:23-56:24, 56:26-56:29, 56:33, 56:35-56:36, 56:38N-56:42, 56:44-56:49, 56:52-56:55:01:02, 59:01:01:01-59:01:01:02, 59:04-59:10N					5	6		8					13								
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>
*13:02:06, 13:74					5	6							13								
*13:02:15, 13:47, 54:31, 55:39, 55:42, 55:61						6		8					13								

Negative Control



