

Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
HLA-B allele ^{1,2,3}																						
*49:01:01:01-49:01:13, 49:25, 49:28-49:30, 49:32-49:36, 49:38-49:40, 49:42-49:46, 49:48-49:49, 49:51-49:59	1					5	6															
*49:02			2	3		5	6															
*49:03					4	5	6										16					
*49:04:01	1							7												19		
*49:04:02	1							7	8											19		
*49:05	1							7	8													
*49:06	1	2				5	6															
*49:07, 49:21 ⁴	1					5	6			9												
*49:08, 49:16 ⁴	1					5	6				10											
*49:09	1				4	5	6															
*49:10, 49:24, 49:31	1					5	6						12									
*49:11	1					5	6							13								
*49:12	1					5	6								14							
*49:13-49:14 ⁴	1					5	6									15						
*49:17	1					5	6								14	15						
*49:18:01-49:18:02	1																				20	
*49:19N	1					5	6		8						14							
*49:20	1					5	6	7														
*49:22	1			3		5	6															
*49:23	1					5	6													17		
*49:26		2				5	6															
*49:27	1					5	6													18		
*49:37, 49:41	1					5																
*49:47	1						6															
*49:50	1							7														
*07:78, 13:31, 13:41, 13:128, 15:303, 54:16, 54:20, 54:33, 55:09, 55:21, 55:46, 55:52, 55:56, 56:01:05, 56:13, 56:43						5			8													
*07:84, 13:35, 40:44, 51:90, 52:12									8											19		
*08:03, 08:52, 08:78, 37:34, 38:30, 44:06, 53:22, 57:09, 57:24												11										
*08:09, 08:84, 13:59, 13:62, 15:83, 27:14, 27:81, 27:130, 27:153, 40:06:01:01-40:06:18, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:93, 40:95, 40:103, 40:127, 40:131, 40:148, 40:161-40:162, 40:165, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268-40:270, 40:275, 40:294-40:295, 40:298:01-40:298:02, 40:306-40:307, 40:311, 40:313, 40:317-40:318, 40:341, 40:348-40:350, 40:354, 40:361N-40:364, 40:373-40:374, 40:376, 40:387-40:389, 40:408, 40:411, 41:44, 42:04, 51:01:01:01-51:01:18, 51:01:20-51:03, 51:05, 51:07:02, 51:09:01-51:12, 51:14, 51:16, 51:18-51:19, 51:21-51:22, 51:24:01-51:24:05, 51:26-51:35, 51:38-51:41N, 51:43, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:80, 51:82-51:89, 51:91, 51:93, 51:95-51:96, 51:98N-51:105, 51:107, 51:109-51:111, 51:113-51:125, 51:127-51:129, 51:131-51:134, 51:136-51:138, 51:140, 51:142-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01-51:161, 51:163-51:171, 51:173Q-51:176, 51:178N-51:193, 51:195-51:204:01, 51:205-51:209, 51:211-51:212, 51:215, 51:217-51:219, 51:221, 51:224, 51:226, 51:228-51:250, 51:252-51:262, 51:264N-51:267, 52:01:01:01-52:01:01:06, 52:01:03, 52:01:05-52:01:08, 52:01:10-52:01:36, 52:02:02, 52:04-52:08, 52:09:02-52:10:02, 52:11, 52:13, 52:16-52:18, 52:20, 52:21:02-52:24, 52:26-52:31:01, 52:32, 52:34-52:38, 52:40, 52:42-52:74, 52:76-52:77, 52:79-52:84, 54:04, 54:11, 55:17, 55:20, 55:28, 55:81, 56:05:01-56:06, 56:15, 56:21, 58:08:01-58:08:02, 59:02, 78:01:01:01-78:10, C*14:92								8														
*13:01:01:01-13:01:14, 13:06-13:07N, 13:11-13:12:02, 13:17, 13:22:01-13:22:02, 13:25, 13:28-13:29, 13:36, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N-13:64, 13:76N-13:78, 13:80, 13:83, 13:87, 13:90, 13:92, 13:95, 13:106-13:107, 13:109, 13:122, 15:86, 15:224, 40:48, 40:71, 40:377, 54:38, 56:02:01-56:02:02, 56:04:01-56:04:02, 56:10, 56:56-56:57, 56:60, 58:73						6								13								
*13:02:01:01-13:02:05, 13:02:07-13:02:14, 13:02:16-13:02:26, 13:02:28-13:03, 13:08-13:09, 13:14, 13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44-13:45, 13:49N, 13:53-13:54, 13:56:01N-13:56:02N, 13:58, 13:65-13:70, 13:75, 13:81-13:82, 13:84-13:85, 13:88, 13:91, 13:93-13:94, 13:96-13:105, 13:111-13:117, 13:119-13:120, 13:123Q-13:126, 13:130, 15:485, 35:60, 46:11, 46:18, 51:15, 51:157, 54:01:01-54:03, 54:05N, 54:07-54:08N, 54:12-54:13, 54:17-54:19, 54:21-54:30, 54:32, 54:34-54:37, 55:02:01:01-55:02:10, 55:07, 55:10, 55:12, 55:16, 55:18-55:19, 55:22, 55:26, 55:30, 55:34-55:35, 55:41, 55:43, 55:47-55:48, 55:50, 55:62-55:63, 55:65, 55:67, 55:69-55:72, 55:77, 55:80, 55:82-55:83N, 55:88-55:89N, 55:96, 56:01:01:01-56:01:04, 56:01:06-56:01:15, 56:07-56:08, 56:14, 56:16-56:17, 56:19N-56:20:02, 56:23-56:24, 56:26-56:29, 56:33, 56:35-56:36, 56:38N-56:42, 56:44-56:49, 56:52-56:55:01:02, 56:58, 56:62, 56:64, 59:01:01:01-59:01:01:02, 59:04-59:10N						5	6		8					13								
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.		1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21

Negative Control

*13:02:06, 13:02:27, 13:74, 13:118, 13:121					5	6								13					
*13:02:15, 13:47, 54:31, 55:39, 55:42, 55:61						6		8						13					
*13:04, 15:04:01:01-15:04:04, 15:16:01:01-15:16:02, 15:67, 15:95, 15:155, 15:222, 15:254, 15:293, 15:310, 15:362, 15:408, 35:265, 40:159, 46:32, 51:37, 54:14, 55:23, 55:68, 56:50								7	8									19	
*13:15, 56:34					4	5	6		8					13					
*13:16, 13:48, 13:89, 15:42, 55:01:01:01-55:01:06, 55:01:08-55:01:16, 55:03, 55:05, 55:11, 55:15, 55:24-55:25, 55:29, 55:31, 55:33, 55:36, 55:38, 55:40, 55:45, 55:54-55:55N, 55:57, 55:59-55:60, 55:64, 55:66, 55:73-55:76, 55:78-55:79, 55:84-55:85, 55:90-55:93, 55:95, 55:97N, 56:25, 56:30, 56:37, 56:59							5	6		8									
*13:18, 54:10, 55:37							5			8				w					
*13:20, 13:110, 39:17, 50:12, 50:16-50:17, 50:41-50:42, 50:51, 55:01:07, C*07:536, C*15:91, C*15:128								6											
*13:23			2					6						13					
*13:46, 15:137, 35:276, 39:06:01-39:06:06, 39:33-39:34, 39:50, 39:57, 39:62, 39:64, 39:83, 39:90, 39:102, 39:109, 39:127, 39:129, 39:132, 39:140, 40:86, 40:330, 41:30, 41:50, 51:13:01-51:13:02, 51:23, 51:63:01-51:63:02, 51:108, 51:172, 51:216, 52:14:01-52:15, 52:39, 55:13, 55:32, 55:58, 56:22, 59:03, 73:01-73:02										7	8								
*13:55			2				5	6		8				13					
*13:72, 15:16:03, 35:37, 45:09									7									19	
*13:73, 56:51, 56:63														w					
*13:127, 45:02, 50:09, 50:33, C*05:152, C*07:463, C*12:183									7										
*13:129									6					12	13				
*14:37, 50:58					4	5	6												
*15:32:02, 18:04:01, 18:125, 40:01:11					4													16	
*15:33, 15:248, 44:02:32, 44:03:16, 44:03:30, 44:07, 44:27:03, 44:49, 44:81, 44:111, 44:145, 44:194, 44:200, 44:225, 44:293, 57:54, 57:96, C*03:87:01-03:87:02, C*03:414, C*05:27, C*05:39, C*05:184, C*08:115							3												
*15:46, 15:53, 15:106, 18:107, 35:47, 35:154, 40:10:01:01-40:10:02, 40:40, 40:49, 40:129, 40:164, 40:210, 40:219, 40:229, 40:325, 40:399N, 41:47, 44:02:13, 44:02:19, 44:02:45, 44:03:37, 44:05:03, 44:09, 44:11, 44:46, 44:50:02, 44:75, 44:90, 44:129, 44:152, 44:170, 44:196, 44:201, 44:254, 44:257, 44:277, 47:01:01:02-47:03, 47:05-47:10														12					
*15:406																		18	
*18:09, 27:01, 27:142, 37:10, 39:31:01:01-39:31:01:02, 39:128, 58:13					2														
*35:235									7									w	
*39:141									7	8					14				
*40:13, 40:19, 40:117, 40:292, 40:394, 53:39, 57:51			1																
*40:47, 40:157, 40:201, 44:55, 44:103, 44:188, 44:212				2	3														
*40:58						4			</										

[illegible]