

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
HLA-B allele ^{1,2,3}																					
*49:01:01:01-49:01:16, 49:25, 49:28-49:30, 49:32-49:36, 49:38-49:40, 49:42-49:46, 49:48-49:49, 49:51-49:60N, 49:62-49:65	1				5	6															
*49:02		2	3		5	6															
*49:03				4	5	6										16					
*49:04:01	1						7												19		
*49:04:02	1						7	8											19		
*49:05	1						7	8													
*49:06	1	2			5	6															
*49:07, 49:21 ⁴	1				5	6			9												
*49:08, 49:16 ⁴	1				5	6				10											
*49:09	1			4	5	6															
*49:10, 49:24, 49:31	1				5	6						12									
*49:11	1				5	6							13								
*49:12	1				5	6								14							
*49:13-49:14 ⁴	1				5	6									15						
*49:17	1				5	6								14	15						
*49:18:01-49:18:02	1																			20	
*49:19N	1				5	6		8						14							
*49:20	1				5	6	7														
*49:22	1		3		5	6															
*49:23	1				5	6											17				
*49:26		2			5	6															
*49:27	1				5	6												18			
*49:37, 49:41	1				5																
*49:47	1				6																
*49:50	1						7														
*49:61	1																				21
*07:28, C*01:115, C*14:79																					w
*07:78, 13:31, 13:41, 13:128, 13:131, 15:303, 54:16, 54:20, 54:33, 54:40, 55:09, 55:21, 55:46, 55:52, 55:56, 56:01:05, 56:13, 56:43					5			8													
*07:84, 13:35, 40:44, 51:90, 52:12								8											19		
*08:03, 08:52, 08:78, 37:34, 38:30, 44:06, 53:22, 57:09, 57:24, 57:119										11											
*08:09, 08:84, 13:59, 13:62, 15:83:01:01-15:83:01:02, 27:14, 27:81, 27:130, 27:153, 40:06:01:01-40:06:22, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:93, 40:95, 40:103, 40:127, 40:131, 40:148, 40:161-40:162, 40:165, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268-40:270, 40:275, 40:294-40:295, 40:298:01-40:298:02, 40:306-40:307, 40:311, 40:313, 40:317-40:318, 40:341, 40:348-40:350, 40:354, 40:361N-40:364, 40:373-40:374, 40:376, 40:387-40:389, 40:408, 40:411, 40:423, 40:425, 40:427, 40:434, 40:445, 41:44, 42:04, 51:01:01:01-51:01:18, 51:01:20-51:03, 51:05, 51:07:02, 51:09:01-51:12, 51:14, 51:16, 51:18-51:19, 51:21-51:22, 51:24:01-51:24:05, 51:26-51:35, 51:38-51:41N, 51:43, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:80, 51:82-51:89, 51:91, 51:93, 51:95-51:96, 51:98N-51:105, 51:107, 51:109-51:111, 51:113-51:125, 51:127-51:129, 51:131-51:134, 51:136-51:138, 51:140, 51:142-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01-51:161, 51:163-51:171, 51:173Q-51:176, 51:178N-51:193, 51:195-51:204:01, 51:205-51:209, 51:211-51:212, 51:215, 51:217-51:219, 51:221, 51:224, 51:226, 51:228-51:250, 51:252-51:260, 51:262, 51:264N-51:273N, 51:276-51:280, 51:282, 51:284-51:292, 51:294-51:299, 51:301-51:305, 51:307, 52:01:01:01-52:01:01:09, 52:01:03, 52:01:05-52:01:08, 52:01:10-52:01:38, 52:01:41-52:01:42, 52:01:44-52:01:46, 52:02:02, 52:04-52:08, 52:09:02-52:10:02, 52:11, 52:13, 52:16-52:18, 52:20, 52:21:02-52:24, 52:27-52:31:01, 52:32, 52:34-52:38, 52:40, 52:42-52:74, 52:76-52:77, 52:79-52:84, 52:86-52:87, 52:89N-52:94N, 54:04, 54:11, 55:17, 55:20, 55:28, 55:81, 56:05:01-56:06, 56:15, 56:21, 58:08:01-58:08:02, 59:02, 78:01:01:01-78:10, C*14:92								8													
*13:01:01:01-13:01:16, 13:06-13:07N, 13:11-13:12:02, 13:17, 13:22:01-13:22:02, 13:25, 13:28-13:29, 13:36, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N-13:64, 13:76N-13:78, 13:80, 13:83, 13:87, 13:90, 13:92, 13:95, 13:106-13:107, 13:109, 13:122, 13:135, 13:138, 15:86, 15:224, 35:432, 35:446, 40:48, 40:71, 40:377, 54:38, 56:02:01:01-56:02:03, 56:04:01:01-56:04:04, 56:10, 56:56-56:57, 56:60, 58:73						6							13								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21

[illegible]

*41:60	1							8			11							20			
*44:02:01:01-44:02:09, 44:02:11-44:02:12, 44:02:14-44:02:18, 44:02:20-44:02:31, 44:02:33-44:02:44, 44:02:46-44:03:15, 44:03:17-44:03:29, 44:03:31-44:03:36, 44:03:38-44:03:50, 44:03:52-44:05:02, 44:05:04, 44:12-44:14, 44:16-44:17, 44:19N, 44:21-44:24, 44:26-44:27:02, 44:28:01-44:33, 44:34:02-44:39, 44:41:01-44:43:02, 44:45, 44:48, 44:51-44:54, 44:56N, 44:58N-44:59:02, 44:61N-44:74, 44:77-44:78, 44:80, 44:82, 44:84:01-44:89, 44:91-44:94, 44:96-44:99, 44:101-44:102, 44:104-44:110, 44:112-44:128:02, 44:132-44:133, 44:135-44:144, 44:146-44:149N, 44:151, 44:153-44:155, 44:157-44:169, 44:171N-44:187, 44:189-44:193, 44:195N, 44:198N-44:199, 44:202-44:209, 44:211, 44:213-44:224, 44:226-44:245, 44:247-44:253, 44:255-44:256, 44:258-44:276, 44:278-44:292, 44:294-44:338, 44:341N-44:344, 44:346, 44:437, 44:439-44:444, 44:446-44:453, 44:456-44:458, 44:460-44:480Q, 47:04, C*16:85			3								12										
*44:02:10, 44:34:01			3	4							12			16							
*44:10, 44:340			3			6					12	13									
*44:15:01:01-44:15:01:02		2	3		5		7														
*44:18	1				5		7				11										
*44:20, 44:47			3				7	8			12										
*44:25, 44:95	1										11	12									
*44:50:01, 44:50:03, 53:30, 57:45, 57:69, 58:111	1										12										
*44:76, 44:150			3								12							w			
*44:79:01-44:79:02			3								12							21			
*44:100, 44:197			3				7	8			12						20				
*44:339			3		5	6	7	8			12	13									
*45:01:01:01-45:01:08, 45:03, 45:05-45:07, 45:11-45:13, 45:15-45:23, 45:25					5		7														
*45:04, 45:08, 45:24, 50:39, 50:47					5																
*45:10					5				9												
*45:14					5		7	8													
*50:01:01:01-50:01:15, 50:04-50:08, 50:10-50:11, 50:15, 50:18-50:19, 50:32, 50:34-50:38, 50:40, 50:44-50:46, 50:48-50:50, 50:52-50:55, 50:57, 50:59-50:69					5	6															
*50:02:01:01-50:02:01:02, 50:56					5	6	7														
*50:13, 50:43					5	6					12										
*50:31					5	6			9												
*51:07:01, 51:214, 51:222, 52:01:02:01-52:01:02:04, 52:01:04, 52:01:09, 52:01:40, 52:01:43, 52:01:47-52:02:01, 52:03, 52:09:01, 52:10:03, 52:21:01, 52:31:02, 52:85			4					8						16							
*51:08:01:01-51:08:04, 51:20, 51:36, 51:44N, 51:141, 51:153, 51:263, 51:274-51:275, 52:19								8			11										
*51:62			4		6																
*51:92:01-51:92:02, 52:75							7	8									w				
*51:97							7	8			11										
*51:106:01, 51:223, 52:25:02			4		6			8													
*51:106:02, 51:225, 52:25:01			4	5	6			8													
*51:126		3						8													
*51:130								8					14								
*51:177								8					14	15							
*51:204:02, 52:33								8		10											
*51:210										10											
*51:251			4				7	8							16						
*51:261, 52:26								8										21			
*51:283		2						8													
*55:27																	19				
*55:44, 55:53, 55:94						6		8													
*55:87					5	6		8					14								
*56:61					5	6		8					13	14	15						
*56:67					5	6		8			12	13									
C*03:433																		21			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21