

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-B allele ^{1,2,3}																
*50:01:01-50:01:04, 50:34-50:36	1			4	5	6	7									
*50:02		2		4	5	6	7									
*50:04	1		3	4	5	6	7									
*50:05	1			4	5	6	7	8								
*50:06	1			4		6	7		9							
*50:07	1			4	5	6	7			10						
*50:08	1			4	5	6	7				11					
*50:09	1				5							12				
*50:10	1			4	5	6	7						13			
*50:11	1			4	5	6	7							14		
*50:12					5	6	7		9							
*50:13	1			4	5	6	7								15	
*50:14	1				5											16
*50:15	1			4	5	6	7									16
*50:16	1				5	6	7				11					
*50:17					5	6	7	8								
*50:18	1			4	5	6	7					12				
*50:19	1			4	5	6	7					12			15	
*50:20	1				5											
*50:31	1			4	5	6	7									
*50:32	1			4	5	6	w									
*50:33, 41:21 ⁴	1				5											
*50:37	1			4	5	6	7									
*07:78, 55:09	1			4			w									
*07:84, 13:35, 13:72	1															
*07:133, 15:143, 15:212, 18:48, 35:19, 39:69, 40:01:01-40:01:05, 40:01:07-40:03, 40:05, 40:07, 40:09, 40:11:01-40:11:02, 40:14:01-40:16, 40:18, 40:20, 40:22N-40:24, 40:27:01-40:27:02, 40:29-40:39, 40:42-40:43, 40:45, 40:50-40:52, 40:54-40:57, 40:60-40:63, 40:65-40:67, 40:69, 40:72:01-40:74, 40:77-40:82, 40:84-40:85, 40:87:01-40:92, 40:94, 40:97-40:102, 40:104-40:105, 40:107-40:108, 40:111-40:116, 40:118N-40:119, 40:121-40:126, 40:128, 40:130:01-40:130:02, 40:132-40:136, 40:138-40:147, 40:149-40:156, 40:158, 40:160:01, 40:163, 40:168-40:169, 40:171-40:176, 40:178-40:184, 40:186-40:187, 40:189, 40:191-40:200, 40:202-40:207, 40:211-40:218, 40:220-40:228, 40:231, 40:233-40:243, 40:245-40:255, 40:257-40:264, 40:266-40:267, 41:02:01-41:04, 41:08, 41:10-41:11, 41:13, 41:15, 41:18-41:19, 41:23-41:24, 41:27, 41:31, 44:131					5											
*07:177																w
*07:182N, 15:18:03, 15:73, A*02:339, C*03:19, C*03:102, C*15:26			3													
*08:09, 08:84, 13:46, 15:83, 39:06:01-39:06:04, 39:33-39:34, 39:57, 39:62, 39:64, 39:83, 41:05, 41:26, 42:04, 51:01:01-51:03, 51:05, 51:07:01-51:09:02, 51:11N-51:14, 51:17-51:22, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35-51:36, 51:38-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:63, 51:65-51:69, 51:71-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:94-51:102, 51:104-51:105, 51:109-51:118N, 51:120-51:134, 51:136-51:138, 51:140-51:147, 51:149N-51:156, 51:158-51:166, 52:01:01-52:07, 52:09-52:11, 52:13-52:14, 52:16-52:24, 52:27-52:36, 54:04, 54:11, 55:13, 55:17, 55:20, 55:28, 55:32, 55:58, 56:05:01-56:06, 56:15, 56:21-56:22, 58:08:01-58:08:02, 59:02-59:03, 78:01:01-78:07	1															
*08:102, 18:80, 27:04:03, 27:23, 27:92, 35:01:10, 35:04:02, 35:90, 35:150, 35:184, 35:201, 38:16, 38:46, 51:56:01-51:56:02, C*02:02:23																16
*13:01:01-13:01:08, 13:06, 13:12:01-13:12:02, 13:17, 13:22:01-13:23, 13:25, 13:28-13:29, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N						6		8								
*13:02:01-13:02:14, 13:02:16, 13:08-13:09, 13:14-13:15, 13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44-13:45, 13:49N, 13:54-13:56N, 13:58, 13:65-13:70, 54:26, 55:22	1			4		6										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*13:02:15, 13:47	1					6										

[illegible]

[illegible]

		19			
	18				
	18				
	18				
17	18				
	18				
	18	19			
	18				
	18				
	18				
	18				
17	18	19	20	21	
17	18	19	20	21	

17		19			
		19			
	18				
	18				
17					
17		19			
	18				
17	18	19	20	21	