

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
HLA-B allele ^{1,2,3}																						
*50:01:01:01-50:01:08, 50:32, 50:34-50:36, 50:38, 50:44, 50:48, 50:50, 50:52-50:53	1			4	5	6	7															
*50:02		2		4	5	6	7															
*50:04	1		3	4	5	6	7															
*50:05	1			4	5	6	7	8														
*50:06	1			4		6	7		9													
*50:07	1			4	5	6	7			10												
*50:08	1			4	5	6	7				11											
*50:09	1				5							12										
*50:10	1			4	5	6	7						13									
*50:11	1			4	5	6	7							14								
*50:12					5	6	7		9													
*50:13, 50:43	1			4	5	6	7								15							
*50:14	1				5											16						
*50:15	1			4	5	6	7									16						
*50:16	1				5	6	7				11											
*50:17					5	6	7	8														
*50:18	1			4	5	6	7					12										
*50:19	1			4	5	6	7					12			15							
*50:20	1				5												17					
*50:31	1			4	5	6	7											18				
*50:33	1				5													19				
*50:37	1			4	5	6	7												20			
*50:39	1			4	5																21	
*50:40	1			4	5	6	7										17					
*50:41	1				5	6	7														21	
*50:42	1				5	6	7															
*50:45	1			4	5	6	7															22
*50:46	1			4	5	6	7		9													
*50:47	1			4	5	6																
*50:49	1			4	5	6																22
*50:51					5	6	7															
*07:78, 55:09	1			4			w											18				
*07:84, 13:35, 13:72	1																	18	19			
*07:133, 08:155, 15:143, 15:212, 18:48, 35:19, 35:274, 35:317, 39:69, 40:01:01-40:01:05, 40:01:07-40:03:01:02, 40:05:01:01-40:05:01:02, 40:07, 40:09, 40:11:01-40:11:02, 40:14:01-40:16, 40:18, 40:20, 40:22N-40:24, 40:27:01-40:27:02, 40:29-40:39, 40:42-40:43, 40:45, 40:50-40:52, 40:54-40:57, 40:60-40:63, 40:65-40:67, 40:69, 40:72:01-40:74, 40:77-40:82, 40:84-40:85, 40:87:01-40:92, 40:94, 40:97-40:102, 40:104-40:105, 40:107-40:108, 40:111-40:116, 40:118N-40:119, 40:121-40:126, 40:128, 40:130:01-40:130:02, 40:132-40:136, 40:138-40:147, 40:149-40:156, 40:158, 40:163, 40:168-40:169, 40:171-40:176, 40:178-40:184, 40:186:01-40:187, 40:189, 40:191-40:200, 40:202-40:207, 40:211-40:218, 40:220-40:228, 40:231, 40:233-40:243, 40:245-40:255, 40:257-40:264, 40:266-40:267, 40:271-40:274, 40:276-40:291N, 40:293, 40:296-40:297, 40:299-40:305, 40:309-40:310, 40:312, 40:314-40:316, 40:319-40:324, 40:326-40:329, 40:331-40:339, 40:344-40:347, 40:351-40:353, 40:355-40:360, 41:02:01-41:04, 41:08, 41:10-41:11, 41:13, 41:15, 41:18-41:19, 41:23-41:24, 41:27, 41:31, 41:36, 41:38-41:40, 41:42-41:43, 41:45N, 41:49, 41:51-41:52, 48:41					5																	
*07:177, 27:157																w						
*07:182N, 15:18:03, A*02:339, C*02:510, C*03:19, C*03:102, C*03:307, C*03:318N, C*15:26			3																			
*08:09, 08:84, 13:46, 15:83, 35:273, 35:276, 39:06:01-39:06:05, 39:33-39:34, 39:57, 39:62, 39:64, 39:83, 39:102, 39:109, 39:127, 42:04, 49:18:02, 51:01:01-51:01:03, 51:05, 51:07:01-51:09:03, 51:11N-51:14, 51:17-51:22, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35-51:36, 51:38-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:63:01-51:63:02, 51:65-51:69, 51:71-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:94-51:102, 51:104-51:105, 51:109-51:118N, 51:120-51:134, 51:136-51:138, 51:140-51:147, 51:149N-51:156, 51:158:01, 51:159-51:171, 51:173Q-51:174:02, 51:176-51:187, 51:189-51:191, 51:193, 51:195-51:196, 51:198-51:211, 51:215, 51:217-51:219, 51:221-51:222, 51:224, 51:226, 52:01:01-52:07, 52:09-52:11, 52:13-52:14, 52:16-52:24, 52:27-52:47, 52:49N-52:54, 52:56-52:62, 52:64-52:66, 52:68-52:71, 54:04, 54:11, 55:13, 55:17, 55:20, 55:28, 55:32, 55:58, 55:81, 56:05:01-56:06, 56:15, 56:21-56:22, 58:08:01-58:08:02, 59:02-59:03, 78:01:01-78:07, 78:09	1																					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*08:102, 08:122, 14:48, 15:345, 18:80, 18:102, 27:04:03, 27:23, 27:92, 35:01:10, 35:04:02, 35:90, 35:150:01-35:150:02, 35:184, 35:201, 38:16, 38:46, 51:56:01-51:56:03, C*02:02:23, C*12:03:31																16						
*13:01:01:01-13:01:12, 13:06, 13:12:01-13:12:02, 13:17, 13:22:01-13:23, 13:25, 13:28-13:29, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N, 13:77-13:78, 13:80, 13:83, 13:87, 13:90, 13:92, 13:95					6		8															
*13:02:01:01-13:02:14, 13:02:16-13:02:21, 13:08-13:09, 13:14-13:15, 13:19, 13:27, 13:30-13:34, 13:37-13:38, 13:40-13:42, 13:44-13:45, 13:49N, 13:54-13:56:02N, 13:58, 13:65-13:70, 13:74-13:75, 13:81-13:82, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:96-13:103N, 54:26, 55:22, 59:09	1			4		6												18				

*13:02:15, 13:47	1				6											18						
*13:03, 13:53, 15:303, 35:60, 46:11, 46:18, 49:01:02, 49:11, 49:45, 51:15, 51:157, 54:01:01-54:03, 54:05N, 54:07-54:08N, 54:12-54:13, 54:17-54:19, 54:21-54:25, 54:27, 54:29-54:30, 54:32-54:37, 55:02:01:01-55:02:10, 55:07, 55:10, 55:12, 55:16, 55:18-55:19, 55:21, 55:26, 55:30, 55:34-55:35, 55:41, 55:43, 55:47-55:48, 55:50, 55:57, 55:62, 55:65, 55:67, 55:69-55:72, 55:77, 55:80, 55:82-55:83N, 56:01:01:01-56:01:04, 56:01:06-56:01:09, 56:07-56:08, 56:14, 56:16-56:17, 56:19N-56:20:02, 56:23-56:24, 56:26-56:30, 56:33-56:35, 56:38N-56:49, 56:52-56:53, 59:01:01:01-59:01:01:02, 59:04-59:08, 59:10N	1		4		6																	
*13:04, 15:04:01:01-15:04:04, 15:16:01:01-15:16:03, 15:67, 15:95, 15:155, 15:222, 15:254, 15:293, 15:310, 15:362, 15:408, 35:37, 35:235, 35:265, 46:32, 49:04:01-49:05, 51:37, 51:90, 51:92:01-51:92:02, 51:197, 51:216, 52:12, 52:63, 54:14-54:15, 55:23, 55:27, 55:68, 56:50, C*12:183	1															19						
*13:07N, 13:11, 13:64, 13:76N, 15:86, 56:02:01-56:02:02, 56:10, 58:73, C*07:536					6																	
*13:13:01-13:13:02, 13:21, 13:26:01-13:26:02, 13:51, 13:73, 13:86, 15:20, 15:25:01-15:25:03, 15:36, 15:62, 15:77, 15:85, 15:154, 15:204, 15:240, 15:250, 15:271, 15:289, 15:366, 15:393, 15:402, 15:407, 18:105, 27:19, 27:30, 27:127, 35:28, 35:50, 35:162, 35:231, 35:280, 35:323, 37:19:01-37:19:02, 40:137, 44:02:32, 44:03:16, 44:03:30, 44:08, 44:40, 44:44, 44:49, 44:55, 44:57, 44:60, 44:81, 44:83, 44:103, 44:111, 44:130, 44:134, 44:145, 44:156, 44:188, 44:200, 44:210, 44:212, 44:225, 48:02:01-48:02:03, 48:17, 48:25, 53:17:01-53:17:02, 53:38, 58:41, C*03:278						8																
*13:16, 55:24	1		4		6	w										18						
*13:18	1		4													18						
*13:20					6	w	8															
*13:36					6										16							
*13:48, 15:42, 52:25:01, 55:01:01-55:01:03, 55:01:05-55:01:06, 55:01:08-55:01:16, 55:03, 55:05, 55:11, 55:15, 55:25, 55:29, 55:31, 55:33, 55:36, 55:38, 55:40, 55:45, 55:54-55:55N, 55:59-55:60, 55:64, 55:66, 55:73-55:76, 55:78-55:79, 55:84-55:85, 56:25, 56:37	1		4		6	w																
*13:59, 27:130, 39:50, 40:93, 40:96, 40:109, 40:340, 40:350, 51:10, 51:16, 51:31, 51:34, 51:82, 51:93, 52:08, 52:67, 78:08	1															18						
*13:62, 27:153, 49:18:01, 51:103, 51:119, 51:158:02	1														16							
*14:19, 14:27, 35:261, 35:264, 37:39, 37:48, 53:07, C*03:110, C*03:250, C*03:267, C*04:08, C*04:34, C*04:147, C*04:212, C*05:27, C*05:39, C*05:151, C*08:41, C*08:115, C*08:138, C*17:07, C*18:08										12												
*14:37	1		4		6	7												22				
*15:46, 15:53, 35:47, 35:154, 40:10:01:01-40:10:02, 40:40, 40:49, 40:58, 40:164, 40:210, 40:219, 40:229, 40:325, 41:47, 44:90, 47:02					5									15								
*15:73, 15:224, 39:17, 56:04			3		6																	
*15:106, 40:129, 44:09, 44:46, 44:75, 44:129, 44:254					5		8							15								
*15:137, 51:107, 51:172	1									12												
*18:107					5									15	16							
*27:14, 27:81	1															17	18					
*35:63, 40:04, 40:59, 40:64, 40:120, 40:160:01-40:160:02, 40:343, 41:48, 44:131					5		8															
*35:84, 53:28						8								16								
*35:217, 40:28, 40:68					5		8							16								
*37:50						w																
*39:90	1													14								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*40:01:06, 40:08, 40:25-40:26, 40:106, 40:232					5										16							
*40:06:01:01-40:06:12, 40:06:14-40:06:16, 40:53, 40:70:01-40:70:02, 40:75, 40:86, 40:103, 40:127, 40:131, 40:148, 40:161-40:162, 40:165, 40:167, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268, 40:270, 40:275, 40:294-40:295, 40:298, 40:306-40:307, 40:311, 40:317-40:318, 40:330, 40:341, 40:348-40:349, 40:354	1				5													18				
*40:06:13, 40:269					5													18				
*40:44, 40:159	1				5													18	19			
*40:48, 40:71			3		5	6																
*40:83	1				5					10									18			
*40:95, 40:313	1				5										16				18			
*40:110	1														15				18			
*40:166					5										w							
*40:170, 41:41					5					10												
*40:208								9														
*40:342	1				5										15				18			
*41:01:01-41:01:05, 41:06-41:07, 41:09, 41:12, 41:14, 41:16, 41:20, 41:22, 41:25, 41:28-41:30, 41:32, 41:34-41:35, 41:37, 41:44, 41:50	1				5																	22
*41:05, 41:26, C*05:152	1																					22
*41:17	1				5																	
*41:21	1				5														19			22
*41:33	1				5										15							

[illegible]