

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*50:01:01:01-50:01:10, 50:32, 50:34-50:36, 50:38, 50:44, 50:48, 50:50, 50:52-50:55, 50:57-50:63	1			4	5	6	7																	
*50:02, 50:56		2		4	5	6	7																	
*50:04	1		3	4	5	6	7																	
*50:05	1			4	5	6	7	8																
*50:06	1			4	5	6	7		9															
*50:07	1			4	5	6	7			10														
*50:08	1			4	5	6	7				11													
*50:09	1			5								12												
*50:10	1			4	5	6	7						13											
*50:11	1			4	5	6	7						14											
*50:12				5	6	7			9															
*50:13, 50:43	1			4	5	6	7							15										
*50:14	1			5											16									
*50:15	1			4	5	6	7								16									
*50:16	1			5	6	7				11														
*50:17				5	6	7	8																	
*50:18	1			4	5	6	7					12												
*50:19	1			4	5	6	7					12			15									
*50:20	1			5													17							
*50:31	1			4	5	6	7										18							
*50:33	1			5														19						
*50:37	1			4	5	6	7												20					
*50:39	1			4	5															21				
*50:40	1			4	5	6	7										17							
*50:41	1			5	6	7															21			
*50:42	1			5	6	7																		
*50:45	1			4	5	6	7															22		
*50:46	1			4	5	6	7		9															
*50:47	1			4	5	6																		
*50:49	1			4	5	6																22		
*50:51				5	6	7																	23	
*07:78, 13:128, 55:09	1			4				w										18						
*07:84, 13:35, 13:72, 13:127	1																	18	19					
*07:133, 08:155, 15:143, 15:212, 18:48, 35:19, 35:274, 35:317, 39:69, 40:01:01-40:01:05, 40:01:07-40:03:01:02, 40:05:01-40:05:01:02, 40:07, 40:09, 40:11:01-40:11:02, 40:14:01-40:16, 40:18, 40:20:01:01-40:20:01:02, 40:22N-40:24, 40:27:01-40:27:02, 40:29-40:39, 40:42-40:43, 40:45, 40:50-40:52, 40:54-40:57, 40:60-40:63, 40:65-40:67, 40:69, 40:72:01-40:74, 40:77-40:82, 40:84-40:85, 40:87:01-40:92, 40:94, 40:97-40:102, 40:104-40:105, 40:107-40:108, 40:111-40:116, 40:118N-40:119, 40:121-40:126, 40:128, 40:130:01-40:130:02, 40:132-40:136, 40:138-40:147, 40:149-40:156, 40:158, 40:163, 40:168-40:169, 40:171-40:176, 40:178-40:184, 40:186:01-40:187, 40:189, 40:191-40:200, 40:202-40:207, 40:211-40:218, 40:220-40:228, 40:231, 40:233-40:243, 40:245-40:255, 40:257-40:264, 40:266-40:267, 40:271-40:274, 40:276-40:291N, 40:293, 40:296-40:297, 40:299-40:305, 40:309-40:310, 40:312, 40:314-40:316, 40:319-40:324, 40:326-40:329, 40:331-40:339, 40:344-40:347, 40:351-40:353, 40:355-40:360, 40:367-40:372N, 40:375, 40:378-40:386, 40:390-40:393, 40:395-40:398, 40:401-40:407, 40:409-40:410, 40:412-40:422, 40:424, 40:426N, 40:428N-40:429, 40:431, 41:02:01:01-41:04, 41:08, 41:10-41:11, 41:13, 41:15, 41:18-41:19, 41:23-41:24, 41:27, 41:31, 41:36, 41:38-41:40, 41:42-41:43, 41:45N, 41:49, 41:51-41:52, 41:55, 41:58, 48:41					5																			
*07:177, 27:157																w								
*07:182N, 15:18:03, 56:63, A*02:339, A*02:510, C*03:19, C*03:102, C*03:307, C*03:318N, C*15:26			3																					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*08:09, 08:84, 13:46, 15:83, 35:273, 35:276, 39:06:01-39:06:06, 39:33-39:34, 39:57, 39:62, 39:64, 39:83, 39:102, 39:109, 39:127, 39:129, 39:132, 39:140-39:141, 42:04, 49:18:02, 51:01:01-51:01:03, 51:05, 51:07:01-51:09:03, 51:11N-51:14, 51:17-51:22, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35-51:36, 51:38-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:63:01-51:63:02, 51:65-51:69, 51:71-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:94-51:102, 51:104-51:105, 51:109-51:118N, 51:120-51:134, 51:136-51:138, 51:140-51:147, 51:149N-51:156, 51:158:01, 51:159-51:171, 51:173Q-51:174:02, 51:176-51:187, 51:189-51:191, 51:193, 51:195-51:196, 51:198-51:211, 51:215, 51:217-51:219, 51:221-51:222, 51:224, 51:226, 51:228-51:260, 51:262-51:273N, 51:275-51:280, 52:01:01-52:07, 52:09:01-52:11, 52:13-52:14:02, 52:16-52:24, 52:27-52:47, 52:49N-52:54, 52:56-52:62, 52:64-52:66, 52:68-52:74, 52:76-52:82, 52:84-52:87, 54:04, 54:11, 55:13, 55:17, 55:20, 55:28, 55:32, 55:58, 55:81, 56:05:01-56:06, 56:15, 56:21-56:22, 58:08:01-58:08:02, 59:02-59:03, 78:01:01-78:07, 78:09-78:10, C*14:92	1																							
*08:102, 08:122, 14:48, 15:345, 18:80, 18:102, 27:04:03, 27:23, 27:92, 35:01:10, 35:04:02, 35:90, 35:150:01-35:150:02, 35:184, 35:201, 38:16, 38:46, 51:56:01-51:56:03, C*02:02:23, C*12:03:31																16								
*13:01:01-13:01:14, 13:06, 13:12:01-13:12:02, 13:17, 13:22:01-13:23, 13:25, 13:28-13:29, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N, 13:77-13:78, 13:80, 13:83, 13:87, 13:90, 13:92, 13:95, 13:106-13:107, 13:109-13:110, 13:122					6		8																	

Negative Control

[illegible]

[illegible]

