

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
HLA-B allele^{1,2,3}																					
*52:01:01:01-52:01:31, 52:22-52:24, 52:25:02, 52:33-52:35, 52:37-52:42, 52:44-52:48, 52:50-52:52, 52:54-52:62, 52:64-52:66, 52:68-52:70, 52:72, 52:76	1		3									12	13								
*52:02:01-52:02:02	1			4								12	13								
*52:03			3		5	6						12	13								
*52:04, 52:09:01-52:09:02 ⁵	1	2	3									12	13								
*52:05	1		3		5	6						12	13								
*52:06:01	1		3						9		11	13									
*52:06:02-52:06:03	1		3								11	13									
*52:07	1		3									12		14							
*52:08	1		3							10		12	13								
*52:10:01-52:10:03, 52:71, 52:74			3			6						12	13								
*52:11			3		5							12	13								
*52:12, 52:26, 52:63	1		3									12	13		15						
*52:13	1		3				7					12	13								
*52:14, 52:49N ⁵	1		3						9			12	13								
*52:15	1		3			6						12	13								
*52:16	1											12	13								20
*52:17	1		3								11	12	13								
*52:18	1								10			13									
*52:19			3				7					12	13								
*52:20, 52:25:01 ⁵	1		3					8				12	13								
*52:21:01-52:21:02	1		3		5							12	13								
*52:27	1		3									12	13								20
*52:28-52:29, 53:17:01-53:17:02 ⁴			3									12	13								
*52:30	1		3									12	13								18
*52:31:01-52:31:02, 53:28	1		3									12	13			16					
*52:32	1											12	13								
*52:36	1		3	4								12	13								
*52:43	1		3									12	13								17
*52:53	1		3									12	13								19
*52:67			3		5				10			12	13								
*52:73, 78:05-78:06 ⁴	1		3									13									
*52:75	1		3						9			12	13		15						
*07:02:01:01-07:02:10, 07:02:12-07:05:02, 07:05:04-07:26, 07:28-07:37:02, 07:39-07:49N, 07:51-07:57, 07:59-07:77, 07:79-07:83, 07:85-07:217, 07:219-07:316N, 08:01:01:01-08:01:39, 08:05, 08:07-08:08N, 08:10-08:16, 08:18-08:51, 08:53:01-08:53:02, 08:55-08:77, 08:79-08:83, 08:85-08:109, 08:111-08:116, 08:118-08:125, 08:127-08:177, 08:179-08:190, 08:192-08:196, 13:07N, 13:36, 13:39, 15:01:01:01-15:01:01:15, 15:01:03-15:01:04, 15:01:06-15:02:12, 15:05:01-15:08:02, 15:10:01-15:12:02, 15:14-15:15, 15:18:01:01-15:19, 15:21:01:01-15:21:01:02, 15:23, 15:25:01-15:35, 15:39:01-15:40:02, 15:42, 15:44-15:46, 15:48-15:53, 15:55-15:58, 15:60, 15:63, 15:65-15:66, 15:70-15:82, 15:84-15:86, 15:88-15:90, 15:92-15:94N, 15:96-15:97, 15:99, 15:101, 15:104-15:122, 15:124-15:126, 15:128-15:129, 15:133-15:136, 15:138-15:150, 15:152-15:154, 15:159-15:161, 15:163-15:167, 15:169-15:172, 15:174-15:176, 15:178-15:195, 15:197:01-15:201, 15:203-15:207, 15:209N, 15:211-15:215, 15:217-15:219, 15:221, 15:223-15:227, 15:229, 15:231-15:234, 15:236-15:238, 15:240-15:241, 15:244-15:250, 15:255, 15:257-15:265, 15:267, 15:269-15:272N, 15:275:01-15:276, 15:279-15:280, 15:283, 15:285-15:292, 15:294N-15:304N, 15:306-15:309, 15:311-15:322, 15:324-15:334, 15:336-15:338, 15:340-15:355, 15:357-15:360, 15:363:01-15:363:02, 15:365-15:367, 15:370-15:375N, 15:377Q-15:378, 15:380N-15:388, 15:390-15:395, 15:398-15:402, 15:404-15:407, 15:409-15:410, 15:412-15:417, 15:419-15:422, 15:425-15:428, 15:430-15:434, 15:436-15:441, 15:443, 15:445, 15:447, 15:449-15:450, 18:04:01-18:04:02, 18:07:01-18:07:02, 18:29, 18:33, 18:79-18:80, 18:98, 18:102, 18:107, 18:125, 27:01-27:04:02, 27:04:04-27:05:30, 27:05:32-27:13, 27:15-27:21, 27:24-27:28, 27:30-27:80, 27:82-27:129, 27:131-27:152, 27:154-27:175, 35:31, 35:43:01, 35:46, 35:57, 35:79, 35:87, 35:185, 35:279, 37:60, 37:67, 39:43, 40:01:01-40:01:05, 40:01:07-40:01:41, 40:01:43-40:01:50, 40:01:52-40:05:01:02, 40:07-40:11:02, 40:14:01-40:16, 40:18, 40:20:01:01-40:25, 40:27:01-40:27:02, 40:29-40:40, 40:42-40:43, 40:45-40:46, 40:48-40:50, 40:52, 40:54-40:69, 40:71-40:74, 40:76-40:82, 40:84-40:85, 40:87:01-40:92, 40:94, 40:97-40:102, 40:104-40:108, 40:111-40:116, 40:118N-40:126, 40:128-40:130:02, 40:132-40:147, 40:150-40:156, 40:158, 40:160:01-40:160:02, 40:163-40:164, 40:166, 40:168-40:173, 40:175-40:176, 40:178-40:184, 40:186:01-40:187, 40:189, 40:191-40:200, 40:202-40:208, 40:210-40:229, 40:231-40:235, 40:237-40:243, 40:245-40:255, 40:257-40:267, 40:271, 40:273-40:274, 40:276-40:291N, 40:293, 40:296-40:297, 40:299-40:300, 40:302-40:305, 40:309-40:310, 40:312, 40:314-40:316, 40:319-40:329, 40:331-40:339, 40:343-40:347, 40:351-40:353, 40:355-40:360, 40:367-40:371, 41:01:01:01-41:20, 41:23-41:42, 41:44-41:45N, 41:47-41:49, 41:51-41:55, 42:01:01-42:02:01:02, 42:05:01-42:10, 42:12-42:24, 44:02:22, 44:03:36, 44:09, 44:12-44:13, 44:46, 44:67, 44:69:01-44:69:02, 44:75, 44:90, 44:97, 44:113, 44:127, 44:129, 44:131, 44:171N, 44:254, 44:263, 44:277, 45:01:01-45:01:03, 45:03-45:08, 45:11-45:21, 45:23, 46:01:01-46:31, 46:33-46:74, 47:02, 48:06, 48:31, 49:21, 49:26, 49:32, 49:34, 50:01:01:01-														14							

Negative Control

*51:10										10	11	13									
*51:15, 53:11-53:13, 53:31, 53:36								8			11	13									
*51:16, 51:82	1										10	11	13								
*51:31						6					10	11	13								
*51:34					5						10	11	13								
*51:36					5		7				11	13									
*51:37, 51:90, 51:216	1										11	13	15								
*51:40, 51:108					5	6					11	13									
*51:54					5	w		8			11	13									
*51:61:01-51:61:02	1				5						11	13									
*51:62, 51:78:01-51:78:02, 51:106:02, 51:225	1							8			11	13									
*51:65	1										11	13						17			
*51:69	1										11	13							18		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
*51:92:01-51:92:02	1								9		11	13	15								
*51:93					5	6				10	11	13									
*51:104	1			4							11	13									
*51:139						6					11	13				16					
*51:157								8			11		14								
*51:185	1	2									11	13									
*51:197											11	13	15								
*51:217						w					11	13									
*51:220, A*24:328											11										
*53:05:01-53:05:02, 53:16, 53:33		2									11	13									
*53:38			3					8				12	13								
*57:01:11, 57:91				4																	
*57:60		2		4									14								
*58:36				4									13								
*58:92							7						13								
*59:03									9		11		14								
*59:09										10	11		14								
*73:01-73:02									9	10											
*78:01:01:02, C*01:30, C*03:384, C*06:207, C*08:51, C*08:114, C*12:87, C*14:76, C*15:39, C*15:130	1																				
*78:08						6				10		13	14								
*83:01													?	?							
C*02:02:23, C*12:03:31, C*14:92					5																
C*05:152									9												
C*07:463, C*12:183															15						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21

Negative Control