

Allele	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*52:01:01:01-52:01:28, 52:01:30-52:01:32, 52:01:34-52:01:37, 52:22-52:24, 52:33-52:35, 52:37-52:38, 52:40-52:42, 52:44-52:48, 52:50-52:52, 52:54-52:61, 52:64-52:66, 52:69-52:70, 52:72, 52:76, 52:78-52:87	1		3									12	13								21	
*52:01:29, 52:25:02, 52:39, 52:62, 52:68	1		3									12	13									
*52:01:33, 52:28-52:29	1		3									12	13								21	
*52:02:01-52:02:02	1			4								12	13								21	
*52:03			3		5	6						12	13								21	
*52:04, 52:09:01-52:09:02 ⁴	1	2	3									12	13								21	
*52:05	1		3		5	6						12	13								21	
*52:06:01	1		3						9		11		13								21	
*52:06:02-52:06:03	1		3								11		13								21	
*52:07	1		3									12		14							21	
*52:08	1		3						10			12	13								21	
*52:10:01-52:10:03, 52:71, 52:74			3		6							12	13								21	
*52:11			3		5							12	13								21	
*52:12, 52:26, 52:63	1		3									12	13		15						21	
*52:13	1		3				7					12	13								21	
*52:14:01-52:14:02	1		3						9			12	13								21	
*52:15	1		3		6							12	13								21	
*52:16	1											12	13							20	21	
*52:17	1		3								11	12	13								21	
*52:18	1								10			13									21	
*52:19			3			7						12	13								21	
*52:20, 52:25:01 ⁴	1		3					8				12	13								21	
*52:21:01-52:21:02	1		3		5							12	13								21	
*52:27, 52:77	1		3									12	13							20	21	
*52:30	1		3									12	13					18			21	
*52:31:01-52:31:02	1		3									12	13		16						21	
*52:32	1											12	13								21	
*52:36	1		3	4								12	13								21	
*52:43	1		3									12	13				17				21	
*52:49N	1		3						9			12	13								21	
*52:53	1		3									12	13						19		21	
*52:67			3		5					10		12	13								21	
*52:73	1		3									13									21	
*52:75	1		3						9			12	13		15						21	
*52:88, 53:28	1		3									12	13			16					21	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*07:02:01:01-07:02:10, 07:02:12-07:05:02, 07:05:04-07:26, 07:28-07:37:02, 07:39-07:49N, 07:51-07:57, 07:59-07:77, 07:79-07:83, 07:85:01-07:217, 07:219-07:359, 08:01:01:01-08:01:43, 08:01:45-08:01:48, 08:05, 08:07-08:08N, 08:10-08:16, 08:18-08:51, 08:53:01-08:53:02, 08:55-08:77, 08:79-08:83, 08:85-08:109, 08:111-08:116, 08:118-08:125, 08:127-08:177, 08:179-08:190, 08:192-08:227, 13:07N, 13:36, 13:39, 14:56, 15:01:01:01-15:01:01:20, 15:01:03-15:01:04, 15:01:06-15:02:12, 15:05:01-15:08:02, 15:10:01:01-15:12:03, 15:14-15:15, 15:18:01:01-15:19, 15:21:01:01-15:21:01:02, 15:23, 15:25:01-15:35, 15:39:01-15:40:02, 15:42, 15:44-15:46, 15:48-15:53, 15:55-15:58, 15:60, 15:63, 15:65-15:66, 15:70-15:82, 15:84-15:86, 15:88-15:90, 15:92-15:94N, 15:96-15:97, 15:99, 15:101, 15:104-15:122, 15:124-15:126, 15:128-15:129, 15:133-15:136, 15:138-15:150, 15:152-15:154, 15:159-15:161, 15:163-15:167, 15:169-15:172, 15:174-15:176, 15:178-15:195, 15:197:01-15:201, 15:203-15:207, 15:209N, 15:211-15:215, 15:217-15:219, 15:221, 15:223-15:227, 15:229, 15:231-15:234, 15:236-15:238, 15:240-15:241, 15:244-15:250, 15:255, 15:257-15:265, 15:267, 15:269-15:272N, 15:275:01-15:276, 15:279-15:280, 15:283, 15:285-15:292, 15:294N-15:304N, 15:306-15:309, 15:311-15:322, 15:324-15:334, 15:336-15:338, 15:340-15:355, 15:357-15:360, 15:363:01-15:363:02, 15:365-15:367, 15:370-15:375N, 15:377Q-15:378, 15:380N-15:388, 15:390-15:395, 15:398-15:402, 15:404-15:407, 15:409-15:410, 15:412-15:417, 15:419-15:422, 15:425-15:428, 15:430-15:434, 15:436-15:441, 15:443, 15:445, 15:447, 15:449-15:461, 15:463N-15:474, 15:476-15:498, 15:501, 15:503-15:515, 15:518-15:520Q, 18:04:01-18:04:02, 18:07:01-18:07:02, 18:29, 18:33, 18:79-18:80, 18:98, 18:102, 18:107, 18:125, 18:155, 27:01-27:04:02, 27:04:04-27:05:30, 27:05:32-27:13, 27:15-27:21, 27:24-27:28, 27:30-27:80, 27:82-27:101, 27:103-27:129, 27:131-27:152, 27:154-27:196, 35:31, 35:32:03, 35:43:01, 35:43:03-35:44,																						

Negative Control

[illegible]

[illegible]

*35:235										9				13	15							
*35:251, 78:04						5	6							13								
*35:291, 35:432, 53:09, 58:73									8					13								
*35:329, 53:01:01:01-53:04, 53:07-53:08:02, 53:10, 53:14-53:15, 53:18, 53:20-53:27, 53:29-53:30, 53:32, 53:34-53:35, 53:37, 53:39, 53:41-53:56												11		13								
*37:01:01:01-37:01:05, 37:01:07-37:01:08, 37:01:10-37:01:11, 37:01:13-37:04:02, 37:07, 37:09, 37:12-37:20, 37:23-37:25, 37:27, 37:29-37:36, 37:38-37:56, 37:59, 37:62-37:66, 37:68-37:78, 37:81, 37:83-37:87			3										12		14					20		
*37:01:06, 37:06:01-37:06:02, 37:28, 37:61, 37:82N, 44:130, 44:156, 44:221, 44:224, 44:258, 49:25													12		14					20		
*37:01:12, 37:22, 37:26, 37:79N			3										12		14							
*37:10			3						8				12		14					20		
*37:21			3									11		14								
*37:80			3										12	13						20		
*38:03													12		w					20		
*38:06-38:07, 38:74												11		w								
*38:73														w							21	
*39:02:01-39:02:03, 39:08, 39:13:01-39:13:02, 39:23, 39:39:01-39:39:02, 39:49, 39:88, 39:101, 39:105, 39:110														w						20		
*39:06:01-39:06:06, 39:33-39:34, 39:57, 39:62, 39:64, 39:83, 39:90, 39:102, 39:109, 39:127, 39:129, 39:132, 39:140-39:141										9				w								
*39:50										9	10			w								
*40:01:06						5	6							14								
*40:06:03												10		w								
*40:06:08												10										
*40:19, 40:117, 40:292, 41:60, 49:18:02													12		14						21	
*40:28	1													14		16						
*40:86, 40:330										9	10			14								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*40:95					5						10			14								
*40:109, 40:340										10		12		14							21	
*40:350			3							10				14								
*40:394		2										12		14							21	
*44:03:40					5	6						12		14		16						
*44:20							7					12		14	15							
*44:62					5	w						12		14		16						
*44:83, 44:134, 49:03			3					8				12		14								
*48:02:01, 48:02:03													13							20		
*48:02:02, 56:36													13	14								
*48:18												12		14						20	21	
*49:18:01					5							12		14							21	
*49:50									9			12		14								
*51:01:01:01-51:01:07, 51:01:09-51:01:14, 51:01:16, 51:01:18-51:01:32, 51:01:34-51:01:39, 51:01:41-51:01:42, 51:01:44-51:01:55, 51:01:57-51:01:72, 51:03, 51:06:01-51:06:04, 51:11N-51:12, 51:17-51:18, 51:21, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35, 51:38-51:39, 51:41N, 51:43, 51:48-51:49, 51:51-51:52, 51:55, 51:57-51:58, 51:60, 51:64, 51:66-51:68, 51:70-51:72, 51:74-51:77, 51:79-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:94-51:96, 51:98N-51:103, 51:105, 51:107, 51:109-51:114, 51:116-51:117, 51:119, 51:121, 51:123-51:134, 51:136-51:138, 51:140, 51:142, 51:145, 51:147, 51:149N-51:152, 51:154-51:156, 51:158:01-51:163, 51:165-51:171, 51:173Q-51:174:02, 51:176-51:179, 51:183-51:184N, 51:186-51:191, 51:193, 51:195-51:196, 51:198, 51:200-51:213, 51:215, 51:218-51:219, 51:221, 51:224, 51:226, 51:229-51:236, 51:238-51:246, 51:248-51:250, 51:252-51:258, 51:260, 51:262, 51:264N-51:273N, 51:277-51:279			1								11		13							21		
*51:01:08, 51:01:15, 51:01:40, 51:01:43, 51:14, 51:45, 51:50, 51:63:01-51:63:02, 51:85, 51:106:01, 51:135, 51:182, 51:192, 51:223, 51:247, 53:06			1									11		13								
*51:01:17, 51:07:01-51:07:02, 51:22, 51:118N, 51:222, 56:21, 58:08:01-58:08:02			1											13							21	
*51:01:33			1						9		11		13								21	
*51:01:56, 51:164			1								11										21	
*51:02:01:01-51:02:03, 51:02:05-51:02:06, 51:59, 51:143-51:144, 51:146, 51:180					5						11		13								21	
*51:02:04, 51:53, 51:81, 51:115, 51:120, 51:148, 51:175, 51:181, 51:199, 51:228, 53:19											11		13								21	
*51:04:01-51:04:02, 51:46, 51:56:01-51:56:03			1								11		13		16						21	
*51:05, 51:23					5	w					11		13								21	
*51:08:01:01-51:08:04, 51:20, 51:44N, 51:141, 51:153, 51:263, 51:275							7				11		13								21	
*51:09:01-51:09:03, 51:19, 51:73, 51:122, 51:194, 51:259, 51:276, 51:280							6				11		13								21	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*51:10										10	11		13								21	
*51:13:01-51:13:02, 51:172			1						9		11		13									
*51:15, 53:11-53:13, 53:31, 53:36								8			11		13									
*51:16, 51:82			1							10	11		13								21	

*51:31						6				10	11		13							21		
*51:34						5				10	11		13							21		
*51:36						5		7			11		13							21		
*51:37, 51:90, 51:216, 51:261	1										11		13		15							
*51:40, 51:108						5	6				11		13							21		
*51:54						5	w		8		11		13									
*51:61:01-51:61:02	1					5					11		13							21		
*51:62, 51:78:01-51:78:02, 51:106:02, 51:225	1								8		11		13									
*51:65	1										11		13				17			21		
*51:69	1										11		13				18			21		
*51:92:01-51:92:02	1								9		11		13		15							
*51:93						5	6			10	11		13							21		
*51:97								7			11		13									
*51:104	1			4							11		13							21		
*51:139							6				11		13			16				21		
*51:157								8			11			14								
*51:185	1	2									11		13							21		
*51:197											11		13		15							
*51:217							w				11		13							21		
*51:227, 51:281	1										11		13			16						
*51:237	1										11			14						21		
*51:274								7			11		13		15							
*53:05:01-53:05:02, 53:16, 53:33		2									11		13									
*53:17:01-53:17:02			3									12	13									
*53:38			3					8				12	13									
*56:06, C*01:30, C*02:142, C*03:384, C*06:207, C*08:51, C*08:114, C*12:87, C*14:76, C*15:39, C*15:130	1																					
*57:01:11, 57:91				4																		
*57:01:32			4	5	6									14		16						
*57:02:01-57:03:04, 57:05, 57:07, 57:09, 57:17, 57:39, 57:42, 57:46, 57:57, 57:63, 57:66, 57:70, 57:80, 57:82, 57:84, 57:94, 57:96, 57:101			4											14						21		
*57:60		2		4										14								
*58:27-58:28:02, 58:65													13							21		
*58:36			4										13									
*58:92							7						13									
*59:03									9		11			14								
*59:09										10	11			14								
*73:01-73:02									9	10												
*78:05-78:06	1		3										13									
*78:08						6				10			13	14								
*78:10						6							13									
A*24:146																				21		
A*24:328										11												
C*02:02:23, C*12:03:31, C*14:92					5																	
C*03:464, C*07:463, C*07:758, C*12:183															15							
C*05:152									9													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22

Negative Control

[illegible]

