

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*53:01:01-53:01:11, 53:25, 53:27, 53:32, 53:34-53:35	1								9	10	11					16								
*53:02	1	2				6			9	10	11					16								
*53:03, 35:203 ⁴			3						9	10	11					16								
*53:04	1			4					9							16								
*53:05	1		3			6			9	10	11					16								
*53:06	1	2				6		8			11					16								
*53:07	1			4			7		9							16								
*53:08:01-53:08:02	1							8			11					16								
*53:09, 53:38					5				9	10	11				15	16								
*53:10	1						7		9	10	11			14		16								
*53:11					5				9	w	11								20					
*53:12					5	6			9	10	11					16								
*53:13					5				9	10	11					16								
*53:14	1								9	10	11			14		16								
*53:15	1		3						9	10	11					16								
*53:16	1		3			6			9	w	11								20					
*53:17:01-53:17:02	1								9	10	11				15	16								
*53:18	1			4					9	10	11					16								
*53:19	1								9							16						22		
*53:20	1								9	10	11					16	17							
*53:21	1								9	10	11					16		18						
*53:22	1				5				9		11													
*53:23	1								9	10	11					16			19					
*53:24	1								9	w	11								20					
*53:26	1						7		9	10	11					16								
*53:28	1	2				6		8			11				15	16								
*53:29	1								9	10	11					16							23	
*53:30									9	10	11					16	17					21		
*53:31				4	5				9							16								
*53:33	1		3	4		6			9							16								
*53:36					5				9							16						22		
*53:37	1								9	10	11					16						22		
*53:39									9	10	11					16	17							
*07:02:01-07:08, 07:10, 07:12-07:16, 07:18:01-07:19, 07:21-07:23, 07:27-07:33:03, 07:35-07:42, 07:44-07:47, 07:49N-07:50, 07:52-07:59, 07:61-07:64, 07:66-07:77, 07:79, 07:81-07:114, 07:116-07:130, 07:132-07:133, 07:135N-07:146, 07:148-07:154, 07:156-07:163, 07:165-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:227-07:234, 07:236, 07:238-07:245, 07:247-07:249, 07:251N-07:256, 08:20, 08:53:01-08:53:02, 08:79, 14:03, 14:47, 15:30, 15:33, 15:45, 15:63, 15:77-15:78:03, 15:116, 15:141, 15:150, 15:168, 15:233, 15:248, 15:269, 15:287, 15:324, 15:334, 15:342, 27:25, 27:108, 27:115, 35:44, 35:118, 35:213, 39:18, 39:36, 40:15-40:16, 40:23, 40:32, 40:98, 40:161, 40:198, 44:146, 44:150, 46:01:06, 46:45, 46:54, 48:05, 48:08								8																
*07:09, 07:11, 07:17, 07:115, 07:246, 57:09, 57:13								8						14										
*07:20, 07:24, 07:60, 15:99, 44:76								8											20					
*07:26, 07:34, 07:43, 07:80, 07:131, 07:147, 13:20, 13:62, 14:01:01-14:02:09, 14:02:11-14:02:13, 14:04, 14:06:01-14:09, 14:11-14:18, 14:20-14:26, 14:28-14:36, 14:38-14:40, 14:42, 14:44-14:46, 14:48-14:49, 15:09-15:10:04, 15:17:01:01-15:17:03, 15:37, 15:72, 15:90, 15:177, 15:196, 15:208, 15:216, 15:230, 15:243, 15:252, 15:273, 15:313, 15:323, 15:329, 15:338-15:339, 15:356, 15:361, 27:04:01-27:04:04, 27:06, 27:10, 27:15, 27:18, 27:20-27:21, 27:24, 27:40, 27:54, 27:63, 27:66N, 27:68-27:69, 27:79, 27:86, 27:91-27:92, 27:100, 27:103, 27:105-27:106, 27:109, 27:112-27:114, 27:120, 27:138, 38:10, 38:32, 39:35, 40:05, 40:26, 40:51, 40:95, 40:148, 40:174, 40:183, 40:309, 42:19, 44:79:01-44:79:02, 47:10, 48:15, 50:14, 50:20, 51:01:40, 51:104, 51:147, 52:02:01-52:02:02, 52:16, 52:32, 55:01:04, 55:01:07, 55:17, 55:28, 55:44, 55:53, 55:58, 56:05:01-56:05:02, 56:21, 78:02:01-78:02:02								8							16									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:48, 07:209, 07:226, 08:13, 08:55, 08:143, 13:01:01-13:01:10,																								

Negative Control

Negative Control

[illegible]

[illegible]

[illegible]

*15:202, 15:239								8			11			15										
*15:218Q					5						11													
*15:303, 50:01:03, 50:32, 54:33				4				8			12													
*15:308								8		w				15	16									
*15:360								8		w			14		16									
*18:01:01:01-18:01:03, 18:01:05-18:02, 18:05-18:06, 18:08, 18:12, 18:17N, 18:23N-18:24, 18:27-18:28, 18:31-18:32, 18:34, 18:38-18:39, 18:42-18:47, 18:49-18:53, 18:55, 18:59-18:60, 18:62-18:65, 18:70-18:78, 18:80-18:100, 18:102-18:104, 18:106, 18:109, 18:111-18:112							6			10	11				15	16		18						
*18:01:04, 18:40							6			10					15	16		18						
*18:03							6			10	11				15	16		18	19					
*18:04, 18:25, 18:68							6			10	11				15	16								
*18:07:01-18:07:02, 18:29, 18:33, 18:79							6			10	11					16		18						
*18:09					5	6				10	11				15	16		18						
*18:10-18:11, 18:35, 18:110										10	11				15	16		18						
*18:13							6			w	11				15			18						
*18:14							6			10	11		14		15	16		18						
*18:15, 18:57							6	8			11				15	16		18						
*18:18							6	7			10	11			15	16		18						
*18:19, 18:30							6	8			11				15			18						
*18:20							6			10	w				15	16		18						
*18:21								8							15			18						
*18:22, 18:69, 18:105		2					6			10	11				15	16		18						
*18:26, 18:41, 18:61, 40:209															15	16		18						
*18:36							6		9	10	11				15	16		18						
*18:37, 18:48, 18:66, 18:107, 39:43, 56:31							6			10	11				16									
*18:54, 18:108			3				6			10	11				15	16		18						
*18:56					5				9	11					15			18						
*18:58									9	10	11				15	16		18						
*18:67	1		3				6			10	11				15	16		18						
*18:101								8			11				15	16		18						
*18:113					5	6					11				15			18						
*27:52			3													16		18						
*35:01:10, 35:46-35:47, 35:207, 35:217									9	10	11				16				21					
*35:02:01-35:02:07, 35:83, 35:129N, 35:182-35:184, 35:211, 35:220, 35:258, 35:270, 35:273, 35:285									9						16				22					
*35:03:01-35:03:05, 35:03:07-35:03:21, 35:06, 35:13, 35:36, 35:55-35:56, 35:59, 35:70, 35:74-35:75, 35:85, 35:98, 35:106, 35:127-35:128, 35:150:01-35:153, 35:155, 35:160, 35:179, 35:181, 35:193, 35:195, 35:198, 35:204-35:205, 35:221, 35:223, 35:235-35:236, 35:242-35:243, 35:246, 35:256, 35:261, 35:274, 35:279, 35:281-35:282, 35:290-35:291, 39:32, 39:74, 40:71, 44:10, 44:178, 56:02, 56:04, 56:11, 56:22					4				9						16									
*35:04:02, 35:251, 78:04									9						16				21					
*35:05:01-35:05:03, 35:51, 35:89, 35:97, 57:01:01-57:01:02, 57:01:04, 57:01:06-57:01:11, 57:01:13-57:01:16, 57:01:18, 57:01:20-57:01:22, 57:06, 57:08, 57:11, 57:15-57:16, 57:18, 57:21, 57:23, 57:29, 57:33-57:35, 57:37-57:38, 57:40, 57:44, 57:48-57:50, 57:52-57:56, 57:58-57:61, 57:65, 57:67-57:68, 57:71, 57:73-57:75, 57:77-57:79N, 58:36, 58:44									9	10	11		14		16									
*35:18, 35:164, 44:28:01-44:28:02									9									20						
*35:20:01-35:20:02, 35:28, 35:226									9	10	11			15	16									
*35:21, 35:188		2				6		8			11				16									
*35:24:01-35:24:02, 35:287		2				6			9	10	11				16									
*35:29:01-35:29:02, 35:69, 35:76, 35:294									9	10	11				16		19							
*35:35										w	11				16									
*35:38, 35:163, 35:169, 38:30, 44:140, 45:02, 82:01-82:03, C*04:34				4					9															
*35:50, 35:197									9	10	11				16		18							
*35:58, 57:22								8			11		14		16									
*35:62									9	w	11													
*35:66										w	11		14											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*35:72									9	10	11			14	15	16								
*35:81		2							9							16						22		
*35:84, 35:231, 35:267				4					9							16		18						
*35:95									9						15	16						22		
*35:108:01							7		9	10	11					16								
*35:109		2		4					9							16								

Negative Control

Negative Control

*35:114, 57:01:05, 57:31, 57:47									9	10			14	16										
*35:136				4					9					16			19							
*35:141, 46:25									9		11			16										
*35:146									9	10	11			16					22					
*35:154									9					16				21	22					
*35:156, 39:33				4					9								20							
*35:157		2												16										
*35:162									9					16		18			22					
*35:166									9	10	11													
*35:172									9								20		22					
*35:174									9	w							20							
*35:186									9	10	11			16		w								
*35:190		2				6			9	10	11			16										
*35:208									9	10	11			16					23					
*35:229									9	w	11					19	20							
*35:232							8				11		14				20							
*35:233, 44:203		2							9					16										
*35:237				4										16										
*35:257, 35:278				4					9					16				21						
*35:260						5			9															
*35:262										w	11													
*35:296				4					9					15	16									
*37:01:01-37:01:05, 37:01:07-37:01:11, 37:03N, 37:05, 37:08, 37:15-37:18, 37:20, 37:22-37:25, 37:27, 37:30N, 37:32-37:33N, 37:36, 37:38, 37:41, 37:43, 37:45-37:47, 37:49-37:54				3			7							15			18							
*37:01:06							7										18							
*37:02				3										15	16		18							
*37:04:01-37:04:02		2		3			7							15			18							
*37:07				3				8						15			18							
*37:09, 37:12, 37:19, 37:44				3										15			18							
*37:10						5	7							15			18							
*37:11, 37:14, 37:29							7							15			18							
*37:13				3			7	8						15			18							
*37:21, 37:26				3			7										18							
*37:31, 37:42N, 37:55				3			7							15										
*37:34		1					7							15			18							
*37:35				3		5	7				11			15			18							
*37:37							7							15			18	19						
*37:39				3	4		7		9					15			18	20						
*37:40				3			7							15	16		18							
*37:48				3	4		7		9					15			18							
*37:56					3		5				11			15			18							
*38:06-38:07		1													16		18							
*38:19, 39:03, 39:24:01-39:24:02, 39:37, 39:76, 48:07, 57:46													14	16										
*39:19:01-39:19:02														16		18	19							
*40:01:06, 40:07-40:08, 40:68, 40:106, 40:162, 40:220, 40:232														16				21						
*40:03, 40:105, 40:267, 40:284										10			14	16										
*40:13														16	17			21						
*40:19, 40:109, 40:117, 40:292														16	17									
*40:20, 40:38, 40:52, 40:59-40:60, 40:159, 46:06, 48:23										10				16										
*40:185, 40:308														15	16									
*41:24					5					11			14											
*44:02:08, 44:02:39, 44:101, 44:220									w															
*44:03:17									w					16										
*44:06, 51:08:01-51:08:02, 51:44N, 51:141		1							9															
*44:15				4		6			9		12													
*44:18				4					9		12					17								
*44:25, 44:95									9							17								
*44:50:01									9						16	17								
*44:83, 44:134					5				9					15										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*44:96, 44:153									9						16								23	
*44:109						6			9						16									
*44:117				w					9															
*44:176						6			9															
*44:190, C*15:39		2																						
*44:209							7		9						16									
*45:01:01-45:01:02, 45:04-45:07, 45:13-45:16, 56:13, 56:43				4					9			12												
*45:03									9			12												

[illegible]

*56:45				4					9			12			16			19						
*57:01:17									9		11			14										
*57:01:19									9	10	11			14	16								23	
*57:02:01-57:02:02, 57:04-57:05, 57:12, 57:28N, 57:42									9					14				20						
*57:03:01-57:03:02, 57:07, 57:17, 57:32, 57:39, 57:66, 57:70									9					14	16									
*57:14:01		2				6			9	10	11			14	16								23	
*57:14:02		2				6			9	10	11			14	16									
*57:19									9	w	11			14				20						
*57:24					5				9		11			14										
*57:27, 57:72									9	w	11			14										
*57:30											11			14				20						
*57:45, 57:51, 57:69									9	10	11			14	16	17								
*57:63								8						14				20						
*57:76				4					9					14	16									
*58:01:01:01-58:01:02, 58:01:04-58:02, 58:07, 58:10N-58:13, 58:15-58:16:02, 58:18, 58:21, 58:23-58:26, 58:29, 58:31N-58:32, 58:39N-58:43, 58:45:01-58:45:02, 58:47-58:60, 58:62-58:63, 58:66, 58:68-58:69, 58:72N	1								9	10	11			13	14	16								
*58:01:03, 58:61	1								9	10				13	14	16								
*58:04, 58:67	1								9	10	11			13		16								
*58:05	1										11			13	14	16								
*58:06, 58:19	1							8			11			13	14	16								
*58:08:01-58:08:02, 58:46	1							8						13	14	16								
*58:09	1	2				6			9	10	11			13	14	16								
*58:14	1								9	10	11			13	14	16							23	
*58:17N, 58:64									9	10	11			13	14	16								
*58:20, 58:27	1													13	14	16								
*58:22	1								9	w	11			13	14			20						
*58:28:01-58:28:02	1								9					13	14	16								
*58:33	1			4					9	10	11			13	14	16								
*58:34	1			4					9					13	14	16								
*58:35, 58:37-58:38	1									10	11			13	14	16								
*58:65	1								9					13	14	16							22	
*78:05-78:06								8							15	16								
*78:08															16			19		21				
A*02:188, A*24:174, C*14:71										w														
C*02:56, C*06:20, C*07:81											w													
C*03:19, C*03:26, C*03:102				w																				
C*03:250, C*03:267, C*05:27, C*08:41, C*15:15, C*17:07				4																				
C*04:08, C*04:147, C*05:39, C*08:115, C*18:08				4														20						
C*06:153, C*07:342, C*14:53										10														
C*07:168							7				w													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control