

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-B allele ^{1,2,3}																																	
*55:01:01-55:01:06, 55:01:08-55:01:11, 55:53, 55:59-55:60	1	2	3												15																		
*55:01:07	1	2	3																														
*55:02:01-55:02:06	1	2											13		15																		
*55:03	1	2	3	4											15																		
*55:04	1				5																												
*55:05	1	2	3			6									15																		
*55:07, 54:01:02 ⁴	1	2							10				13		15																		
*55:08, 55:51	1					7																											
*55:09	1	2	3								11										22												
*55:10	1	2									12		13		15						21												
*55:11	1	2	3						9						15																		
*55:12	1	2						8					13		15																		
*55:13	1	2													15									24							31		
*55:14	1							9																									
*55:15	1	2	3							11					15																		
*55:16	1	2											13		15								23										
*55:17	1														15		17														30		
*55:18		2											13		15			18							25								
*55:19	1	2											13		15			18															
*55:20	1														15		17			20													
*55:21	1	2													15							22	23										
*55:22	1	2									11		13																				
*55:23	1	2													15							22		24									
*55:24	1	2	3							11																							
*55:25	1	2	3						9						15			18															
*55:26	1	2											13		15										25								
*55:27	1	2													15		17					22		24									
*55:28	1	2													15		17																
*55:29	1	2	3												15							23											
*55:30	1	2											13		15			19															
*55:31	1	2	3												15			18															
*55:32	1	2													15									24									
*55:33	1	2	3												15									24									
*55:34		2											13		15										25	26							
*55:35	1	2											13		15																30		
*55:36	1	2	3												15			19															
*55:37	1	2											w		15							22											
*55:38	1		3												15																30		
*55:39	1	2											13		15		17																
*55:40	1	2	3												15										25								
*55:41	1	2											13		15															28			
*55:42	1	2											13		15											26							
*55:43	1	2											13		15													27				31	
*55:44	1	2	3												15															28			
*55:45	1	2	3												15											26							
*55:46	1	2													15													27					
*55:47		2											13		15											26							
*55:48	1	2								11			13		15																		
*55:49	1			4	5																												
*55:50	1	2											13		15															29			
*55:52	1	2	w												15							22											
*55:54	1	2	3										13		15																		
*55:55N	1	2	3												15																	31	
*55:56	1	2													15					20													
*55:57	1	2													15																		
*55:58	1	2													15										24				28			31	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	

Negative Control

[illegible]

[illegible]

[illegible]

*51:21, 51:101, 51:143, 78:07																15	17															30
*51:35, 51:99, 51:131							7										16															
*51:61:01-51:61:02, 52:21							7										16	17				22	23									
*51:62			3																													
*51:63																	16															
*51:90, 52:12							7										16	17				22		24								
*51:97																	16			20				24								
*51:106, 52:25			3														16															
*51:116, 52:16, 52:27							7										16	17			21											
*51:127							7										16	17								27					31	
*54:01:01, 54:01:03-54:01:04, 54:05N, 54:08N, 54:12-54:13, 54:17-54:19, 54:22-54:23, 54:25, 54:27-54:29	1	2														13	14	15														
*54:02	1	2														13	14	15					23									
*54:03	1	2														13	14	16														
*54:04	1	2															14	15	17													
*54:06, 54:09	1																14															
*54:07, 54:24		2														13	14	15														
*54:10	1	2													w	14	15				22											
*54:11	1															14	15	17														
*54:14	1	2														14	15				22		24									
*54:15	1															14	15	17			22		24									
*54:16	1	2														14	15								27							
*54:20	1	2	w														14	15				22										
*54:21	1	2														13	14	15			21											
*54:26	1	2								11						13	14															
*54:30		2														13	14	15		18												
*56:01:01-56:01:03, 56:01:06, 56:07, 56:24, 56:26, 56:28N																																