

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-B allele ^{1,2,3}																																	
*55:01:01-55:01:06, 55:01:08-55:01:16, 55:53, 55:59-55:60, 55:64, 55:73-55:76	1	2	3												15																		
*55:01:07	1	2	3																														
*55:02:01:01-55:02:10, 55:61, 55:63, 55:65, 55:67, 55:69-55:71, 55:77, 55:80	1	2											13	15																			
*55:03	1	2	3	4											15																		
*55:04	1				5																												
*55:05	1	2	3			6									15																		
*55:07	1	2								10			13	14	15																		
*55:08, 55:51	1						7																										
*55:09	1	2	w								11											22											
*55:10	1	2										12	13	15								21											
*55:11	1	2	3						9						15																		
*55:12	1	2						8					13	15																			
*55:13	1	2													15									24							31		
*55:14	1								9																								
*55:15	1	2	3								11				15																		
*55:16	1	2											13	15									23										
*55:17	1														15		17														30		
*55:18	1	2											13	15	18										25								
*55:19	1	2											13	15	18																		
*55:20	1														15		17			20													
*55:21	1	2													15								22	23									
*55:22	1	2									11		13																				
*55:23, 55:68	1	2													15							22	24										
*55:24	1	2	3								11																						
*55:25	1	2	3						9						15		18																
*55:26, 55:62 ⁵	1	2											13	15											25								
*55:27	1	2													15		17					22	24										
*55:28	1	2													15		17																
*55:29	1	2	3												15								23										
*55:30	1	2											13	15				19															
*55:31	1	2	3												15		18																
*55:32	1	2													15									24									
*55:33	1	2	3												15									24									
*55:34	1	2											13	15											25	26							
*55:35	1	2											13	15																	30		
*55:36	1	2	3												15			19															
*55:37	1	2											w	15								22											
*55:38	1		3												15																30		
*55:39	1	2											13	15		17																	
*55:40	1	2	3												15										25								
*55:41	1	2											13	15																28			
*55:42	1	2											13	15												26							
*55:43	1	2											13	15														27			31		
*55:44	1	2	3												15															28			
*55:45	1	2	3												15											26							
*55:46	1	2													15															27			
*55:47	1	2											13	15												26							
*55:48	1	2									11		13	15																			
*55:49	1			4	5																												
*55:50	1	2			5								13	15																	30		
*55:52	1	2	w												15								22										
*55:54	1	2	3										13	15																			
*55:55N	1	2	3												15																	31	
*55:56	1	2													15						20												
*55:57	1	2													15															28			
*55:58	1	2													15									24					28			31	
*55:66	1	2	3					8							15																		
*55:72	1												13	15																	30		
*55:78, 15:42 ⁴	1	2	3												15																		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	

Negative Control

