

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
HLA-B allele ^{1,2,3}																																
*57:01:01:01-57:01:04, 57:01:06-57:01:36, 57:38, 57:50, 57:54, 57:56, 57:59-57:60, 57:64-57:65, 57:67:02-57:68, 57:71-57:72, 57:75, 57:81, 57:86-57:88, 57:90-57:92, 57:95, 57:97, 57:100, 57:102-57:108	1	2	3										13																			31
*57:01:05, 57:47, 57:74		2	3										13																			31
*57:02:01-57:02:02, 57:42	1			4	5							12	13																			31
*57:03:01:01-57:03:04, 57:46, 57:66, 57:70, 57:84, 57:96, 57:101	1		3	4								12	13																			31
*57:04:01, 57:04:03	1				5	6							13						19													31
*57:04:02	1				5	6													19													31
*57:05	1			4									13										23									31
*57:06	1	2	3		6								13																			31
*57:07	1		3	4				8				12	13																			31
*57:08	1	2	3							10			13																			31
*57:09	1			4					9		11	12	13																			31
*57:10	1	2	3										13	14																		31
*57:11													13	15																		31
*57:12	1			4	5							12	13			16																31
*57:13		2			5				9				13				17	19														31
*57:14:01-57:14:02	1	2	3								11		13																			31
*57:15, 57:37, 57:55 ⁴	1	2	3				7						13																			31
*57:16	1	2	3					8																								31
*57:17	1		3	4								12	13	14																		31
*57:18	1	2	3										13			16																31
*57:19	1	2			5								13																			31
*57:20	1	2	3																	20												31
*57:21	1	2	3										13								21											31
*57:22	1	2	3						9				13				17															31
*57:23	1	2	3										13					18														31
*57:24	1	2									11		13																			31
*57:25, 57:41 ⁴	1	2	3										13						19													31
*57:26	1	2	3										13							20												31
*57:27	1	2	3										13				17															31
*57:28N	1				5								13																			31
*57:29	1	2	3										13	15																		31
*57:30	1	2			5	6							13																			31
*57:31		2	3										13				17															31
*57:32	1		3				6						13																			31
*57:33	1	2	3										13	15							21											31
*57:34	1	2	3										13									22										31
*57:35	1	2	3										13										23									31
*57:36	1	2	3																				23									31
*57:39	1		3	4								12	13															28				31
*57:40	1	w	3										13									22	24									31
*57:43	1	2	3								11		13						19													31
*57:44	1	2	3										13											25								31
*57:45, 57:62, 57:69, 57:89, 57:109	1	2	3																						25							31
*57:48, 57:85, 57:110	1	2	3										13																			31
*57:49, 57:93	1	2	3					8					13																			31
*57:51	1	2	3																									28				31
*57:52	1	2											13																			31
*57:53, 57:76	1		3										13																			31
*57:57	1		3	4					9			12	13				17															31
*57:58	1	2	3										13													26						31
*57:61	1	2											13															27				31
*57:63	1			4	5				9			12	13				17															31
*57:67:01	1	2	3										13										23									31
*57:73	1	2	3										13															28				31
*57:77	1	2	3										13											24								31
*57:78	1	2	3										13																29			31
*57:79N	1	2	3										13																	30	31	
*57:80	1		3	4								12	13													26						31
*57:82-57:83													13																			31
*57:94	1		3	4			7																									

