

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
HLA-B allele ^{1,2,3}																															
*58:01:01:01:58:01:19, 58:32-58:35, 58:37, 58:41-58:42, 58:45:01-58:45:02, 58:47-58:50, 58:53-58:56, 58:62-58:64, 58:67-58:71, 58:73-58:75, 58:77, 58:79-58:81 ⁵	1	2											13																		
*58:01:20	1	2											13															28			
*58:02:01:58:02:02, 58:38, 58:43, 58:60	1		3								11		13																		
*58:04		2		4									13																		
*58:05	1	2			5								13																		
*58:06, 58:46	1		3			6	7						13																		
*58:07	1		3					8				11	13																		
*58:08:01	1					6	7		9				13														27		29		
*58:08:02	1					6	7						13														27		29		
*58:09, 58:76	1	2								10			13														27				
*58:10N, 58:13 ⁴	1	2										12	13																		
*58:11	1	2												14																	
*58:12	1	2											13		15																
*58:14	1	2											13							21											
*58:15	1	2		4									13																		
*58:16:01-58:16:02	1												13															28			
*58:17N	1	2											13				18														
*58:18	1												13					19									28				
*58:19	1	2				6	7						13																		
*58:20	1													14															29		
*58:21, 58:29 ⁴	1	2											13						20												
*58:22	1	2											13				17														
*58:23	1	2							9				13																		
*58:24	1	2								10			13																		
*58:25	1		3								11		13				18														
*58:26	1												13									23									
*58:27	1												13			16													29		
*58:28:01-58:28:02, 58:65	1	2											13		16																
*58:31N	1	2											13									23									
*58:36		2											13		15																
*58:39N		2											13						20					24							
*58:40	1	2											13										24								
*58:44		2							9				13																		
*58:51	1	2											13								22										
*58:52	1	2											13									23			25						
*58:57	1	2											13												25						
*58:58, 58:66 ⁴	1	2											13													26					
*58:59:01-58:59:02	1	2											13																29		
*58:61	1												13																		
*58:72N	1	2							9				13											24							
*58:78	1	2				6							13																		
*07:02:01-07:02:34, 07:02:36-07:04, 07:07-07:08, 07:10, 07:13, 07:15-07:16, 07:19-07:24, 07:26-07:31, 07:33:01-07:33:03, 07:35-07:39, 07:41-07:47, 07:49N-07:50, 07:52, 07:54-07:57, 07:59-07:68:01, 07:68:03, 07:70-07:79, 07:81-07:89, 07:91-07:96:02, 07:98-07:104, 07:106-07:111N, 07:113-07:122, 07:124-07:136:02, 07:139, 07:141-07:151:01, 07:152-07:154, 07:156-07:161N, 07:165-07:175, 07:177-07:181N, 07:183-07:198, 07:200, 07:203-07:205, 07:208, 07:211-07:212, 07:214-07:218, 07:221, 07:224, 07:228-07:234, 07:236, 07:238-07:247, 07:250-07:257, 07:259-07:260, 07:263, 07:265-07:268, 07:272N-07:275, 08:156, 13:31, 14:02:07, 15:01:01:01-15:01:04, 15:01:06-15:01:39, 15:03:01:01-15:03:01:02, 15:03:03, 15:03:05-15:04:04, 15:06, 15:08:01-15:12, 15:14-15:16:03, 15:18:01:01-15:19, 15:23-15:24:02, 15:26N-15:30, 15:32:01-15:35, 15:37-15:39:01, 15:40:01-15:40:02, 15:43, 15:45-15:47:02, 15:49-15:50, 15:53-15:54, 15:56-15:58, 15:60-15:61, 15:63-15:67, 15:69-15:74, 15:76, 15:78:01-15:79N, 15:81-15:82, 15:87, 15:90, 15:92-15:99, 15:101, 15:103-15:104, 15:108-15:110, 15:113, 15:115-15:120, 15:122, 15:125, 15:127-15:129, 15:131-15:135, 15:137-15:138, 15:140-15:143, 15:146-15:150, 15:152-15:153, 15:156-15:161, 15:163-15:164, 15:166-15:167, 15:169, 15:171-15:175, 15:178, 15:180-15:184, 15:187, 15:189-15:193, 15:195, 15:197:01, 15:198-15:199, 15:201-15:203, 15:205-15:206, 15:209N-15:212, 15:215, 15:217, 15:219-15:221, 15:225-15:227, 15:229, 15:231-15:234, 15:236, 15:238-15:239, 15:241-15:249, 15:251-15:264, 15:266-15:267, 15:269-15:270, 15:272N, 15:274, 15:276-15:282, 15:285-15:288, 15:290, 15:292-15:296, 15:298-15:300, 15:303-15:304N, 15:306-15:307, 15:309-15:311, 15:313-15:318, 15:320-15:323, 15:326-15:327, 15:333-15:340, 15:342-15:344, 15:346-15:347, 15:349-15:355, 15:359, 15:362-15:365, 15:367-15:373, 15:375N-15:377Q, 15:379-15:382, 15:384-15:392, 15:395, 15:397-15:398, 18:19, 18:21, 18:30, 18:57, 18:101, 27:04:01-27:04:05, 27:06, 27:10, 27:15, 27:18, 27:20-27:21, 27:25, 27:40, 27:54, 27:63, 27:66N, 27:68-27:69, 27:79, 27:86, 27:91-27:92, 27:100, 27:103, 27:105-27:109, 27:112-27:115, 27:120, 27:138, 27:149, 35:43:01, 35:79, 35:185, 37:07, 39:18, 39:36, 40:23, 40:26, 40:32, 40:51, 40:98, 40:148, 40:158, 40:161, 40:183, 40:198, 46:01:01-46:05, 46:07N-46:08, 46:10, 46:14-46:17, 46:20, 46:22-46:24, 46:26-46:32, 46:34-46:42, 46:44-46:66, 46:68, 47:10, 48:08, 49:01:02, 49:22, 50:47, 54:06, 54:33, 55:21, 55:56, 56:03						6								14																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
*07:02:35, 07:68:02, 07:151:02, 14:12, 14:47, 49:41, 50:32, 54:20, 55:09, 55:52						6	w							14																	

Negative Control

[illegible]

[illegible]

[illegible]

[illegible]