

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23
HLA-B allele ¹																							
*070201-070207	1	2	3	4	5																		
*0703		2	3	4		6																	
*0704	1	2	3		5		7																
*070501-070503	1	2		4	5			8	9														
*0706	1	2		4	5			8															
*0707	1		3	4	5					10													
*0708		2	3	4		6					11												
*0709	1	2	3	4	5							12											
*0710	w	2	3	4	5						11												
*0711	1	2	3	4	5							12			15		17						
*0712	1		3	4	5					10			13					18					
*0713		2	3	4	5									14	15								
*0714	1			4	5					10			13										
*0715	1	2	3	4	5										15								
*0716, 3707 ²		2	3	4		6										16							
*0717	1	2		4	5							12											
*071801	1		3	4	5					10			13										
*071802	1		3	4	5					10			13										
*0719	1				5		7																
*0720	1	2	3	4	5													19				23	
*0721	1	2	3	4	5														20				
*072201-072202	1	2	3	4	5																21		
*0723	1	2	3	4	5																	22	
*0724	1	2	3	4	5																		23
*0725	1	2	3		5		7																
*0726	1	2	3		5									14									
*0727		2	3	4		6										16	17						
*0728	1	2		4	5																		
*0729	1	2	3	4			7																
*0730	1	2	3	4	5																		
*0731	1	2	3	4	5								13										
*0732		2		4		6		8			11												
*0733	1				5																		
*0734	1	2			5			8					13	14									
*0735	1	2	3	4	5													19					
*0736	1	2	3	4	5													18					
*0737		2	3	4												16							
*0738		2	3	4		6					11					16		18					
*0739	1	2	3	4	5						11												
*0740	1				5																		
*0741	1	2	3	4	5																		
*0742	1	2	3	4	5																		
*0743	1	2	3		5								13	14									
*0744	1	2	3	4	5																		
*0745	1	2	3	4	5																		
*0746	1	2	3	4	5												17						
*0747	1	2	3	4	5					10					15								
*0748	1	2	3		5									14									
*0749N	1	2	3	4	5				9														
*0750, 4808 ²		2	3	4		6										16							
*0751	1	2	3	4	5														20				
*0752	1	2	3	4	5		7														21		
*0753	1				5																		
*0754	1	2	3	4	5										15			18					
*0755	1	2	3		5																		
*0756	1	2	3	4	5																		
*0757	1	2	3	4	5										15		17						
*0758	1	2	3	4	5																		
*0759	1	2	3	4	5			8											20				

*0760	1				5															
*0761	1	2	3	4	5															
*0762	1	2	3	4	5															
*0763	1	2	3	4	5															
*0764	1	2	3	4	5															
*0765		2	3	4																
*0766	1	2	3		5															
*0767N	1	2	3	4	5															
*0768	1	2	3	4	5															
*080101-0802, 0805, 0808N-0810, 0812, 0816, 0818, 0819N, 0822-0824,0827, 0829, 0830N, 0832-0834, 0836, 0838, 3587, 4101-410302, 4105, 4107							7													
*0803					7								18							
*0804							7													
*0806													15				17			
*0807, 0811, 4104							7													
*0813													14							
*0814							7													
*0815					7								15							
*0817							7						15		17					
*0820, 0831, 1306, 1542, 1583, 1586, 3560, 3904, 3912, 4051, 4106, 4618, 5121, 5136, 5401, 5405N, 5407, 5408N, 5410, 5412, 5413, 5417, 550101-5503, 5505, 5507, 5511-5513, 5515, 5516, 5519, 5521, 5525, 5526, 5529-5534, 5610, 5623, 5820, 5901-5903																				
*0821							7													
*0825													14							
*0826							7													
*0828, 0837					7				12											
*0835							7				12									
*1301-130204, 1307N-1309, 1311, 1313, 1315-1323, 3588, 4053, 5110, 5116, 5131, 5134, 5208, 5509, 5522, 5524																				
*1314, 4004, 40060101-400602, 4011, 4064, 4070, 4075, 4083, 4086						6							16							
*1401, 1407N, 1408, 180102, 1803, 2712, 2716, 3915, 3925N, 3932, 3933, 3938Q, 3940N, 3943, 4104					6															
*1405																				
*1503, 1547, 1554, 1561, 1562, 1564, 1569, 1598, 9503, 9523, 9527, 9532, 9551, 4440, 4444																				
*1507, 1545, 1555, 9526, 3505, 3522, 3551, 3558, 3572, 3589, 4612, 5314, 5818																				
*151101-151102, 1576, 9501, 9543, 9548, 5406, 5409, 5508, 5603, 5604, 560502, 5615, 5618, 5621, 8301																				
*1514, 3545, 3571, 4410, 4414, 4417, 4443, 4445, 4451, 4462, 4509, 4617, 5123, 5707, 5807, 8201, 8202																19				
*1568					6								16							
*1571, 180101, 180103, 1802, 1805, 1806, 1808-1813, 1815, 1817N-1824, 1827, 1828, 1830-1832, 370101-3706, 3708, 3710, 3711, 3713, 3714, 3808, 4020, 4026, 4028, 4044						6							16							
*1591, 9531																	19			

*9538			3					10		12		14					
*1804, 7801, 780202, 7803																	
*1807, 1829, 2723, 3550, 3584, 5157						6				11				16			
*1814						6								16			
*1826, 3712, 380201-380202, 3804, 3815, 3905, 3911						6								16			
*2701						5				11				16			
*2702-270402, 270502-270508, 270510, 2706, 2708-2710, 2713-2715, 2717, 2719, 2721, 2725, 2726, 2728, 2730, 2731, 2735, 2737, 2738, 2740-2742						5				11							
*270509						5											
*2707, 2711, 2724, 2733, 2743						5				11							
*2718, 2729, 4802						6								16			
*2720, 2727						5				11							
*2732						5				11							
*2734						5				11							
*2736						5				11			14				
*2739										11							
*3531																	
*3566		2	3	4						12					18		
*3709, 4003, 4009, 4018, 4024, 4089						6								16			
*380101, 3809-3812, 3816						6								16	18		
*380102						6								16	18		
*3803, *390201-390202, 3923, 3939						6											
*3805															18		
*3806, 3807						6				11				16	18		
*3813, 3814						6									18		
*3817						6								16	17		
*39010101-39010102L, 390103-390106, 390601-390602, 3909, 3918-391902, 3922, 3926, 3928, 3930, 3931, 3934-3936, 3941, 3942, 3944, 3945						6											
*3903, 3914, 3924						6											
*3907						?								?			
*3908, 391301-391302						6								16			
*3910, 3916						6											
*3917						6											
*3920						6							15	16	17		
*3927						6							15				
*3929						6											
*3937						6								16			
*400101-400106, 4007, 401401-401403, 4022N, 4025, 4036, 4043, 4047-4049, 4054, 4055, 4061, 4062, 4065, 4066, 4069, 407201-407202, 4076, 4077, 4081, 4084, 4087, 4088							8		10			14					
*400201-400203, 4019, 4027, 4029, 4037, 4040, 4056, 4057, 4078, 4082, 4085, 4090, 4091						6								16			
*4005						6								16			
*4008, 4013						6				11				16			
*4010, 4021							8		10				14				
*4012, 4046							8		10				14				
*4015, 4016		2		4			8										
*4023				4			8		10								
*4030, 4034							8					13	14				
*4031		2	3										14				
*4032		2	3	4													
*4033			3						10				14				

[illegible]

[illegible]

24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45
						30									39						
						30									39						
															39						
																				44	
															39						
															39						
							31														
	25	26																			
	25	26																			
24																					
	25																				45
		26																			
			27																		
				28																	
					29																
						30															
							31														
					29			32							39						
24																					
						30									39						
			27																		
		26																			
					29																
					29																
	25																				
24																					
																					45
		26																			
						30	31								39						
							31														
																40					
																	41				

[illegible]

[illegible]

24						30	31									43	
						30											
							31	32									
24																	
24						30											
						30	31										
24																	
						30	31										
24																	
24																	
24																	
						30											
						30											
24	25	26					31									43	
24						30											
24						30											
24						30											
24						30											45
24						29			32								
									32								
24						29			32					39			
									32					39			
									32								
									32					39			
							31	32						39			
24																	
						30											
						30											
24						30											
24	25	26					31									43	
24																	
24							30										45
							30										
24							30	31									45
							30										45
							30										45
24							30										45
							30										45
24							30										45
							30										45
24							30	31									45
24							30	31									45
														40			

[illegible]