

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

HLA-B*07 SSP subtyping

Well¹⁰

Well ¹⁰																																																																																
41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80																																									
90	195	140	150	105	425	215	280	200	75	345	290	235	190	260	230	215	440	190	145	205	415	65	210	385	435	230	335	190	125	245	165	85	345	105	170	335	110	65	405																																									
Length of spec.																																																																																
PCR product(s)																																																																																
Length of int. pos. control ¹																																																																																
1070	1070	1070	1070	1070	1070	800	1070	1070	1070	1070	800	800	1070	1070	1070	1070	1070	800	1070	1070	800	1070	1070	1070	1070	800	1070	1070	1070	1070	1070	1070	800	1070	1070	1070	1070	1070	1070																																									
41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80																																									
																																								Well No. HLA-B allele ^{2,3} *07:02:01-07:02:25, 07:02:27, 07:02:29-07:02:37, 07:124, 07:127, 07:132-07:134, 07:136, 07:141-07:145, 07:150-07:152, 07:155, 07:157-07:159, 07:166, 07:170-07:171, 07:173-07:174 *07:02:26 *07:02:28 *07:03																																								
																																								*07:04, 07:146 *07:05:01-07:05:06 *07:06, 07:138, 07:176 *07:07																																								
																																								*07:08 *07:09 *07:10 *07:11																																								
																																								*07:12 *07:13 *07:14 *07:15																																								
																																								*07:16 *07:17 *07:18:01-07:18:02, 07:163 *07:19																																								
																																								*07:20 *07:21 *07:22:01-07:22:02 *07:23																																								
																																								*07:24 *07:25 *07:26, 07:147, 07:164 *07:27																																								
																																								*07:28 *07:29 *07:30 *07:31																																								
																																								*07:32 *07:33 *07:34 *07:35, 07:153																																								
																																								*07:36 *07:37 *07:38 *07:39 *07:40 *07:41																																								
																																								*07:42, 07:118 ⁵ *07:43 *07:44 *07:45 *07:46 *07:47 *07:48																																								
																																								*07:49N, 07:169 ⁶ *07:50, 48:08 ⁴ *07:51 *07:52 *07:53 *07:54																																								
41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	Well No.																																								
41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	Well No.																																								
																																								*07:55																																								
																																								*07:56:01-07:56:02																																								

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

																63																				C*03:124, C*04:49, C*05:15, C*07:262							
																																								C*03:137, C*12:82			
												53																												C*04:77			
				48								53																												C*07:38:01-07:38:02			
																																								C*07:102			
																																								C*08:08			
41	42	43	44	45	46	47	48	49	50	51	52	53	54	55	56	57	58	59	60	61	62	63	64	65	66	67	68	69	70	71	72	73	74	75	76	77	78	79	80	Well No.			