

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-B allele ^{1,2,3}																
*78:01:01-78:01:02	1	2					7			10		12				
*78:02:01	1		3							10		12				
*78:02:02	1		3				7			10		12				
*78:03	1	2		4			7			10		12				
*78:04			3		5					10		12				
*78:05	1		3			6				10		12				
*78:06	1		3			6				10		12	13			
*78:07	1	2					7					12			15	
*78:08	1	2			5		7					12				
*07:02:01-07:02:44, 07:04-07:07, 07:09-07:15, 07:17-07:26, 07:28-07:31, 07:33-01-07:36, 07:39-07:46, 07:48-07:49N, 07:51-07:64, 07:66-07:68:03, 07:70-07:84, 07:86-07:133, 07:135N-07:179, 07:181N-07:205, 07:208-07:218, 07:220-07:227, 07:229-07:231N, 07:233-07:234, 08:123, 08:132, 14:21, 15:76, 15:101, 15:255, 27:01-27:11, 27:13, 27:15, 27:17, 27:19-27:21, 27:24-27:28, 27:30-27:38, 27:40-27:51, 27:53-27:58, 27:60-27:76, 27:78-27:80, 27:82-27:91, 27:93-27:101, 27:103-27:118, 27:120-27:128, 27:131-27:133, 38:26, 40:166, 42:01:01-42:02, 42:05:01-42:06, 42:08-42:10, 42:12-42:20, 44:90, 44:97, 45:06, 46:01:01-46:10, 46:12-46:17, 46:19-46:39, 46:42-46:48, 46:51Q-46:60, 46:62-46:63, 54:06, 54:09, 55:01:07, 55:04, 55:09, 55:14, 55:24, 55:49, 56:03, 56:09-56:12, 56:18, 56:31-56:32, 67:01:01-67:05, 81:01-81:07, 83:01									9							
*07:65		2					7		9			12				
*07:134, 35:76, 35:186, 35:229									9			12				
*07:206		2					7		9							
*08:03, 08:52, 08:78, 40:13, 40:19, 40:117, 40:292, 48:18, 49:18, 57:05, A*24:146														14		
*08:09, 08:84, 13:46															15	
*08:21, 08:25, 13:48, 38:30, 39:32, 39:74, 40:05, 40:26, 40:28, 40:71, 40:174, 44:10, 44:14, 44:62, 44:77, 44:82, 44:184, 44:213, 57:28N										10						
*08:23, 08:54, 08:110, 13:39, 14:18, 15:01:01:01-15:01:04, 15:01:06-15:01:29, 15:01:31-15:08:02, 15:10:01-15:12, 15:14-15:15, 15:18:01-15:21, 15:25:01-15:29, 15:31-15:32:01, 15:33-15:35, 15:37-15:40, 15:44-15:47:02, 15:49-15:56, 15:60, 15:62-15:66, 15:69, 15:72, 15:74-15:75, 15:77-15:82, 15:84-15:88, 15:90-15:93, 15:96-15:99, 15:102-15:114, 15:116-15:129, 15:131-15:147, 15:149N, 15:151-15:154, 15:156, 15:158-15:161, 15:163, 15:165-15:167, 15:169-15:174, 15:176, 15:178-15:179:02, 15:181N-15:188, 15:190N-15:195, 15:197-15:201, 15:203-15:207, 15:209N-15:215, 15:217-15:221, 15:223-15:228, 15:231-15:235, 15:237-15:238, 15:240-15:242, 15:244-15:253, 15:257-15:266, 15:269-15:272N, 15:274-15:290, 15:292-15:293, 15:295-15:302N, 15:304N-15:307, 15:309-15:323, 15:325-15:328, 15:331-15:338, 35:01:39, 35:08:02, 35:19, 35:25-35:26, 35:46-35:47, 35:49, 35:63, 35:207, 35:210, 35:217, 35:222, 35:248, 40:01:01-40:01:10, 40:01:12, 40:01:14-40:01:39, 40:07, 40:10:01-40:10:02, 40:12, 40:14:01-40:16, 40:21-40:23, 40:25, 40:30-40:34, 40:36, 40:38, 40:42, 40:45-40:46, 40:48-40:49, 40:51-40:52, 40:54-40:55, 40:59-40:62, 40:65, 40:67, 40:69, 40:72:01-40:73, 40:76-40:77, 40:79-40:81, 40:84, 40:87:01-40:88, 40:100-40:102, 40:106, 40:108, 40:112-40:114:02, 40:116, 40:118N, 40:121, 40:123-40:126, 40:128-40:130:02, 40:132, 40:134-40:141, 40:146, 40:150-40:156, 40:158, 40:160:01-40:160:02, 40:163, 40:168, 40:170-40:172, 40:175, 40:178-40:179, 40:182-40:183, 40:186-40:187, 40:191-40:199, 40:204, 40:207-40:208, 40:212-40:213, 40:215-40:219, 40:221-40:223, 40:227-40:228, 40:231, 40:233-40:242, 40:245, 40:247, 40:249-40:253, 40:257-40:265N, 40:272-40:273, 40:277-40:282, 40:285-40:286N, 40:288, 41:02:01-41:04, 41:08, 41:10-41:11, 41:13, 41:15, 41:18-41:19, 41:23-41:24, 41:31, 44:09, 44:46, 44:75, 44:129, 44:196, 45:01:01-45:05, 45:07, 45:09-45:11, 45:13, 47:02-47:03, 48:22, 50:01:01-50:01:04, 50:02, 50:04-50:20, 50:31-50:33, 50:35, 50:37-50:41, 57:38, 57:60			3													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*08:32, 18:01:01:01-18:03, 18:05-18:08, 18:10-18:11, 18:13-18:15, 18:17N-18:24, 18:26-18:36, 18:38-18:47, 18:50-18:53, 18:55-18:65, 18:69-18:78, 18:80-18:105, 35:03:12, 35:03:18, 35:29:01-35:29:02, 35:50, 35:84, 35:136, 35:153, 35:197, 35:231, 35:243, 35:249, 35:267, 35:269, 39:19:01-39:19:02												12				

Negative Control

[illegible]

*42:04, 54:20, 55:01:01-55:01:06, 55:01:08-55:01:13, 55:03, 55:05, 55:11, 55:15, 55:17, 55:20-55:21, 55:25, 55:28-55:29, 55:33, 55:36, 55:38, 55:40, 55:44-55:45, 55:52-55:60, 55:64, 55:66, 55:68									9							15
*44:06, 51:42, 51:45, 51:135, 53:06								8			11					
*46:11, 54:03, 56:01:01-56:01:08, 56:07-56:08, 56:14-56:16, 56:20:01-56:20:02, 56:22, 56:24, 56:26-56:29, 56:35-56:36, 56:38N-56:41						5			9	10						
*46:18, 54:01:01-54:02, 54:04-54:05N, 54:07-54:08N, 54:11-54:19, 54:21-54:25, 54:27-54:29, 54:31-54:32, 55:02:01-55:02:08, 55:07, 55:10, 55:12-55:13, 55:16, 55:19, 55:23, 55:26-55:27, 55:30, 55:32, 55:34-55:35, 55:39, 55:41-55:43, 55:46-55:48, 55:50, 55:61-55:63, 55:65, 55:67, 55:69						5			9							15
*46:40		w					7		9			12				
*46:50			3						9							
*46:61, 54:33, 55:08, 55:51, 56:02, 56:04, 56:13, 56:25, 56:30, 56:37, 56:43, 82:01-82:03									9	10						
*49:03						6		8			11					
*49:09	1															
*51:01:01, 51:01:03-51:01:07, 51:01:10-51:01:14, 51:01:16-51:01:30, 51:01:33-51:01:38, 51:01:41-51:01:42, 51:01:44-51:01:49, 51:01:51, 51:03, 51:07:01, 51:08:01-51:08:02, 51:11N-51:12, 51:20, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35, 51:38-51:39, 51:41N, 51:43-51:44N, 51:48-51:49, 51:51-51:53, 51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:72, 51:74-51:77, 51:79-51:80, 51:83-51:84, 51:86-51:88, 51:91, 51:95-51:96, 51:98N-51:100, 51:102, 51:105, 51:107, 51:109-51:111, 51:113-51:115, 51:117-51:118N, 51:120-51:121, 51:123-51:131, 51:133-51:134, 51:136-51:138, 51:140-51:142, 51:145, 51:149N-51:152, 51:154-51:156, 51:158:01-51:171, 51:173Q-51:174, 51:177-51:178N, 51:181, 51:183-51:184N, 58:08:01	1							8		10	11				14	
*51:01:02, 51:01:09, 51:07:02, 51:89, 51:103, 51:116, 51:119, 51:132, 51:176, 51:179, 52:18, 58:08:02	1									10	11					14
*51:01:08, 51:01:15, 51:01:43, 51:13:01-51:14, 51:50, 51:63, 51:97, 51:106, 51:182	1							8		10	11					
*51:01:31-51:01:32, 51:01:39, 51:01:50, 51:16-51:18, 51:82, 51:94	1							8			11					14
*51:01:40, 51:78:01-51:78:02	1									10	11					
*51:02:01, 51:02:04-51:02:05, 51:04, 51:06:01-51:06:02, 51:23, 51:46, 51:56:01-51:56:02, 51:59, 51:64, 51:139, 51:175								8		10	11					14
*51:02:02, 51:148, 51:180, 53:19, 58:28, 58:65										10	11					14
*51:02:03, 51:34, 51:81, 51:146								8			11					14
*51:05					w					10	11					14
*51:09:01, 51:19, 51:73, 51:122	1				5			8		10	11					14
*51:09:02	1				5					10	11					14
*51:10					5			8			11					14
*51:15					5					10	11					
*51:21, 51:101, 51:153	1							8			11					14 15
*51:22	1			4				8		10	11					14
*51:31	1				5			8			11					14
*51:36, 51:143								8			11					14 15
*51:37, 51:85, 51:90, 51:92:01-51:92:02, 51:172	1							8			11					
*51:40, 51:108					5			8		10	11					14
*51:54					w					10	11					
*51:62								8		10	11					
*51:93					5						11					14
*51:104, 51:112, 51:147, 52:02:01	1							8		10						14
*51:144, 58:27											11					14
*51:157					5			8			11					15
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*52:01:01:01-52:01:01:03, 52:01:03, 52:01:05-52:01:08, 52:01:10, 52:01:12-52:01:18, 52:01:20, 52:04-52:05, 52:07, 52:13, 52:15, 52:17, 52:19, 52:22-52:24, 52:27-52:31:01, 52:33-52:36, 52:38, 52:40-52:42	1					6				10	11					14
*52:01:02, 52:01:04, 52:01:09, 52:06:01-52:06:02, 52:21, 52:31:02	1					6		8		10	11					14
*52:01:11, 52:01:21, 52:08	1					6					11					14
*52:01:19	1					6				10						14
*52:02:02, 52:16, 52:32	1									10						14
*52:03					5	6		8		10	11					14
*52:09	1		3			6		8		10	11					14

Negative Control

*52:10:01-52:10:02	1				5	6			10	11			14			
*52:10:03	1				5	6		8	10	11			14			
*52:11						6			10	11			14			
*52:12, 52:26	1					6				11						
*52:14, 52:20, 52:25, 52:39	1					6			10	11						
*52:37	1					6				11			14	15		
*53:05, 53:16, 53:33			3							11						
*53:36									10	11						
*54:10, 55:37					w				9					15		
*54:30, 55:18, 56:23			3		5									15		
*55:31			3						9					15		
*56:05:01	1	2					7		9	10						
*56:05:02	1								9	10						
*56:06	1	2					7		9	10		12				
*56:21	1								9	10			14			
*56:34	1				5				9	10						
*56:42, 56:44			3		5					10						
*57:01:20								8								
*57:02:01-57:03:02, 57:07, 57:09, 57:17, 57:39, 57:42, 57:46, 57:57, 57:63, 57:66, 57:70										10			14			
*59:02					5								14	15		
*73:01-73:02	1				5				9							
A*23:01:14		w						8								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

Negative Control