

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-B allele ^{1,2,3}																
*78:01:01:01-78:01:02	1	2					7			10		12				
*78:02:01	1		3							10		12				
*78:02:02	1		3				7			10		12				
*78:03	1	2		4			7			10		12				
*78:04			3		5					10		12				
*78:05	1		3			6				10		12				
*78:06	1		3			6				10		12	13			
*78:07	1	2					7					12			15	
*78:08	1	2			5		7					12				
*78:09	1		3				7			10		12	13			
*07:02:01:01-07:02:52, 07:04-07:07, 07:09-07:15, 07:17-07:26, 07:28-07:31, 07:33-01-07:36, 07:39-07:46, 07:48-07:49N, 07:51-07:64, 07:66-07:68:03, 07:70-07:84, 07:86-07:133, 07:135N-07:179, 07:181N-07:205, 07:208-07:218, 07:220-07:227, 07:229-07:231N, 07:233-07:294, 08:123, 08:132, 14:21, 14:57, 15:76, 15:101, 15:255, 27:01-27:11, 27:13-27:15, 27:17, 27:19-27:21, 27:24-27:28, 27:30-27:38, 27:40-27:51, 27:53-27:58, 27:60-27:76, 27:78-27:91, 27:93-27:101, 27:103-27:118, 27:120-27:128, 27:130-27:136, 27:138-27:152, 27:154-27:161, 38:26, 40:166, 42:01:01-42:02:01:02, 42:05:01-42:06, 42:08-42:10, 42:12-42:23, 44:90, 44:97, 45:06, 46:01:01-46:10, 46:12-46:17, 46:19-46:31, 46:33-46:39, 46:42-46:48, 46:51Q-46:71, 54:06, 54:09, 54:26, 54:38, 55:01:07, 55:04, 55:08-55:09, 55:14, 55:22, 55:24, 55:49, 55:51, 56:02:01-56:04, 56:09-56:12, 56:18-56:19N, 56:31-56:32, 56:51, 67:01:01-67:07, 81:01-81:08, 82:01-82:03, 83:01									9							
*07:65		2					7		9			12				
*07:134, 35:76, 35:186, 35:229, 46:40:02									9			12				
*07:206		2					7		9							
*08:03, 08:52, 08:78, 15:361, 40:13, 40:19, 40:109, 40:117, 40:292, 40:340, 41:46, 48:18, 49:18:01-49:18:02, 57:02:01-57:03:03, 57:05, 57:07, 57:09, 57:17, 57:39, 57:42, 57:46, 57:57, 57:63, 57:66, 57:70, 57:80, 57:82, 57:84, A*24:146														14		
*08:09, 08:84, 13:46, 13:53, 39:06:01-39:06:05, 39:34, 39:57, 39:62, 39:64, 39:83, 39:90, 39:109, 39:127, 55:78, 59:01:01:01-59:01:01:02, 59:03, 59:05-59:08, C*05:152, C*07:463, C*12:183															15	

ntrol

[illegible]

*15:32:02, 18:125, 35:08:05, 40:01:11, 40:58			3				7												
*15:42, 15:83, 41:01:01-41:01:05, 41:05-41:07, 41:09, 41:12, 41:14, 41:16, 41:20-41:22, 41:25-41:26, 41:28-41:30, 41:32-41:35, 41:37, 41:44, 41:50, 45:08, 50:45, 50:49, 54:30, 55:18, 56:23			3															15	
*15:202, 15:239, 15:376, 35:10, 35:13, 35:16, 35:28, 35:80, 35:226, 35:270, 35:318, 40:185, 40:308			3				6					12							
*18:04:01, 35:42:02			3				7					12							
*18:09, 18:54, 18:67, 18:136, 37:08, 38:06-38:07, 53:01:01-53:04, 53:07-53:15, 53:18, 53:20-53:27, 53:29, 53:31-53:32, 53:34-53:37, 53:40-53:47, 58:01:01:01-58:02:02, 58:04-58:07, 58:09-58:26, 58:29, 58:31N-58:35, 58:37-58:43, 58:45:01-58:63, 58:66-58:90											11								
*18:12:01-18:12:02, 35:69, 37:11, 37:37, 40:209, 40:350							6					12							
*18:49, 18:79, 18:108, 18:132, 35:203												12	13						
*27:52, 37:60										9	11								
*27:102										9							14		
*35:01:10, 35:04:02, 35:251			3			5						12							
*35:27, 35:56			3									12	13						
*35:37, 35:235, 35:265, 35:273			3								10	12							
*35:60, 35:276			3									12						15	
*35:185			3				7					12	13						
*35:329, 53:05:01-53:05:02, 53:16, 53:33			3									11							
*37:05							6					12	13						
*37:14							6					11	w						
*37:58							6												
*40:01:06			3			5													
*42:04, 46:18, 54:01:01-54:02, 54:04-54:05N, 54:07-54:08N, 54:10-54:25, 54:27-54:29, 54:31-54:32, 54:34-54:37, 55:01:01-55:01:06, 55:01:08-55:03, 55:05, 55:07, 55:10-55:13, 55:15-55:17, 55:19-55:21, 55:23, 55:25-55:30, 55:32-55:48, 55:50, 55:52-55:77, 55:79-55:85												9							15
*44:03:40						5													
*44:06, 51:42, 51:45, 51:62, 51:135, 51:220, 53:06									8			11							
*46:11, 46:32, 54:03, 54:33, 56:01:01:01-56:01:09, 56:07-56:08, 56:13-56:16, 56:20:01-56:20:02, 56:22, 56:24-56:30, 56:35-56:41, 56:43, 56:46-56:50, 56:52-56:53												9	10						
*46:40:01			w					7		9			12						
*46:50				3						9									
*49:03							6		8			11							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16			
*51:01:01:01-51:01:01:21, 51:01:03-51:01:07, 51:01:10-51:01:14, 51:01:16-51:01:31, 51:01:33-51:01:38, 51:01:41-51:01:42, 51:01:44-51:01:56, 51:03, 51:07:01, 51:08:01-51:08:02, 51:11N-51:12, 51:17-51:18, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35, 51:38-51:39, 51:41N, 51:43-51:44N, 51:48-51:49, 51:51-51:53, 51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:72, 51:74-51:77, 51:79-51:80, 51:83-51:84, 51:86-51:88, 51:91, 51:94-51:96, 51:98N-51:100, 51:102, 51:105, 51:107, 51:109-51:111, 51:113-51:115, 51:117-51:118N, 51:120-51:121, 51:123-51:131, 51:133-51:134, 51:136-51:138, 51:140-51:142, 51:145, 51:149N-51:151, 51:154-51:156, 51:158:01-51:171, 51:173Q-51:174:02, 51:177-51:178N, 51:181, 51:183-51:184N, 51:186-51:191, 51:193, 51:195-51:196, 51:198, 51:200-51:207, 51:209-51:212, 51:215, 51:219, 51:221-51:222, 58:08:01	1							8		10	11					14			
*51:01:02, 51:01:09, 51:07:02, 51:89, 51:103, 51:116, 51:119, 51:132, 51:176, 51:179, 51:218, 52:18, 58:08:02	1									10	11					14			
*51:01:08, 51:01:15, 51:01:43, 51:13:01-51:14, 51:37, 51:50, 51:63:01-51:63:02, 51:85, 51:90, 51:92:01-51:92:02, 51:97, 51:106, 51:172, 51:182, 51:192, 51:214, 51:216, 51:223	1							8		10	11								
*51:01:32, 51:01:39, 51:16, 51:82, 51:152	1							8			11					14			
*51:01:40, 51:78:01-51:78:02	1									10	11								
*51:02:01:01-51:02:01:02, 51:02:03-51:02:05, 51:23, 51:146, 51:175, 51:199, 51:213								8		10	11					14			
*51:02:02, 51:180										10	11					14			
*51:04, 51:06:01-51:06:03, 51:10, 51:34, 51:46, 51:56:01-51:56:03, 51:59, 51:64, 51:81								8			11					14			
*51:05					w					10	11					14			

Negative Control

*51:09:01, 51:09:03, 51:19, 51:73, 51:122	1				5			8	10	11			14			
*51:09:02	1				5				10	11			14			
*51:15									10	11						
*51:20, 51:208	1	w						8	10	11			14			
*51:21, 51:101, 51:153	1							8		11			14	15		
*51:22	1			4				8	10	11			14			
*51:31	1				5			8		11			14			
*51:36, 51:143								8		11			14	15		
*51:40, 51:108					5			8	10	11			14			
*51:54					w				10	11						
*51:93					5					11			14			
*51:104, 51:112, 51:147, 52:02:01	1							8	10				14			
*51:139, 51:194					5			8		11			14			
*51:144, 51:148, 53:19, 58:27-58:28:02, 58:65										11			14			
*51:157, 51:197								8		11				15		
*51:185	1		3					8	10	11			14			
*51:217	1				w			8	10	11			14			
*52:01:01:01-52:01:01:04, 52:01:03, 52:01:05-52:01:08, 52:01:10-52:01:18, 52:01:20, 52:01:22-52:01:28, 52:04-52:05, 52:06:03-52:07, 52:13, 52:15, 52:17, 52:19, 52:22-52:24, 52:27-52:31:01, 52:33-52:36, 52:38, 52:40-52:61, 52:64-52:66, 52:69	1					6			10	11			14			
*52:01:02:01-52:01:02:03, 52:01:04, 52:01:09, 52:06:01-52:06:02, 52:21, 52:31:02	1					6	8		10	11			14			
*52:01:19	1					6			10				14			
*52:01:21, 52:08	1					6				11			14			
*52:01:29, 52:68	1		3			6			10	11						
*52:02:02, 52:16, 52:32	1								10				14			
*52:03					5	6		8	10	11			14			
*52:09	1		3			6		8	10	11			14			
*52:10:01-52:10:02	1				5	6			10	11			14			
*52:10:03	1				5	6		8	10	11			14			
*52:11						6			10	11			14			
*52:12, 52:14, 52:20, 52:25:01-52:26, 52:39, 52:62-52:63	1					6			10	11						
*52:37	1					6				11			14	15		
*52:67						6				11			14			
*55:31			3						9					15		
*56:05:01	1	2					7		9	10						
*56:05:02, 56:34	1								9	10						
*56:06	1	2					7		9	10		12				
*56:21	1								9	10				14		
*56:45									9	10		12				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*57:01:20								8								
*59:02														14	15	
*73:01-73:02	1								9							
A*23:01:14		w						8								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

Negative Control

NC