

INTERPRETATION TABLE																								
HLA-B*08 SSP subtyping																								
Amplification patterns of the B*08:01 to B*08:106 alleles																								
	Well ¹⁴																							
	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Length of spec.	215	215	245	255	245	205	215	170	165	85	290	385	85	115	75	105	95	425	245	140	465	130	85	215
PCR product(s)							250	215		170			290	280				470	415	205			235	
Length of int. pos. control ¹	800	800	1070	1070	1070	1070	1070	1070	1070	800	800	1070	1070	1070	1070	800	1070	800	1070	1070	1070	1070	800	800
5'-primer(s) ²	363	97	97	97	97	97	97	412	412	475	355	44	355	363	106	539	363	44	44	97	44	636	106	97
	5'-AgC ³	5'-TCg ³	5'-TCg ³	5'-TCg ³	5'-TCg ³	5'-TCg ³	5'-TCg ³	5'-ATA ³	5'-ATg ³	5'-Cg ³	5'-TCC ³	5'-ggC ³	5'-TCA ³	5'-Agg ³	5'-CCA ³	5'-gCT ³	5'-AgC ³	5'-ggC ³	5'-ggC ³	5'-TCg ³	5'-ggC ³	5'-CAC ³	5'-CCA ³	5'-TCT ³
										560	357		560	363						363			560	
										5'-CgA ³	5'-Tgg ³		5'-CgC ³	5'-Agg ³	530					5'-AgC ³			5'-CCT ³	
														5'-ggT ³										
3'-primer(s) ³	538	272	299	309	292	251	269	539	538	603	603	259	603	603	142	603	418	299	119	261	341	724	302	272
	5'-gTC ³	5'-TgA ³	5'-TCT ³	5'-gTg ³	5'-gTA ³	5'-CCg ³	5'-Agg ³	5'-TCC ³	5'-gTC ³	5'-gTg ³	5'-gTg ³	5'-gTC ³	5'-gTg ³	5'-gTg ³	5'-TgA ³	5'-gTg ³	5'-gTC ³	5'-TCA ³	5'-ggg ³	5'-gTC ³	5'-CgT ³	5'-CTA ³	5'-ggT ³	5'-TgA ³
			299		309	272	302											343	289	461			603	
			5'-TCT ³		5'-ATC ³	5'-Tgg ³	5'-ggT ³											5'-T ³	5'-AgC ³	5'-gCg ³			5'-gTg ³	
							307																	
							5'-CCA ³																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{4,5}																								
*08:01:01-08:01:20, 08:71, 08:73, 08:75, 08:77, 08:80-08:81, 08:83, 08:88, 08:90-08:93, 08:96-08:99, 08:101, 08:104-08:106	1	2	3												15									
*08:02	1	2		4											15									
*08:03, 08:78	1	2			5		7								15									
*08:04	1		3			6									15									
*08:05	1	2	3				7								15									
*08:07	1	2	3						9						15									
*08:08N	1	2	3							10					15									
*08:09, 08:84		2	3								11				15									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Length of spec.	215	215	245	255	245	205	215	170	165	85	290	385	85	115	75	105	95	425	245	140	465	130	85	215
PCR product(s)							250			170			290	280				470	415	205			235	

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]

[illegible]