

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.								
HLA-B allele ^{1,2,3}	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	HLA-B allele ^{1,2,3}								
*08:01:01-08:01:31, 08:30N, 08:73, 08:75, 08:77, 08:80-08:81, 08:83, 08:88, 08:90-08:93, 08:96-08:99, 08:106, 08:112, 08:114-08:115, 08:118-08:121, 08:124-08:125, 08:127, 08:130, 08:138, 08:141-08:142, 08:144, 08:147, 08:149, 08:152-08:155 ⁶	1	2	3												15																32																		*08:01:01-08:01:31, 08:30N, 08:73, 08:75, 08:77, 08:80-08:81, 08:83, 08:88, 08:90-08:93, 08:96-08:99, 08:106, 08:112, 08:114-08:115, 08:118-08:121, 08:124-08:125, 08:127, 08:130, 08:138, 08:141-08:142, 08:144, 08:147, 08:149, 08:152-08:155 ⁶								
*08:02, 08:117	1	2		4											15																	32																		*08:02, 08:117							
*08:03	1	2			5		7								15																	32																47	*08:03								
*08:04	1		3			6									15																	32																			*08:04						
*08:05	1	2	3				7								15																	32																			*08:05						
*08:07	1	2	3					9							15																		32																			*08:07					
*08:08N	1	2	3						10						15																		32																			*08:08N					
*08:09		2	3							11					15																		32																			*08:09					
*08:10	1	2	3								12				15																		32																			*08:10					
*08:11, 08:76 ⁵	1	2	3										13		15																		32																				*08:11, 08:76 ⁵				
*08:12:01-08:12:03			2	3										14	15																		32																				*08:12:01-08:12:03				
*08:13, 08:143			2	3											15	16																	32																				*08:13, 08:143				
*08:14, 08:87	1	2	3												15		17																32																				*08:14, 08:87				
*08:15	1	2													15			18															32																				*08:15				
*08:16			2	3										14	15				19														32																				*08:16				
*08:17	1		3			6									15					20			23										32																				*08:17				
*08:18	1	2	3												15						21												32																				*08:18				
*08:19N, 08:109 ⁵	1	2	3												15						22												32																				*08:19N, 08:109 ⁵				
*08:20			2	3				8							15							22											32			36																*08:20					
*08:21, 08:47 ⁵	1	2	3												15							23											32																				*08:21, 08:47 ⁵				
*08:22	1	2	3												15					20							27						32																				*08:22				
*08:23	1	2	3		5										15																			32																				*08:23			
*08:24	1	2	3			6									15																			32																				*08:24			
*08:25			2	3											15	16						23												32																				*08:25			
*08:26:01-08:26:03	1																					24											32							40													*08:26:01-08:26:03				
*08:27	1	2	3							11					15																			32																				*08:27			
*08:28	1	2	3												15										25																													*08:28			
*08:29	1	2	3												15												27							32																				*08:29			
*08:31	1	2	3												15													28																										*08:31			
*08:32	1	2	3												15														29					32																				*08:32			
*08:33, 08:53:01-08:53:02 ⁵	1	2	3												15																			32			36																	*08:33, 08:53:01-08:53:02 ⁵			
*08:34	1	2	3												15																		31	32																				*08:34			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.								
*08:35	1	2	3												15										25							32																							*08:35		
*08:36	1	2	3				7								15														29			32																								*08:36	
*08:37	1	2	3					9							15										25																																*08:37

[illegible]

[illegible]

[illegible]

*07:29	3								15								24								39								43								40								Negative Control	*07:29
*07:32, 07:207, 35:18, 35:44, 51:05, 51:29, 51:54, 51:61:01-51:61:02, 51:82	8																																																	*07:32, 07:207, 35:18, 35:44, 51:05, 51:29, 51:54, 51:61:01-51:61:02, 51:82
*07:55, 07:100, 07:113, 15:07:01-15:07:03, 15:45, 15:68, 15:126, 15:207, 15:331, 46:12, C*02:60																																									43									*07:55, 07:100, 07:113, 15:07:01-15:07:03, 15:45, 15:68, 15:126, 15:207, 15:331, 46:12, C*02:60
*07:56:01-07:56:02, 07:72, 38:28, 39:63, 49:03																																	40																	*07:56:01-07:56:02, 07:72, 38:28, 39:63, 49:03
*07:86, 14:11, 15:03:01:01-15:03:05, 15:09-15:10:04, 15:18:01-15:18:06, 15:23, 15:37, 15:47:01-15:47:02, 15:49, 15:52, 15:54, 15:61-15:62, 15:64:01-15:64:02, 15:69, 15:72, 15:74, 15:80, 15:90-15:91, 15:93, 15:98-15:99, 15:103, 15:108, 15:114-15:115, 15:119, 15:123-15:124, 15:127, 15:131-15:134, 15:151, 15:153, 15:156, 15:158, 15:161, 15:173, 15:176, 15:186, 15:197-15:198, 15:200, 15:210, 15:220-15:221, 15:226N, 15:229, 15:235, 15:238, 15:242-15:243, 15:251-15:253, 15:255, 15:263, 15:266, 15:274-15:275:02, 15:281-15:282, 15:290, 15:292-15:294N, 15:306-15:307, 15:311-15:314, 15:323, 15:329, 15:335, 15:337-15:338, 15:351, 15:354, 15:369, 18:25, 18:68, 38:33, 39:04, 40:12, 48:22, 52:16, 52:27, 54:21, 56:16, 57:58, 57:80, 58:12									w																																									*07:86, 14:11, 15:03:01:01-15:03:05, 15:09-15:10:04, 15:18:01-15:18:06, 15:23, 15:37, 15:47:01-15:47:02, 15:49, 15:52, 15:54, 15:61-15:62, 15:64:01-15:64:02, 15:69, 15:72, 15:74, 15:80, 15:90-15:91, 15:93, 15:98-15:99, 15:103, 15:108, 15:114-15:115, 15:119, 15:123-15:124, 15:127, 15:131-15:134, 15:151, 15:153, 15:156, 15:158, 15:161, 15:173, 15:176, 15:186, 15:197-15:198, 15:200, 15:210, 15:220-15:221, 15:226N, 15:229, 15:235, 15:238, 15:242-15:243, 15:251-15:253, 15:255, 15:263, 15:266, 15:274-15:275:02, 15:281-15:282, 15:290, 15:292-15:294N, 15:306-15:307, 15:311-15:314, 15:323, 15:329, 15:335, 15:337-15:338, 15:351, 15:354, 15:369, 18:25, 18:68, 38:33, 39:04, 40:12, 48:22, 52:16, 52:27, 54:21, 56:16, 57:58, 57:80, 58:12
*07:92-07:93, 38:27, 39:27, 40:73																	18																																	*07:92-07:93, 38:27, 39:27, 40:73
*07:102																									21																									*07:102
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.	
*07:160																									19																								*07:160	
*07:186	3																																								39								*07:186	
*07:195, 40:281																																	27																*07:195, 40:281	
*07:206									8																																40								*07:206	
*07:253																	w																																*07:253	
*13:39, 13:67, 15:57, 15:135, 55:12, 56:41, 78:06, C*05:30																									23																								*13:39, 13:67, 15:57, 15:135, 55:12, 56:41, 78:06, C*05:30	

[illegible]

[illegible]

[illegible]