































<p>*07:01:01:01-07:01:02:08, 07:01:04, 07:01:06-07:01:12, 07:01:14-07:01:24, 07:01:26-07:01:35, 07:01:37-07:01:45, 07:01:47-07:01:67, 07:01:69-07:01:73, 07:01:75-07:01:93, 07:44, 07:58-07:59, 07:65, 07:70, 07:77-07:78:02, 07:81-07:83, 07:86, 07:89, 07:91, 07:93:01-07:95, 07:98N, 07:103, 07:106, 07:111, 07:113, 07:115-07:116, 07:118-07:120, 07:122, 07:124, 07:128-07:129, 07:131:01-07:132, 07:134, 07:150Q-07:151, 07:153, 07:162, 07:164N-07:166, 07:170, 07:176, 07:180, 07:182, 07:188-07:191N, 07:197, 07:200-07:201, 07:203-07:207, 07:212, 07:214-07:215, 07:222-07:224, 07:227N-07:228, 07:235Q-07:237, 07:246:01-07:246:02, 07:249, 07:253-07:257:04, 07:266-07:269, 07:276-07:282, 07:293, 07:296-07:301, 07:303, 07:310-07:311, 07:313, 07:331-07:332, 07:337, 07:362, 07:366, 07:369-07:371, 07:377, 07:385-07:387, 07:407-07:408, 07:410, 07:412, 07:417, 07:421-07:422, 07:424, 07:427, 07:430-07:431, 07:435, 07:440-07:444, 07:448, 07:450, 07:453-07:455, 07:458, 07:460, 07:462, 07:468-07:471, 07:475, 07:479, 07:481, 07:488, 07:490-07:492, 07:499, 07:502, 07:506, 07:508, 07:515, 07:521:01, 07:527, 07:531, 07:537, 07:540, 07:545-07:547, 07:549-07:550, 07:554, 07:557, 07:561, 07:570-07:571, 07:581, 07:588, 07:591, 07:602-07:604, 07:610, 07:614, 07:617, 07:619, 07:623-07:624, 07:633N-07:636, 07:638, 07:642-07:643, 07:647, 07:649-07:650:02, 07:652-07:654, 07:657-07:658, 07:660, 07:662-07:663Q, 07:669-07:670, 07:673, 07:677, 07:683, 07:685, 07:690N-07:691, 07:694, 07:696, 07:699, 07:701, 07:705, 07:708, 07:712-07:714, 07:718, 07:720, 07:728, 07:732, 07:734, 07:749N-07:750N, 07:757, 07:762, 07:771, 07:774, 07:777-07:778, 07:782, 07:787N-07:788, 07:794-07:796N, 07:800, 07:802, 07:804N, 07:806, 07:808N, 07:811, 07:813-07:814, 07:816, 07:820N, 07:823-07:825, 07:832-07:833, 07:843-07:844</p>							<p>*07:01:01:01-07:01:02:08, 07:01:04, 07:01:06-07:01:12, 07:01:14-07:01:24, 07:01:26-07:01:35, 07:01:37-07:01:45, 07:01:47-07:01:67, 07:01:69-07:01:73, 07:01:75-07:01:93, 07:44, 07:58-07:59, 07:65, 07:70, 07:77-07:78:02, 07:81-07:83, 07:86, 07:89, 07:91, 07:93:01-07:95, 07:98N, 07:103, 07:106, 07:111, 07:113, 07:115-07:116, 07:118-07:120, 07:122, 07:124, 07:128-07:129, 07:131:01-07:132, 07:134, 07:150Q-07:151, 07:153, 07:162, 07:164N-07:166, 07:170, 07:176, 07:180, 07:182, 07:188-07:191N, 07:197, 07:200-07:201, 07:203-07:207, 07:212, 07:214-07:215, 07:222-07:224, 07:227N-07:228, 07:235Q-07:237, 07:246:01-07:246:02, 07:249, 07:253-07:257:04, 07:266-07:269, 07:276-07:282, 07:293, 07:296-07:301, 07:303, 07:310-07:311, 07:313, 07:331-07:332, 07:337, 07:362, 07:366, 07:369-07:371, 07:377, 07:385-07:387, 07:407-07:408, 07:410, 07:412, 07:417, 07:421-07:422, 07:424, 07:427, 07:430-07:431, 07:435, 07:440-07:444, 07:448, 07:450, 07:453-07:455, 07:458, 07:460, 07:462, 07:468-07:471, 07:475, 07:479, 07:481, 07:488, 07:490-07:492, 07:499, 07:502, 07:506, 07:508, 07:515, 07:521:01, 07:527, 07:531, 07:537, 07:540, 07:545-07:547, 07:549-07:550, 07:554, 07:557, 07:561, 07:570-07:571, 07:581, 07:588, 07:591, 07:602-07:604, 07:610, 07:614, 07:617, 07:619, 07:623-07:624, 07:633N-07:636, 07:638, 07:642-07:643, 07:647, 07:649-07:650:02, 07:652-07:654, 07:657-07:658, 07:660, 07:662-07:663Q, 07:669-07:670, 07:673, 07:677, 07:683, 07:685, 07:690N-07:691, 07:694, 07:696, 07:699, 07:701, 07:705, 07:708, 07:712-07:714, 07:718, 07:720, 07:728, 07:732, 07:734, 07:749N-07:750N, 07:757, 07:762, 07:771, 07:774, 07:777-07:778, 07:782, 07:787N-07:788, 07:794-07:796N, 07:800, 07:802, 07:804N, 07:806, 07:808N, 07:811, 07:813-07:814, 07:816, 07:820N, 07:823-07:825, 07:832-07:833, 07:843-07:844</p>
Well No.							Well No.
C allele <sup>1,2,3</sup>	1	2	3	4	5	6	C allele <sup>1,2,3</sup>





HLA-C High 2020 9K7

Well No.																																																	Well No.											
C allele <sup>1,2,3</sup>	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	Well No.
*07:02:09																																																												*07:02:09
*07:02:32																																																												*07:02:32
*07:02:42, 07:291																																																												*07:02:42, 07:291
*07:02:63, 07:290, 07:484N																																																												*07:02:63, 07:290, 07:484N
*07:02:65, 07:02:94, 07:02:98, 07:171, 07:258, 07:396, 07:558:01-01-07:558:01:02, 07:609, 07:680-07:681, 07:710																																																												*07:02:65, 07:02:94, 07:02:98, 07:171, 07:258, 07:396, 07:558:01-01-07:558:01:02, 07:609, 07:680-07:681, 07:710
*07:02:75, 07:679 <sup>4</sup>																																																												*07:02:75, 07:679 <sup>4</sup>
*07:03																																																												*07:03
*07:04:01:01-07:04:10, 07:04:13-07:04:19, 07:63, 07:139, 07:142, 07:199:01-07:199:02, 07:272, 07:323-07:324, 07:329N, 07:354, 07:358, 07:365, 07:378, 07:394-07:395, 07:420, 07:426, 07:428, 07:459, 07:466-07:467, 07:480, 07:487, 07:501, 07:523, 07:534-07:535, 07:552, 07:569, 07:585-07:586, 07:600:01N-07:600:02N, 07:622, 07:651, 07:655, 07:664, 07:672N, 07:674, 07:693, 07:742, 07:751N, 07:780, 07:797N, 07:831, 07:838-07:839N																																																												*07:04:01:01-07:04:10, 07:04:13-07:04:19, 07:63, 07:139, 07:142, 07:199:01-07:199:02, 07:272, 07:323-07:324, 07:329N, 07:354, 07:358, 07:365, 07:378, 07:394-07:395, 07:420, 07:426, 07:428, 07:459, 07:466-07:467, 07:480, 07:487, 07:501, 07:523, 07:534-07:535, 07:552, 07:569, 07:585-07:586, 07:600:01N-07:600:02N, 07:622, 07:651, 07:655, 07:664, 07:672N, 07:674, 07:693, 07:742, 07:751N, 07:780, 07:797N, 07:831, 07:838-07:839N
*07:04:11																																																												*07:04:11
*07:04:12, 07:336, 07:357, 07:361, 07:698																																																												*07:04:12, 07:336, 07:357, 07:361, 07:698
*07:05																																																												*07:05
*07:06:01:01-07:06:01:08, 07:06:03, 07:330:02, 07:613, 07:620, 07:706, 07:721, 07:760, 07:803, 07:827, 07:841																																																												*07:06:01:01-07:06:01:08, 07:06:03, 07:330:02, 07:613, 07:620, 07:706, 07:721, 07:760, 07:803, 07:827, 07:841
*07:06:02, 07:776N																																																												*07:06:02, 07:776N
*07:07																																																												*07:07
*07:08																																																												*07:08
*07:09																																																												*07:09
*07:10																																																												*07:10
*07:11																																																												*07:11
*07:12																																																												*07:12
*07:13:01:01-07:13:01:02																																																												*07:13:01:01-07:13:01:02
*07:14																																																												*07:14
*07:16																																																												*07:16
*07:17:01:01-07:17:02, 07:17:04, 07:599																																																												*07:17:01:01-07:17:02, 07:17:04, 07:599
*07:17:03																																																												*07:17:03
*07:18:01:01-07:18:05, 07:71, 07:517, 07:607, 07:621, 07:687, 07:805, 07:815, 07:819Q																																																												*07:18:01:01-07:18:05, 07:71, 07:517, 07:607, 07:621, 07:687, 07:805, 07:815, 07:819Q
*07:19																																																												*07:19
*07:20:01, 07:263																																																												*07:20:01, 07:263
*07:20:02																																																												*07:20:02
*07:21																																																												*07:21
Well No.																																																	Well No.											
C allele <sup>1,2,3</sup>	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	Well No.										
*07:22, 07:35, 07:404 <sup>4</sup>																																																												*07:22, 07:35, 07:404 <sup>4</sup>
*07:23																																																												*07:23
*07:24, 07:579																																																												*07:24, 07:579
*07:25, 07:137:01, 07:138, 07:352 <sup>4</sup>																																																												*07:25, 07:137:01, 07:138, 07:352 <sup>4</sup>
*07:26:01:01-07:26:03, 07:821N																																																												*07:26:01:01-07:26:03, 07:821N

Negative control



















HLA-C High 2020 9K7

Well No.	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	Well No.	
*15:38																																																														*15:38
*15:40																																																														*15:40
*15:43																																																														*15:43
*15:45, 15:67																																																														*15:45, 15:67
*15:51, 15:75, 15:84Q																																																														*15:51, 15:75, 15:84Q
*15:55																																																														*15:55
*15:61																																																														*15:61
*15:62																																																														*15:62
*15:63																																																														*15:63
*15:64, 15:66, 15:76, 15:105Q																																																														*15:64, 15:66, 15:76, 15:105Q
*15:65																																																														*15:65
*15:71																																																														*15:71
*15:74																																																														*15:74
*15:77																																																														*15:77
*15:83, 15:184																																																														*15:83, 15:184
*15:85																																																														*15:85
*15:87																																																														*15:87
*15:90																																																														*15:90
*15:100																																																														*15:100
*15:102																																																														*15:102
*15:107																																																														*15:107
*15:110																																																														*15:110
*15:113																																																														*15:113
*15:114																																																														*15:114
*15:116																																																														*15:116
*15:124, 15:192																																																														*15:124, 15:192
*15:130																																																														*15:130
*15:135																																																														*15:135
*15:138																																																														*15:138
*15:143																																																														*15:143
*15:144																																																														*15:144
*15:148																																																														*15:148
*15:158																																																														*15:158
*15:169																																																														*15:169
*15:171																																																														*15:171
*15:178																																																														*15:178
*15:181																																																														*15:181
*15:190																																																														*15:190
*15:203																																																														*15:203
*15:208																																																														*15:208
Calle1 <sup>1,2,3</sup>	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	1	2	3	4	5	6	7	8	9	0	Well No.	
*16:01:01:01-16:01:01:15, 16:01:03-16:01:04, 16:01:07-16:01:15, 16:01:18-16:01:25, 16:01:27-16:01:29, 16:01:31-16:01:36, 16:13-16:14, 16:16Q-16:18, 16:23-16:24, 16:26, 16																																																														







