

KIR Genotyping 2019 3K9 R03.xlsx

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27
KIR allele ^{1, 2, 3}																											
2DL1*001-038	1																										
2DL2*0010101-015		2																									
2DL3*0010101-01102, 013N-026, 028-035			3																								
2DL3*01201-01202, 027			w																								
2DL4*00101-00602, 010, 01201-01202, 014-016, 018, 021-026, 028-034, 036-041				4																				24			
2DL4*007-00902, 011, 013, 017, 019, 020, 027, 035				4																							
2DL5A*0010101-00105, 0050101, 0050103-0050104, 01201-01202					5	6																				26	
2DL5A*0050102					5	6																					
2DL5A*014-015, 021-024					5	6																				?	
2DL5B*0020101-0020104, 0020107, 004, 00601, 00603-0070101, 0080101-00802, 009-01302					5		7																		25		
2DL5B*0020106, 00202, 0070102, 00803, 01303, 016-020					5		7																		?		
2DL5B*003, 00602					5		7																				
2DS1*001								8													21						
2DS1*0020101-009								8														22					
2DS2*0010101-009									9																		
2DS3*00101-008										10																	
2DS4*0010101-00106, 01101-01102, 014-017, 019-020											11																
2DS4*0030101-0030104, 0040101-0040102, 0060101-0060102, 007-010, 012, 013, 018												12															
2DS5*001-003-00502, 00801-009, 011-017													13												?		
2DS5*0020101-0020104													13													26	
2DS5*006, 007, 010													13														
3DL1*0010101-002, 0050101-009, 01501-018, 020010-0200102, 022-035, 038, 040-044, 051-054, 057, 059-077, 079-081N, 086-090, 092-0103, 0109, 111-116, 118														14													
3DL1*00401-00404, 019, 021, 036, 037, 039, 056, 072, 91, 110, 117														14									23				
3DL2*0010101-112															15												
3DL3*00101-057																16											
3DS1*010-014, 045-049N, 050, 055, 058, 082-085, 104-108																	17										
3DS1*078														14			17										
2DP1*00101-004, 006-024																		18									
3DP1*001-002, 007, 0090101-00902																			19			22			25		
3DP1*0030101-0030402, 005, 006, 008, 010																				19	20						
3DP1*004																				19		22			25	26	
3DP1*011-012																				19		22			?		
3DP1*013-015																				19	20				?		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27

Negative Control