

KIR HLA Ligand 2021 1N2

Well No.	1	2	3	4	5	6	7	8
HLA allele ^{1,2}								
A*01:95, 01:289, 01:298, 02:81, 02:87, 02:112, 02:124, 02:129, 02:136, 02:571, 02:829, 03:152, 03:219, 11:178, 11:190, 23:01:01:01-23:68, 23:70-23:89, 23:91N-23:109, 24:02:01:01-24:03:04, 24:05:01-24:11N, 24:13:01-24:15, 24:17:01:01-24:18, 24:20:01:01-24:27, 24:29-24:43, 24:45N-24:64, 24:66-24:88, 24:90:01N-24:99, 24:101-24:108, 24:110-24:128, 24:130-24:210, 24:212-24:240N, 24:242-24:289, 24:291-24:372, 24:374-24:405, 24:407-24:423, 24:425N-24:470, 24:472-24:515, 25:01:01:01-25:58, 25:60-25:70, 29:13, 31:07-31:08, 31:10, 32:01:01:01-32:146, 68:36, 68:249					5			
B*07:02:01:01-07:02:77, 07:02:79-07:26, 07:28-07:35, 07:37:01-07:37:02, 07:39-07:80, 07:82-07:148, 07:150-07:179, 07:181N-07:191, 07:193-07:218, 07:220-07:235, 07:237-07:272N, 07:274-07:409, 08:01:01:01-08:01:56, 08:04:01-08:05, 08:07-08:51, 08:53:01-08:77, 08:79-08:116, 08:118-08:125, 08:127-08:130, 08:132-08:135, 08:137-08:138, 08:140-08:152, 08:154-08:156, 08:158-08:254, 08:256-08:263, 13:09, 13:39, 13:67, 14:01:01:01-14:92, 15:01:01:01-15:01:04, 15:01:06-15:12:03, 15:14-15:15:01:02, 15:18:01:01-15:21:01:02, 15:25:01:01-15:35, 15:37-15:40:02, 15:42, 15:44-15:58, 15:60-15:66, 15:68-15:86, 15:88, 15:90-15:94N, 15:96-15:99, 15:101-15:114, 15:116-15:129, 15:131-15:156, 15:158-15:161, 15:163-15:167, 15:169-15:176, 15:178-15:195, 15:197:01-15:207, 15:209N-15:215, 15:217-15:221, 15:223-15:229, 15:231-15:253, 15:255, 15:257-15:267, 15:269-15:272N, 15:274-15:276, 15:278-15:338, 15:340-15:355, 15:357-15:360, 15:363:01-15:376, 15:378-15:395, 15:397-15:402, 15:404-15:407, 15:409-15:410, 15:412-15:417, 15:419-15:422, 15:425-15:429, 15:431-15:441, 15:443-15:445, 15:447-15:461, 15:463N-15:499, 15:501-15:504, 15:506-15:515, 15:517-15:522, 15:524-15:531, 15:533-15:545, 15:547-15:549N, 15:551-15:554, 15:556-15:574, 18:01:01:01-18:08, 18:10-18:15, 18:17N-18:53, 18:55-18:66, 18:68-18:135:01:02, 18:137-18:178, 18:180-18:193, 18:196-18:197, 27:08, 27:12:01:01-27:12:01:03, 27:18, 27:26, 27:33, 27:40, 27:42, 27:44, 27:89, 27:153, 27:204, 35:01:01:01-35:01:38, 35:01:40-35:02:08, 35:02:10-35:43:01, 35:43:03-35:72, 35:74-35:221, 35:223-35:247, 35:249-35:292, 35:294-35:328, 35:330-35:423, 35:425-35:470, 35:472-35:496, 37:05, 37:11, 37:37, 37:67, 38:156-38:159, 39:01:01:01-39:01:01:15, 39:01:03:01-39:01:14, 39:01:16-39:20, 39:22-39:103, 39:105-39:169, 40:01:01-40:12:02, 40:14:01-40:16:01:02, 40:18, 40:20:01:01-40:40, 40:42-40:46, 40:48-40:75, 40:77-40:95, 40:97-40:108, 40:111-40:116, 40:118N-40:156, 40:158-40:187, 40:189-40:200, 40:202-40:291N, 40:293-40:339, 40:341-40:393, 40:395-40:399N, 40:401-40:468, 41:01:01:01-41:25, 41:27-41:45N, 41:47-41:59, 41:61-41:66, 42:01:01:01-42:02:02, 42:04-42:21, 42:23-42:28, 44:09, 44:46:01-44:46:02, 44:75, 44:90, 44:129, 44:131, 44:254, 44:277, 44:438N, 45:01:01:01-45:01:08, 45:01:10-45:15, 45:17-45:25, 45:27, 46:01:01:01-46:84, 47:02, 48:01:01:01-48:17, 48:19-48:50, 50:01:01:01-50:02:01:02, 50:04-50:20, 50:31-50:72N, 54:01:01:01-54:11, 54:13-54:43, 55:01:01:01-55:05, 55:07-55:89N, 55:91-55:102, 55:104-55:110, 56:01:01:01-56:06, 56:08-56:20:02, 56:22-56:45, 56:47-56:64, 56:66-56:74, 56:76, 57:12, 58:64, 67:01:01-67:07, 73:01:01:01-73:03, 78:01:01:02-78:10, 81:01:01:01-81:10, 82:01:01:01-82:04, 83:01							7	
B*07:27, 07:236, 07:273, 08:126, 15:43, 18:54, 18:195, 27:03-27:05:36, 27:05:38-27:07:06, 27:09-27:11, 27:13:01-27:17, 27:19:01:01-27:21:02, 27:23-27:25, 27:27-27:29, 27:31-27:32, 27:34-27:39, 27:41, 27:43, 27:45-27:48, 27:50:01-27:52, 27:54-27:56, 27:58-27:61, 27:63-27:64N, 27:66N-27:74, 27:76, 27:78-27:82, 27:84-27:88, 27:90:01-27:94N, 27:96:01-27:101, 27:103-27:118, 27:120-27:125, 27:127-27:133, 27:135-27:141, 27:143-27:152, 27:154-27:155, 27:158-27:162, 27:164, 27:166-27:170, 27:173-27:175, 27:177-27:180, 27:182-27:187, 27:189-27:196, 27:198-27:202, 27:205-27:212N, 27:214, 27:216-27:231, 27:233-27:235, 35:329, 37:01:01:01-37:01:12, 37:01:03-37:04:02, 37:06:01-37:09, 37:12-37:13, 37:15-37:33N, 37:35-37:36, 37:38-37:66, 37:68-37:95, 38:17, 40:188, 44:257, 47:01:01:02-47:01:05, 47:05-47:10, 53:03, 53:55, 55:90, 56:46						6		
B*07:36, 07:38, 07:81, 07:180, 07:219, 08:03, 08:52, 08:78, 14:93, 15:13:01-15:13:03, 15:16:01:01-15:17:07, 15:23-15:24:02, 15:67, 15:87, 15:95, 15:157, 15:162, 15:168, 15:177, 15:196, 15:208, 15:216, 15:222, 15:230, 15:254, 15:268, 15:273, 15:356, 15:361-15:362, 15:396, 15:403, 15:408, 15:411, 15:418, 15:423-15:424, 15:442, 15:446, 15:462, 15:500, 15:516, 15:523, 15:532, 15:546Q, 15:550, 15:555, 18:67, 18:136, 18:194, 27:02:01:01-27:02:05, 27:30, 27:53, 27:57, 27:62, 27:65N, 27:75, 27:77, 27:83, 27:95, 27:102, 27:119, 27:126, 27:134, 27:157, 27:163, 27:171-27:172, 27:176N, 27:181, 27:188, 27:197, 27:203, 27:213, 27:232, 35:471, 37:34, 38:01:01:01-38:01:17, 38:05-38:07, 38:09-38:14, 38:16, 38:19-38:22, 38:24-38:28, 38:30-38:34N, 38:36-38:42, 38:51-38:61, 38:63, 38:65-38:71, 38:73, 38:77-38:78, 38:80N-38:81, 38:83N, 38:86, 38:161, 38:166, 38:168-38:169, 40:13, 40:19, 40:109, 40:117, 40:292, 40:340, 40:394, 41:46, 41:60, 44:06, 44:18, 44:25, 44:50:01, 44:50:03, 44:95, 48:18, 49:01:01:01-49:01:16, 49:03-49:14, 49:16-49:69, 51:01:01:01-51:24:05, 51:26-51:46, 51:48-51:53, 51:55-51:77, 51:79-51:324, 52:01:01:01-52:01:28, 52:01:30-52:19, 52:21:01-52:100, 53:01:01:01-53:02, 53:04-53:08:02, 53:10, 53:14-53:30, 53:32-53:35, 53:37, 53:39-53:54, 53:56-53:62, 54:12, 55:103, 56:21, 56:65, 57:01:01:01-57:11, 57:13-57:138, 58:01:01:01-58:02:02, 58:04-58:16:02, 58:18-58:29, 58:31N-58:63, 58:65-58:127, 59:01:01:01-59:07, 59:09-59:12				4				
B*07:149, 08:02, 08:117, 08:255, 13:01:01:01-13:04, 13:06-13:08, 13:10-13:23, 13:25-13:38, 13:40-13:66, 13:68-13:148, 15:36, 15:89:01-15:89:02, 15:115, 15:256, 15:339, 15:505, 18:09, 27:01, 27:142, 37:01:02, 37:10, 38:02:01:01-38:04, 38:08, 38:15, 38:18, 38:23, 38:29, 38:35, 38:43-38:50, 38:62, 38:64, 38:72, 38:74-38:76, 38:79, 38:82, 38:84-38:85, 38:160, 38:162-38:165N, 38:167, 40:47, 40:96, 40:110, 40:157, 40:201, 44:02:01:01-44:02:44, 44:02:46-44:05:02, 44:05:04, 44:07-44:08, 44:10-44:17, 44:19N-44:24, 44:26-44:45, 44:47-44:49, 44:51-44:74, 44:76-44:89, 44:91-44:94, 44:96-44:128:02, 44:130, 44:132-44:151, 44:153-44:169, 44:171N-44:195N, 44:197-44:245, 44:247-44:253, 44:255-44:256, 44:258-44:276, 44:278-44:344, 44:346, 44:437, 44:439-44:453, 44:456-44:496, 47:04, 49:02, 51:54, 51:78:01-51:78:02, 52:20, 53:09, 53:11-53:13, 53:31, 53:36, 53:38, 56:07, 59:08 44:456-44:492, 47:04, 49:02, 51:54, 51:78:01-51:78:02, 52:20, 53:09, 53:11-53:13, 53:31, 53:36, 53:38, 56:07, 59:08			3					
B*27:165, 37:14, 47:03							w	
Well No.	1	2	3	4	5	6	7	8

KIR HLA Ligand 2021 1N2

C*01:02:01:01-01:02:50, 01:02:52-01:13, 01:15:01-01:45, 01:47-01:153, 01:155-01:170, 01:172-01:199, 02:27:01-02:27:02, 02:65, 02:87, 02:115, 02:131, 03:02:01-03:06:02, 03:08-03:09, 03:11:01-03:11:02, 03:13:01-01-03:14, 03:16-03:28, 03:30-03:44, 03:46-03:98, 03:100-03:162, 03:164-03:267, 03:269-03:296:02, 03:298-03:394, 03:396N-03:398, 03:400-03:414, 03:416-03:437, 03:439-03:449N, 03:451-03:460, 03:462N-03:500, 03:502-03:542, 04:11, 04:29, 04:36, 04:55, 04:114, 04:172, 04:346, 04:383, 05:20, 06:11, 06:82, 06:147, 06:210, 06:217, 06:248, 06:252, 07:01:01:01-07:01:02:08, 07:01:04-07:02:77, 07:02:79-07:04:02:01, 07:04:03-07:06:04, 07:08, 07:10-07:33N, 07:35-07:75, 07:77-07:78:01, 07:79-07:114, 07:116-07:294, 07:296-07:314:03, 07:316-07:327, 07:329N-07:360, 07:362-07:405, 07:407-07:425, 07:427-07:458, 07:460-07:477, 07:479-07:558:01:02, 07:560-07:575:02, 07:577-07:597, 07:599-07:655, 07:657-07:672N, 07:674-07:715:02, 07:717-07:897, 08:01:01:01-08:09, 08:11-08:63, 08:65-08:145, 08:147-08:217, 12:02:01-12:02:22, 12:02:24-12:03:66, 12:06-12:08, 12:10:01-12:20, 12:22-12:32, 12:34, 12:36-12:40, 12:42Q-12:59, 12:61-12:71, 12:73-12:134, 12:136-12:145, 12:147-12:153, 12:155Q-12:229, 12:231-12:298, 12:300-12:329N, 14:02:01:01-14:11, 14:13-14:48, 14:50-14:107, 14:109-14:123, 15:07:01:01-15:07:01:02, 15:25, 15:43, 15:85, 15:144, 15:181, 16:01:01:01-16:01:39, 16:04:01:01-16:04:01:03, 16:04:03-16:04:05, 16:06-16:08, 16:10-16:11, 16:13-16:18, 16:20-16:24, 16:26-16:36, 16:38-16:45, 16:49-16:59, 16:61-16:62, 16:64-16:68, 16:71-16:73, 16:75-16:76, 16:78-16:83, 16:86-16:87, 16:92-16:93, 16:95-16:98, 16:100, 16:105-16:106, 16:109-16:114, 16:116-16:119, 16:122, 16:124-16:128, 16:130-16:131, 16:134-16:135, 16:137-16:139, 16:141-16:142, 16:146-16:152, 16:154, 16:157-16:162, 16:164-16:165, 16:168-16:175, 17:22	1								
C*01:14, 01:154, 02:02:01-02:02:03, 02:02:06-02:26:03, 02:28-02:40:02, 02:42-02:64, 02:66-02:86, 02:88-02:114, 02:116-02:130, 02:132-02:186, 02:188-02:197, 03:07:01:01-03:07:02, 03:10, 03:15, 03:29, 03:45, 03:163, 03:268, 03:297, 03:450, 03:461, 04:01:01:01-04:01:01:29, 04:01:01:31-04:01:131, 04:03:01:01-04:10, 04:12-04:20, 04:23-04:28, 04:30-04:35:02, 04:37-04:54:02, 04:56-04:113, 04:115N-04:171, 04:173N-04:249, 04:251-04:345, 04:347-04:382Q, 04:384-04:432, 05:01:01:01-05:01:56, 05:03-05:19, 05:21-05:244N, 06:02:01:01-06:02:01:47, 06:02:03-06:10, 06:12-06:81, 06:83-06:146, 06:148-06:209, 06:211:01:01N-06:216, 06:218-06:247, 06:249-06:251, 06:253-06:311, 07:07, 07:09, 07:76:01-07:76:02, 07:315, 07:328, 07:406, 07:559, 07:598, 07:656, 08:10, 12:04:01-12:05:02, 12:09, 12:21, 12:33, 12:41, 12:60, 12:72, 12:135, 12:146, 12:154, 14:12, 14:49, 14:108, 15:02:01:01-15:05:07, 15:05:09-15:06:03, 15:08:01-15:13:01:02, 15:15-15:19, 15:21-15:24, 15:26-15:42, 15:44:01-15:84Q, 15:86-15:143, 15:145N-15:180, 15:182-15:228, 16:02:01:01-16:02:18, 16:09, 16:12, 16:19, 16:25, 16:37, 16:46-16:48, 16:60, 16:63, 16:69-16:70, 16:74, 16:77N, 16:84, 16:88-16:91, 16:99, 16:101-16:104, 16:107-16:108, 16:115, 16:120-16:121, 16:123N, 16:132N-16:133, 16:136, 16:140, 16:143-16:145, 16:153, 16:155-16:156, 16:163, 16:166-16:167, 16:176, 17:01:01:02-17:21, 17:23-17:57, 18:01:01:01-18:14	2								
Well No.	1	2	3	4	5	6	7	8	Negative Control