

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

Olerup SSP® HLA-B*57:01

Product number:	101.572-12 – including <i>Taq</i> polymerase 101.572-12u – without <i>Taq</i> polymerase
Lot number:	27X
Expiry date:	2017-April-01
Number of tests:	12
Number of wells per test:	19+1
Storage - pre-aliquoted primers:	dark at -20°C
- PCR Master Mix:	-20°C
- Adhesive PCR seals	RT
- Product Insert	RT

This Product Description is only valid for Lot No. 27X.

Complete product documentation consists of generic Instructions for Use (IFU), lot specific Product Insert, Worksheet and Certificate.

CHANGES COMPARED TO THE PREVIOUS OLERUP SSP® HLA-B*57:01 LOT (10V).

The format of the Product Insert and Worksheet have been changed.

The HLA-B*57:01 primer set, specificity and interpretation tables have been updated for the HLA-B alleles described since the previous *Olerup SSP®* HLA-B*57:01 lot was made (**Lot No. 10V**). The kit design is based on IMGT/HLA database 3.17.0.

As of lot series V, the Specificity Table is included in the lot-specific Product Insert, and the Interpretation Table is included in the Worksheet.

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

The primers of the wells detailed below have been exchanged, added or modified compared to the previous lot.

Well	5'-primer	3'-primer	rationale
7	-	Added	3'-primer exchanged for the B*57:14 allele.
10	Moved	Moved	Primer pair moved to well 18, 3'-primer moved to well 17.
16	Added	Added	Negative Control moved to well 20, primer pair added for the B*57:67 allele.
17	New	New, added	New primer pair added for the B*57:68 allele, 3'-primer added from well 10.
18	Added	Added	Primer pair added from well 10
19	New	New	New Primer pair added for the B*57:72 allele.
20	-	-	Updated Negative Control added from well 16.

Change in revision R01 compared to R00:

1. Primer mix 17 has tendencies of unspecific amplifications. A footnote has been added in the Specificity Table.

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

Well **20** contains Negative Control primer pairs, that will amplify more than 95% of the *Olerup SSP*® HLA Class I, DRB, DQB1, DPB1 and DQA1 amplicons as well as all the amplicons generated by the control primer pairs matching the human growth hormone gene.

HLA-specific PCR product sizes range from 75 to 200 base pairs.
The PCR product generated by the positive control primer pair is 430 base pairs.

Length of PCR product	105	200	105	80	75	80	85
5'-primer¹	164	340	440	45	45	43	36
	5'-CAC ^{3'}	5'-Agg ^{3'}	5'-TTA ^{3'}	5'-Tgg ^{3'}	5'-Tgg ^{3'}	5'-Tgg ^{3'}	5'-TAC ^{3'}
							36
							5'-TAT ^{3'}
3'-primer²	231	2nd I	507	59	58	57	47
	5'-TgC ^{3'}	5'-AAA ^{3'}	5'-TTg ^{3'}	5'-CTC ^{3'}	5'-ggC ^{3'}	5'-CTC ^{3'}	5'-ACA ^{3'}
							48
							5'-gCA ^{3'}
							48
							5'-gCC ^{3'}
							52
							5'-TgT ^{3'}
A*	+	+	+				
B*	+	+	+				
C*	+	+	+				
DRB1				+	+		
DRB3				+	+		
DRB5				+			
DQB1					+		
DPB1						+	
DQA1							+

¹The nucleotide position for HLA class I genes and the codon for HLA class II genes, in the 2nd or 3rd exon, matching the specificity-determining 3'-end of the primer is given. Nucleotide and codon numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

²The nucleotide position for HLA class I genes and the codon for HLA class II genes, in the 2nd or 3rd exon or the 2nd intron, matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide and codon numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

PRODUCT DESCRIPTION

HLA-B*57:01 SSP subtyping

CONTENT

The primer set contains 5'- and 3'-primers for identifying the B*57:01:01 to 57:01:19 alleles.

PLATE LAYOUT

Each test consists of 20 PCR reactions in a 24 well cut PCR plate. Wells 21 to 24 are empty.

1	2	3	4	5	6	7	8
9	10	11	12	13	14	15	16
17	18	19	NC	empty	empty	empty	empty

The 24 well cut PCR plate is marked with 'B*57:01' in silver/gray ink.

Well No. 1 is marked with the Lot No. '27X'.

Wells 1 to 19 – HLA-B*57:01 primers.

Well 20 – Negative Control (NC).

A faint row of numbers is seen between wells 1 and 2 or wells 7 and 8 of the PCR trays. These stem from the manufacture of the trays, and should be disregarded.

The PCR plates are heat-sealed with a PCR-compatible foil.

Please note: When removing each 24 well PCR plate, make sure that the remaining plates stay sealed. Use a scalpel or a similar instrument to carefully cut the foil between the plates.

INTERPRETATION

Due to the sharing of sequence motifs between HLA-B alleles some non-HLA-B*57 alleles will be amplified by primer mixes 1 to 17. In addition, a few HLA-A and HLA-C alleles will be amplified by primer mixes 5, 7, 10, 11 and 13 to 17. For further details see Specificity Table.

UNIQUELY IDENTIFIED ALLELES

HLA-B*57:01 will give rise to a unique amplification pattern by the primers in the HLA-B*57:01 kit¹.

The HLA-B*57:01 typing kit cannot distinguish the silent mutations in the B*57:01:01 to B*57:01:19 alleles.

¹HLA-B alleles listed on the IMGT/HLA web page 2014-July-25, release 3.17.0, www.ebi.ac.uk/imgt/hla.

²Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information
SPECIFICITY TABLE

HLA-B*57:01 SSP subtyping

Specificities and sizes of the PCR products of the 19+1 primer mixes used for HLA-B*57:01 SSP subtyping

Primer Mix	Size of spec. PCR product ¹	Size of control band ²	Amplified HLA-B*57:01 alleles	Other amplified HLA-B alleles ^{3,4}
1⁵	90 bp	800 bp	*57:01:01-57:01:19	*57:15, 57:17-57:19, 57:21-57:35, 57:37-57:44, 57:46-57:50, 57:52-57:61, 57:63-57:68, 57:70-57:72 58:36
2	220 bp	800 bp	*57:01:01-57:01:19	*57:03:01-57:03:02, 57:06-57:08, 57:10, 57:14-57:18, 57:20-57:23, 57:25-57:27, 57:29, 57:31-57:41, 57:43-57:51, 57:53-57:60, 57:62, 57:64-57:72, 35:208, 40:30, 40:34, 44:153, 55:14, 58:14
3⁵	95 bp 170 bp 215 bp	800 bp		*57:04, 57:32, 44:153 *57:06, 57:18 *57:27
4⁵	100 bp	1070 bp		*57:15, 57:20, 57:29, 57:62, 07:120, 15:214, 18:81, 35:250, 40:150, 51:165
5⁵	90 bp 165 bp 200 bp 245 bp	800 bp		*57:16, 57:34, 58:54 *57:23 *57:46, 57:64 *57:07, 57:26, 44:153, 55:14, C*06:72
6⁵	90 bp	1070 bp		*57:02:01-57:03:02, 57:07, 57:09, 57:12, 57:17, 57:39, 57:42, 57:46, 57:57, 57:63, 57:66, 57:70, 40:30, 40:34 *57:08, 57:65
7⁵	205 bp 100 bp 140 bp 185 bp 215 bp 250 bp	1070 bp		*57:53 *57:13, 57:31, 07:227, 40:30, 40:34, C*06:72 *57:50 *57:09, 57:24 *57:13, 57:14 57:25, 07:227, 40:30, 40:34
8	185 bp	1070 bp		*57:10, 57:44, 07:219
9⁵	110 bp 150 bp	800 bp		*57:33 *57:21, 57:40, 14:20, 35:127
10	205 bp	1070 bp		*57:13, 57:22, 57:57, 57:63, 07:227, 55:14, C*06:72^w *57:43
11⁵	240 bp 100 bp	1070 bp	*57:01:01-57:01:19	*57:06, 57:08, 57:10, 57:13-57:16, 57:18-57:27, 57:29-57:31, 57:33-57:38, 57:40 ^w , 57:41, 57:43-57:45, 57:47-57:52, 57:54-57:56,

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

			57:58-57:62, 57:64-57:65, 57:67-57:69, 57:71-57:72, 35:208, 55:14, 58:14, C*06:72
12^{5,6}	75 bp 100 bp 140 bp	1070 bp	*57:35 *57:36 *57:08, 57:38, 57:60, 15:87
13⁵	105 bp 155 bp	1070 bp	*57:45, 57:51, 57:69, 57:71, A*02:285 *57:58, 07:219, 58:12, A*02:42, A*02:310 *57:49
14⁵	200 bp 100 bp	1070 bp	*57:59, 15:116, 40:63, 40:92, 44:169, 44:182, 46:43, C*03:129, 03:234, C*17:07 *57:47, 57:54, 15:33, 15:248, 49:22, 51:126, C*03:87, C*05:27, C*05:39
15	145 bp 185 bp 295 bp	1070 bp	*57:56 *57:48, A*33:12 *57:55
16⁵	65 bp	1070 bp	*57:67, 58:14, C*06:72
17^{5,7}	90 bp 180 bp	1070 bp	*57:04, 57:41, 44:153, C*06:72 *57:68
18	170 bp	1070 bp	*57:37
19	215 bp	1070 bp	*57:72
20⁸	-	-	Negative Control

¹ Alleles are assigned by the presence of specific PCR product(s). However, the sizes of the specific PCR products may be helpful in the interpretation of HLA-B*57:01 SSP typings.

When the primers in a primer mix can give rise to HLA-specific PCR products of more than one length this is indicated if the size difference is more than 20 base pairs. Size differences of 20 base pairs or less are not given. For high resolution SSP kits, the alleles listed are specified according to amplicon length.

Nonspecific amplifications, i.e. a ladder or a smear of bands, may sometimes be seen. GC-rich primers have a higher tendency of giving rise to nonspecific amplifications than other primers.

PCR fragments longer than the control bands may sometimes be observed. Such bands should be disregarded and do not influence the interpretation of the SSP typings.

PCR fragments migrating faster than the control bands, but slower than a 400 bp fragment may be seen in some gel read-outs. Such bands can be disregarded and do not influence the interpretation of the SSP typings.

Some primers may give rise to primer oligomer artifacts. Sometimes this phenomenon is an inherent feature of the primer pair(s) of a primer mix. More often it is due to other factors such as too low amount of DNA in the PCR reactions, taking too long time in setting up the PCR reactions, working at elevated room temperature or using thermal cyclers that are not pre-heated.

² The internal positive control primer pairs amplify segments of the human growth hormone gene. The internal positive control bands are 1070 or 800 base pairs respectively, well distribution as outlined in the table. Well number 1 contains the shorter, 800 bp, internal positive control band. The well distribution of the internal controls can help in orientation of the kit on gel photo, as well as allow for kit identification. In the presence of a specific amplification the intensity of the control band often decreases.

³ For several HLA Class I alleles 1st and/or 4th exon(s) and beyond, as well as intron nucleotide sequences, are not available. In these instances it is not known whether some of the primers of the SSP sets are completely matched with the target sequences or not. Assumption is made that unknown sequences in these regions are conserved within allelic groups.

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: 27X

Lot-specific information

⁴Due to the sharing of sequence motifs between HLA-B alleles some non-HLA-B*57 alleles will be amplified by primer mixes 1 to 17. In addition, a few HLA-A and HLA-C alleles will be amplified by primer mixes 5, 7, 10, 11 and 13 to 17.

⁵HLA-specific PCR products shorter than 125 base pairs have a lower intensity and are less sharp than longer PCR products.

⁶Primer mix 12 has a tendency to giving rise to primer oligomer formation.

⁷Primer mix 17 has tendencies of unspecific amplifications.

⁸Primer mix 20 contains a negative control, which will amplify more than 95% of HLA amplicons as well as the amplicons generated by the control primer pairs matching the human growth hormone gene. HLA-specific PCR product sizes range from 75 to 200 base pairs and the PCR product generated by the HGH positive control primer pair is 430 base pairs.

‘w’, may be weakly amplified.

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: 27X

Lot-specific information

PRIMER SPECIFICATION

Well No.	1	2	3	4	5	6	7	8	9	10	11	12
Length of spec.	90	220	95	100	90	90	100	185	110	205	100	75
PCR product			170		165	205	140		150	240		100
			215		200		185					140
					245		215					
							250					
Length of int.	800	800	800	1070	800	1070	1070	1070	800	1070	1070	1070
pos. control ¹												
5'-primer(s) ²	209	362	362	209	130	320	362	103	352	362	362	209
	5'-ggC 3'	5'-ggT 3'	5'-ggT 3'	5'-ggC 3'	5'-AgT 3'	5'-CCC 3'	5'-ggT 3'	5'-CCT 3'	5'-ACg 3'	5'-ggT 3'	5'-ggT 3'	5'-ggC 3'
				704	200	362		122	353			
				5'-TgT 3'	5'-TCg 3'	5'-ggT 3'		5'-CCT 3'	5'-CAA 3'			
					209				392			
					5'-ggA 3'				5'-CgA 3'			
					362							
					5'-ggT 3'							
3'-primer(s) ³	256	539	418	259	256	2 nd I	421	256	463	527	419	244
	5'-CCC 3'	5'-TCA 3'	5'-gTC 3'	5'-CTT 3'	5'-CCC 3'	5'-TCg 3'	5'-ggT 3'	5'-CCC 3'	5'-gCT 3'	5'-CCT 3'	5'-Cgg 3'	5'-CTT 3'
			481	263	523	412	463			559	419	268
			5'-gTA 3'	5'-gTT 3'	5'-ACA 3'	5'-gTT 3'	5'-gCg 3'			5'-Cgg 3'	5'-CAG 3'	5'-gTg 3'
			500	271	559	521	505					302
			5'-ggA 3'	5'-CAC 3'	5'-CgT 3'	5'-ggA 3'	5'-gCC 3'					5'-ggg 3'
			537	774	572		538					302
			5'-Agg 3'	5'-ggT 3'	5'-gCg 3'		5'-gTC 3'					5'-ggC 3'
							559					320
							5'-CTC 3'					5'-Agg 3'
							583					
							5'-gTg 3'					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12

Well No.	13	14	15	16	17	18	19
Length of spec.	105	100	145	65	90	170	215
PCR product	155	125	185		180		
	200		295				
Length of int.	1070	1070	1070	1070	1070	1070	1070
pos. control ¹							
5'-primer(s) ²	97	467	106	362	362	878	362
	5'-TCg 3'	5'-CTg 3'	5'-CCA 3'	5'-ggT 3'	5'-ggT 3'	5'-gCA 3'	5'-ggT 3'
	142	485	395				
	5'-TCT 3'	5'-CAA 3'	5'-gCC 3'				
	193	499	757				
	5'-CCA 3'	5'-TCT 3'	5'-CCA 3'				
3'-primer(s) ³	256	559	212	387	409	916	534
	5'-CCC 3'	5'-CAG 3'	5'-ggT 3'	5'-TCg 3'	5'-ATA 3'	5'-gAC 3'	5'-Tgg 3'
			539		502		
			5'-TCA 3'		5'-CTT 3'		
			916				
			5'-gAC 3'				
Well No.	13	14	15	16	17	18	19

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

¹The internal positive control primer pairs amplify segments of the human growth hormone gene. The internal positive control bands are 1070 or 800 base pairs respectively, well distribution as outlined in the table. Well number 1 contains the shorter, 800 bp, internal positive control band. The well distribution of the internal controls can help in orientation of the kit on gel photo, as well as allow for kit identification. In the presence of a specific amplification the intensity of the control band often decreases.

²The nucleotide position matching the specificity-determining 3'-end of the primer is given. Nucleotide numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

³The nucleotide position matching the specificity-determining 3'-end of the primer is given in the anti-sense direction. Nucleotide numbering as on the www.ebi.ac.uk/imgt/hla web site. The sequence of the 3 terminal nucleotides of the primer is given.

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: 27X

Lot-specific information

CELL LINE VALIDATION SHEET																					
HLA-B*57:01 SSP typing kit																					
					Prod. No.:	Well ²															
						1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
						201327601	201327602	201327603	201327604	201327605	201327606	201441007	201327608	201327609	201441010	201327611	201327612	201327613	201327614	201327615	201441016
	IHCW cell line ¹		B*																		
1	9001	SA	*07:02		-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
2	9280	LK707	*52:01		*73:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
3	9011	E4181324	*52:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
4	9275	GU373	*15:10		*53:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
5	9009	KAS011	*37:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
6	9353	SM	*39:01		*51:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
7	9020	QBL	*18:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
8	9025	DEU	*35:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
9	9026	YAR	*38:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
10	9107	LKT3	*54:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
11	9051	PITOUT	*44:03			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
12	9052	DBB	*57:01			+	+	-	-	-	-	-	-	-	-	+	-	-	-	-	-
13	9025	JESTHOM	*27:05			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
14	9071	OLGA	*15:01		*15:20	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
15	9075	DKB	*40:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
16	9037	SWEIG007	*40:02			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
17	9282	CTM3953540	*08:01		*55:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
18	9257	32367	*14:01		*56:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
19	9038	BM16	*18:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
20	9059	SLE005	*40:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
21	9064	AMALA	*15:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
22	9056	KOSE	*35:03			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
23	9124	IHL	*40:02		*56:02	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
24	9035	JBUSH	*38:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
25	9049	IBW9	*14:02			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
26	9285	WT49	*58:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
27	9191	CH1007	*07:05		*51:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
28	9320	BEL5GB	*44:02		*44:03	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
29	9050	MOU	*44:03			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
30	9021	RSH	*42:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
31	9019	DUCAF	*18:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
32	9297	HAG	*41:02			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
33	9098	MT14B	*40:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
34	9104	DHIF	*38:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
35	9302	SSTO	*44:02			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
36	9024	KT17	*15:01		*35:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
37	9065	HHKB	*07:02			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
38	9099	LZL	*15:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
39	9315	CML	*08:01		*27:05	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
40	9134	WHONP199	*13:02		*46:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
41	9055	H0301	*14:02			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
42	9066	TAB089	*46:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
43	9076	T7526	*46:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
44	9057	TEM	*38:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
45	9239	SHJO	*42:01		*50:01	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
46	9013	SCHU	*07:02			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
47	9045	TUBO	*51:01			-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-
48	9303	TER-ND	*35:01		*44:03	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-	-

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

CELL LINE VALIDATION SHEET				
HLA-B*57:01 SSP typing kit				
				Well ²
				17 18 19
				Prod. No.: 201441017 201441018 201441019
IHWC cell line ¹		B*		
1	9001 SA	*07:02		- - -
2	9280 LK707	*52:01	*73:01	- - -
3	9011 E4181324	*52:01		- - -
4	9275 GU373	*15:10	*53:01	- - -
5	9009 KAS011	*37:01		- - -
6	9353 SM	*39:01	*51:01	- - -
7	9020 QBL	*18:01		- - -
8	9025 DEU	*35:01		- - -
9	9026 YAR	*38:01		- - -
10	9107 LKT3	*54:01		- - -
11	9051 PITOUT	*44:03		- - -
12	9052 DBB	*57:01		- - -
13	9025 JESTHOM	*27:05		- - -
14	9071 OLGA	*15:01	*15:20	- - -
15	9075 DKB	*40:01		- - -
16	9037 SWEIG007	*40:02		- - -
17	9282 CTM3953540	*08:01	*55:01	- - -
18	9257 32367	*14:01	*56:01	- - -
19	9038 BM16	*18:01		- - -
20	9059 SLE005	*40:01		- - -
21	9064 AMALA	*15:01		- - -
22	9056 KOSE	*35:03		- - -
23	9124 IHL	*40:02	*56:02	- - -
24	9035 JBUSH	*38:01		- - -
25	9049 IBW9	*14:02		- - -
26	9285 WT49	*58:01		- - -
27	9191 CH1007	*07:05	*51:01	- - -
28	9320 BEL5GB	*44:02	*44:03	- - -
29	9050 MOU	*44:03		- - -
30	9021 RSH	*42:01		- - -
31	9019 DUCAF	*18:01		- - -
32	9297 HAG	*41:02		- - -
33	9098 MT14B	*40:01		- - -
34	9104 DHIF	*38:01		- - -
35	9302 SСТО	*44:02		- - -
36	9024 KT17	*15:01	*35:01	- - -
37	9065 HHKB	*07:02		- - -
38	9099 LZL	*15:01		- - -
39	9315 CML	*08:01	*27:05	- - -
40	9134 WHONP199	*13:02	*46:01	- - -
41	9055 H0301	*14:02		- - -
42	9066 TAB089	*46:01		- - -
43	9076 T7526	*46:01		- - -
44	9057 TEM	*38:01		- - -
45	9239 SHJO	*42:01	*50:01	- - -
46	9013 SCHU	*07:02		- - -
47	9045 TUBO	*51:01		- - -
48	9303 TER-ND	*35:01	*44:03	- - -

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: 27X

Lot-specific information

¹The provided cell line HLA specificities are retrieved from the <http://www.ihwg.org/hla> web site. The specificity of an individual cell line may thus be subject to change.

²The specificity of each primer solution in the kit has been tested against 48 well characterized cell line DNAs and where applicable, additional cell line DNAs.

No DNAs carrying the alleles to be amplified by primer solutions 3 to 5, 7 to 10 and 12 to 19 were available. The specificities of the primers in primer solutions 3 to 5, 7, 8, 10 and 12 to 17 were tested by separately adding additional 5'-primers, respectively 3'-primers. In primer solutions 9 and 18 it was only possible to test the 3'-primer, the 5'-primers were not possible to test.

In primer solution 19 it was only possible to test the 5'-primer, the 3'-primers were not possible to test.

In primer solutions 3 to 7, 10 to 12 and 15 one to three 3'-primers were not possible to test, and in primer solutions 4, 5, 8, 14 and 15 one to three 5'-primers were not possible to test.

101.572-12 – including *Taq* pol., IFU-01

101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

101.572-12 – including *Taq* pol., IFU-01

101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

101.572-12 – including *Taq* pol., IFU-01

101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

101.572-12 – including *Taq* pol., IFU-01
101.572-12u – without *Taq* pol., IFU-02

Visit www.olerup-ssp.com for
“Instructions for Use” (IFU)

Lot No.: **27X**

Lot-specific information

ADDRESSES:

Manufacturer:

Olerup SSP AB, Franzengatan 5, SE-112 51 Stockholm, Sweden.

Tel: +46-8-717 88 27

Fax: +46-8-717 88 18

E-mail: info-ssp@olerup.com

Web page: <http://www.olerup-ssp.com>

Distributed by:

Olerup GmbH, Löwengasse 47 / 6, AT-1030 Vienna, Austria.

Tel: +43-1-710 15 00

Fax: +43-1-710 15 00 10

E-mail: support-at@olerup.com

Web page: <http://www.olerup.com>

Olerup Inc., 901 S. Bolmar St., Suite R, West Chester, PA 19382

Tel: 1-877-OLERUP1

Fax: 610-344-7989

E-mail: info.us@olerup.com

Web page: <http://www.olerup.com>

For information on *Olerup* SSP distributors worldwide, contact **Olerup GmbH**.