

Olerup SSP[®] DRB5

Product number: 101.123-24/06 – including *Taq* pol.
101.123-24u/06u – without *Taq* pol.
Lot number: 49R
Expiry date: 2015-May-01
Number of tests: 24 test – Product No. 101.123-24/24u
6 tests – Product No. 101.123-06/06u
Number of wells per test: 16

CHANGES COMPARED TO THE PREVIOUS DRB5 LOT (35N):

The primers of the wells detailed below have been exchanged, added or modified compared to the previous lot.

Well	5'-primer	3'-primer	rationale
4	Added	Added	Primer pair added for the DRB5*02:06 allele.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

DRB5*01:01 to DRB5*01:14 and DRB5*02:02 to DRB5*02:06, i.e. all the currently recognized DRB5 alleles, give rise to unique amplification patterns, www.ebi.ac.uk/imgt/hla, 2012-July-12, release 3.9.0.

RESOLUTION IN DRB5 HOMOZYGOTES:

Good.

INFLUENCE ON THE INTERPRETATION OF DRB5 SUBTYPINGS BY NON-DRB5 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.